Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

BX 11
BX 11S

DE

EN

ES

FR

Enjoy mobility.

Inhalt

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Contents

1. Preliminary remark	60	4.6.1. Seat sag	77
1.1. Signs and Symbols	61	4.6.2. Back sag	78
2. Important safety instructions	62	4.7. Brakes	78
2.1. General safety instructions	62	4.7.1. Actuation of the locking brake	79
2.2. Tipping Hazard	63	4.7.2. Operation with brake lever extension (optional)	79
2.3. Safety in vehicles	64	4.7.3. Drum brake (optional)	80
2.4. Participation in road traffic	64	4.8. Wheels and tyres	81
2.5. Brakes	64	4.8.1. Thru axle	81
2.6. Purpose	65	4.8.2. Drive wheels	81
2.7. Indications	65	4.8.3. Castor wheels	81
2.8. Contraindications	65	4.9. Anti-tip rollers and tipping aid (optional)	82
2.9. Declaration of conformity	65	4.9.1. Tipping aid (optional)	82
2.10. Responsibility	66	4.9.2. Anti-tip rollers (optional)	82
2.11. Useful life	66	4.9.3. Lap belt	83
3. Product and delivery overview	67	4.10. Wheel camber	83
3.1. Delivery inspection	67	4.11. Spoke protector (optional)	83
3.2. Scope of delivery	67	4.12. Attach pockets (optional)	83
3.3. Type plate and serial number	68	5. Accessories	84
3.4. Overview	69	5.1. Legrest horizontally adjustable	84
4. Assembly / Adaptations	71	5.2. Amputation support adjustable	84
4.1. Fold and unfold	71	5.3. Tipping aid	84
4.2. Legrests	72	5.4. Anti-tip rollers	84
4.2.1. Foot plate foldable	72	5.5. Angle-adjustable back with adjustable sliding tubes	84
4.2.2. Legrests can be swivelled down	73	5.6. Tyre casing	84
4.2.3. Horizontally adjustable leg supports (optional)	74	5.7. Spoke protector	84
4.3. Amputation leg support (optional)	75	5.8. Drum brake	84
4.3.1. Active side panels	75	5.9. Brake lever extension	85
4.3.2. Plug-in side panels with height adjustment (optional)	76	5.10. Plug-in side panels with height adjustment	85
4.4. Backrest angle	76	5.11. Lap belt	85
4.4.1. Backrest (standard)	76	5.12. Cane holder	85
4.4.2. Angle-adjustable Backrest (optional)	76	6. Use	85
4.5. Push handle height	77	6.1. Entering and exiting from the side	85
4.5.1. Push handles (standard)	77	6.2. Entering and exiting from the front	86
4.5.2. Push handles (optional)	77	6.3. Climbing over stairs or a high step	86
4.6. Fitting seat and back	77	6.4. Driving with the wheelchair	87
		6.5. Check before setting off	88

6.5.1.	Checking the brakes	88
6.5.2.	Checking the tyres	88
7.	Technical data	89
8.	Care instructions for the user	92
8.1.	Cleaning and disinfection	92
8.2.	Disinfection	92
9.	Maintenance note	93
9.1.	Maintenance advice for the trade	94
9.2.	Adjustment work for the trade	94
9.3.	Centre of gravity, seat angle and Seat height	94
9.3.1.	Adjusting the drive wheel	94
9.3.2.	Height of the front wheel	95
9.3.3.	Angle of the steering axle	95
9.3.4.	Wheelbase	95
9.4.	Adjusting the locking brakes	96
9.5.	Adjusting the drum board	97
9.6.	Adjusting the anti-tip rollers	97
9.7.	Changing the tyre casing and tyre tube	98
10.	Repair	104
11.	Transport	104
11.1.	Transport of the wheelchair	104
11.2.	Carriage in motor vehicles	104
12.	Passing on the wheelchair	104
13.	Storage / Shipping	105
14.	Disposal	105
15.	Warranty	106

1. Preliminary remark

Dear user,

You have chosen a high-quality wheelchair from Bischoff & Bischoff. We thank you for your trust.

This instruction manual contains all the necessary information for using the BX 11 wheelchair in the version with a full-length footplate and the BX 11S wheelchair with swing-off legrests.

The wheelchair has been designed to offer you many advantages and to meet your needs. The BX 11 / BX 11S can be excellently adapted to your individual body dimensions thanks to the various insertion options and the optimum adaptability with regard to seat width, seat height, seat depth, back height, etc. The BX 11 / BX 11S can also be adapted to your individual body dimensions.

The wheelchair is designed for both indoor and outdoor use for active users who are dependent on a wheelchair. Before using your new wheelchair for the first time, please read and follow the operating instructions carefully. This is an integral and necessary part of the wheelchair. Keep the operating instructions within easy reach and give them to anyone who passes on the wheelchair.

For users with visual impairments, this document is accessible as a PDF file from our website www.bischoff-bischoff.com.

Notice!

Despite careful research and editing, incorrect information may have crept into our operating instructions. We reserve the right to make technical changes. Illustrations may contain accessories. The current version of the operating instructions as well as information on products and possible recalls can be found in the download area on our website: www.bischoff-bischoff.com

1.1. Signs and Symbols

Important! Indicates particularly useful information in the respective context.

Attention! Indicates particularly safety-relevant information. Observe operating instructions!

2. Important safety instructions

2.1. General safety instructions

- To avoid falls and dangerous situations, first practice using your new wheelchair on level, manageable terrain. An accompanying person is recommended.
- Before using the wheelchair, check that all attached parts are properly fastened.
- Only lift the wheelchair by firmly mounted parts (see chapter 6.3).
- Ensure that the quick-release axles are correctly seated in the axle mounts, otherwise there is a risk of injury.
- When shifting balance due to body movements or loading the wheelchair, the risk of tipping to all sides increases.
- Do not use the wheelchair to transport more than one person or loads. Observe the intended use (see chapter 2.6)!
- When adjusting the wheelchair, there is a risk of trapping body parts!
- Keep clothing and body parts away from gaps between moving parts!
- When negotiating obstacles (steps, etc.), have an attendant present. Use the tipping aid (see chapter 4.9).
- Only climb stairs and larger obstacles with the help of at least two escorts (see chapter 6.3). Use facilities such as ramps, lifts or stair-climbing aids if available (Fig. A).
- Do not drive down a kerb or other steps (Fig. B).
- When driving on a slope/gradient, note that there is a danger of tipping over from just a few degrees. The use of anti-tip rollers (see chap. 4.9.) is recommended (Fig. C).
- Avoid collisions of any kind!
- Only ride on level, firm ground.
- When getting in or out of the wheelchair, apply both parking brakes (see chapter 4.7).
- Observe the road traffic regulations in road traffic.
- Check the passive lighting (reflectors) of your wheelchair before each journey to ensure that it is intact and visible to other road users.

- Use the parking brake only for parking.
- The seat materials are resistant to flammability according to EN 1021-2 and 8191-2. Nevertheless, keep cigarettes and other possible sources of fire away from your product.
- Protect the wheelchair from sunlight to avoid burns from heated components.
- Protect the wheelchair from the cold.
- When transferring between wheelchair and bed (see chap. 6.1, 6.2) always
 - apply both parking brakes.
 - If possible, swing the side part of the entry/exit side backwards.
 - swivel the foot plates up or away.
- Reporting of incidents:

If you, as the operator, user or their relatives, discover possible defects or functional limitations, please contact an authorised specialist dealer immediately. Operators, users or their relatives should inform their specialist dealer who provided the product of any suspected serious incidents that have had or may have had, directly or indirectly, any of the following consequences and which affect them. You can also report this to the competent federal authority.

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of the state of health of a patient, user or other person,
- a serious threat to public health.

2.2. Tipping Hazard

The anti-tip castors on both sides (see chapter 4.9) prevent the wheelchair from tipping backwards as far as possible. Please note that driving with anti-tip castors is not recommended on uneven terrain, lifting platform and ramps.

Ensure sufficient distance upwards, downwards and to the sides.

2.3. Safety in vehicles

The BX 11 / BX 11S is not suitable as a vehicle seat. It must not be used as such under any circumstances. It does not meet the requirements of ISO 7176-19. Failure to do so may result in serious injury and death in the event of an accident.

You can tell that your wheelchair is not approved as a seat in motor vehicles for the transport of mobility-impaired persons by the symbol on the type plate, (Fig. D).

2.4. Participation in road traffic

The wheelchair is designed for indoor and outdoor use. Please note that you are participating in public road traffic and must comply with the regulations of the Road Traffic Act. Do not endanger other participants by driving recklessly, this applies especially on pavements.

2.5. Brakes

Service brake:

When driving, brake the wheelchair using the handrims. Please note that your hands may get hot.

Parking brake/parking:

As soon as the wheelchair is stationary, push both brake levers forward. The wheelchair stands safely braked if it does not can no longer be pushed away and both brake levers are pushed forward as far as they will go. Pull the lever back towards you to release the brake (see chapter 4.7.1).

The function of the parking and service brake complies with the provisions of the StVZO.

Drum brake:

The wheelchair can optionally be equipped with a drum brake that can only be operated by an attendant via the brake levers on the push handles.

2.6. Purpose

The wheelchair is designed exclusively for increasing the mobility and for transporting disabled adults with the indicated indications (see chapter 2.6). The maximum user weight is 135 kg.

To be noted:

We can only provide a warranty if the product is used under the specified conditions and for the intended purposes.

2.7. Indications

The provision of a wheelchair is suitable for adults who are unable to walk or have a severe walking disability,

- e.g. through:
 - Paralysis
 - Limb loss
 - Limb defect/deformity
 - Joint contractures (not on both arms)
 - Joint damage (not to both arms)
 - Other illnesses

2.8. Contraindications

- Wheelchair provision is unsuitable for adult people with:
 - Incapacity to sit
 - Joint damage / joint contractures in both arms
 - Perceptual disorders
 - Severe balance disorders
 - Reduced and insufficient vision
 - Severe limitations of cognitive abilities

2.9. Declaration of conformity

We Bischoff & Bischoff GmbH declare under our sole responsibility that the wheelchair BX 11 / BX 11S complies with all requirements of MDR 2017/745 that are applicable.

2.10. Responsibility

We shall only provide a warranty if

- the product is used under the specified conditions and for the intended purpose,
- changes, extensions, repairs and maintenance work are only carried out by such persons who have been authorised by us to do so and the changes or extensions have been approved by Bischoff & Bischoff and
- the wheelchair is used in compliance with all instructions for use.

2.11. Useful life

The expected product life is up to 5 years. This is only possible if the product is used as intended and the safety, care and maintenance instructions in this user manual are followed.

3. Product and delivery overview

3.1. Delivery inspection

All Bischoff und Bischoff GmbH products are subjected to a proper final inspection in our company and provided with the CE mark (see chapter 3.3).

The wheelchair is delivered in a special box in factory settings. After unpacking, keep the carton if possible. It is used for possible later storage or return of the product.

Delivery and instruction are usually carried out by qualified medical supply retailers.

In case of shipment by rail or forwarding agent, the goods must be checked immediately for transport damage in the presence of the deliverer. Check the contents for completeness (see chapter 3.2) and integrity. In case of irregularities or damage, contact our customer service.

3.2. Scope of delivery

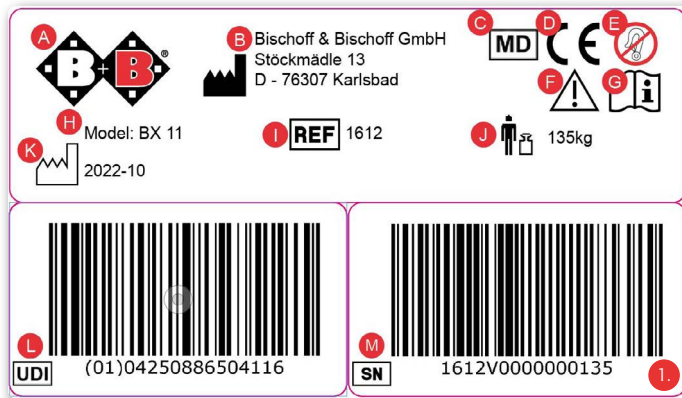
To avoid damage during transport, plug-in attachments are supplied as standard.

After receiving the goods, please immediately check the contents for completeness. The contents consist of:

- Repackaging
- Wheelchair (pre-assembled)
- Back cushion
- 2 leg rests (on the BX 11S)
- This instruction manual
- Accessories, if applicable

3.3. Type plate and serial number

The type plate (Fig. 01) and the serial number are located at the bottom of the cross structure.

- A** Manufacturer's logo
- B** Manufacturer's specification
- C** MD Medical device
- D** CE mark
- E** Wheelchair not crash tested
- F** Attention! - Observe instructions for use
- G** Important! - Read instructions for use
- H** Model designation
- I** Model number
- J** Max. user weight
- K** Production date
- L** UDI number
- M** Serial number

3.4. Overview

BX 11S (Abb. 2)

- A** Push handles
- B** Back upholstery
- C** Steering wheels
- D** Active side part
- E** Seat cushion
- F** Drive wheel
- G** Thru axle
- H** Parking brake
- I** Push rims
- J** Swing-away legrests

BX 11 (Abb. 03)

- A** Push handles
- B** Back upholstery
- C** Steering wheels
- D** Active side part
- E** Seat cushion
- F** Drive wheel
- G** Thru axle
- H** Parking brake
- I** Push rims
- J** Continuous foot plate

4. Assembly / Adaptations

4.1. Fold and unfold

Your new Bischoff & Bischoff wheelchair is delivered assembled and folded in a Bischoff & Bischoff original cardboard box. In order to avoid transport damage, plug-in attachments are supplied separately.

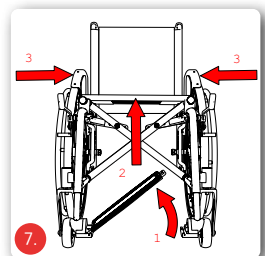
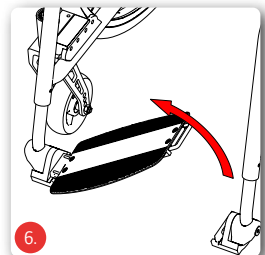
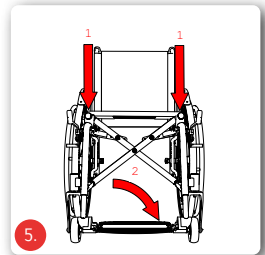
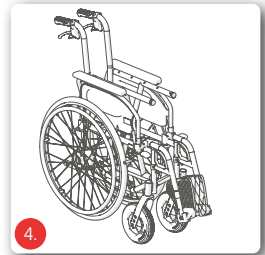
Unfold

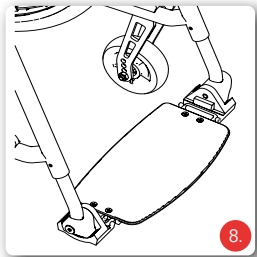
- Position the folding wheelchair next to you. (Fig. 04)
- Tilt the folding wheelchair towards you so that one drive wheel is unloaded.
- Push the seat tubes apart and hold them with the flat of your hand. Press down until it is locked in the holder on the frame (Fig. 05).
- Fold down the foot plate(s) (Fig. 05).

Wrinkles

- Position the folding wheelchair next to you.
- Fold the foot plate(s) upwards (Fig. 06, Fig. 12).
- Loosen the Velcro fastener of the back cover (under the seat).
- If present, remove the seat cushion, then pull the seat tensioning upwards in the middle. Then remove the tubes of the seat. Fold together with the flat of your hand, making sure that these are only pressed on the underside so that the fingers are not trapped in the seat.
- To ensure that the wheelchair is fully folded, you can additionally push the wheelchair together as far as possible using the armrests or the wheels. (Fig. 07).

 Watch your fingers when folding and unfolding, especially in the folding area, as there is a danger of getting stuck.

4.2. Legrests

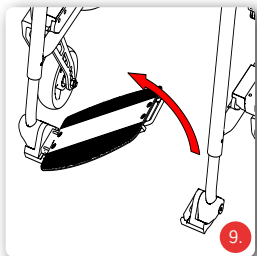
4.2.1. Foot plate foldable

The BX 11 is equipped with a continuous, fold-down foot plate (Fig. 08).

You can fold up the foot plate to the side, adjust the angle and set it to the lower leg length of the user.

Fold the foot plate up and down


- Reach under the foot plate and fold it up sideways (Fig. 09).
- As soon as you are sitting in the wheelchair, fold down the footplate again. Then press the footplate down until you hear it click into place.

Adjusting the legrest length

The foot plate can be adjusted 8 times in 15 mm steps to the lower leg length adjustable.

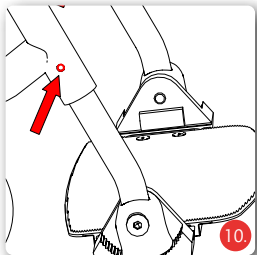
- To do this, remove both hexagon socket screws on the leg support tube (inside and outside) and adjust to the desired length (Fig. 10).
- Tighten the screws again.

Adjusting the angle of the foot plate

The angle of the foot plate can be adjusted (Fig. 11):

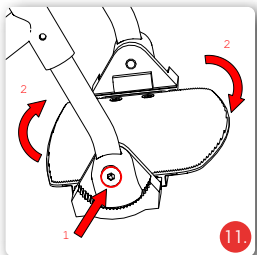- Unscrew the hexagon socket screw.
- Rotate the sprocket to adjust the angle of the footplate to the desired position
- Tighten the screw again.

Before driving off, make sure that the legrests are firmly seated!

4.2.2. Legrests can be swivelled down

The BX 11S is equipped with removable leg supports (Fig. 12). You can individually fold up the foot plates to the side, adjust the angle and set them to the lower leg length of the user. You can remove the legrests without tools, swivel them away and then reassemble.

Fitting the leg rests

- Hold the leg support by the upper part so that it points outwards.
- Insert the leg support with the guide profile into the frame hole (Fig. 13).
- Swivel the legrest forward until it clearly audibly engages (Fig. 13).
- Fold down the foot plate if necessary.

Swivel away and remove the legrests

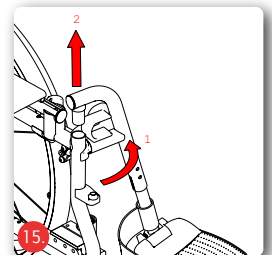
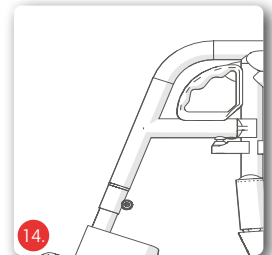
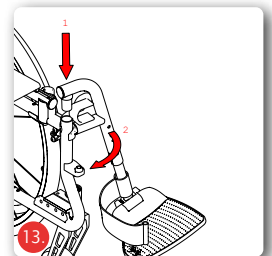
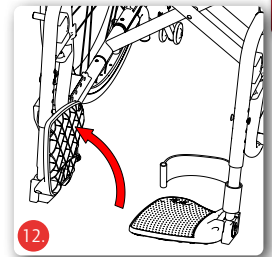
To make it easier for you to get in and out of your folding wheelchair, you can swing the leg supports outwards or inwards without having to remove them.

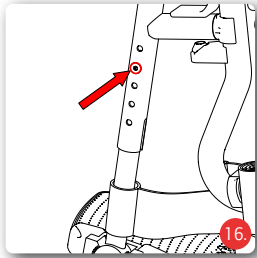
- fold up the foot plates if necessary.
- pull the handle upwards (Fig. 14) and pull the legrest upwards.
- swivel outwards (Fig. 15).
- In the outward position, the legrest can be pulled out upwards (Fig. 15).

Before you set off, refit the legrests as described above.

Never climb onto the footplate!


Adjusting the legrest length

The foot plates can be adjusted 5 times in 20 mm steps to the length of the lower leg.

- To do this, remove both hexagon socket screws on the leg support tube (inside and outside) and adjust to the desired length (Fig. 16).
- Tighten the screws again.

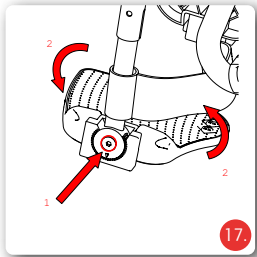Angle adjustable foot plates

The angle of the foot plates can be adjusted (Fig. 17).

- Unscrew the hexagon socket screw.
- Turn the sprocket to set the foot plate in the desired position.
- - Tighten the screw again.

4.2.3. Horizontally adjustable leg supports (optional)

Assembly, disassembly and swivelling are carried out in the same way as for the standard leg supports. To adjust, loosen the clamping lever, then adjust the legrest to the desired length and tighten the clamping lever

The horizontal adjustment works as follows:

- push the locking lever forward.
- by raising or lowering the footplate support tube, bring the legrest into the desired position.
- Release the locking lever; the legrest locks automatically.

Never climb onto the foot plate and watch your fingers when swivelling, there is a danger of getting caught.

Before driving off, make sure that the legrests are firmly seated!

Make all settings the same on both sides.

The legrest must be swivelled away or removed in order to sit down. Never sit on the legrest. Danger of tipping over!

4.3. Amputation leg support (optional)

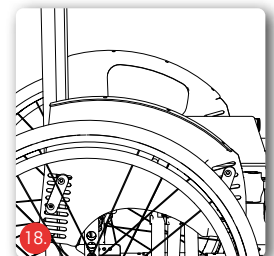
Assembly, disassembly and swivelling are the same as for the standard legrests, adjustment is the same as for the adjustable legrests, and horizontal adjustment is the same as for the horizontally adjustable legrests.

4.3.1. Active side panels

The wheelchair is equipped with an aluminium clothes guard as standard (Fig. 18). This is attached to the front and rear of the frame. The height of the clothes guard must be adjusted if the seat position or the tyre size is changed.

Your dealer will tell you how to adjust the clothes guard in the assembly instructions.


4.3.2. Plug-in side panels with height adjustment (optional)

The pluggable side sections are equipped with height-adjustable armrests. You can remove the side part completely.

Armrest height

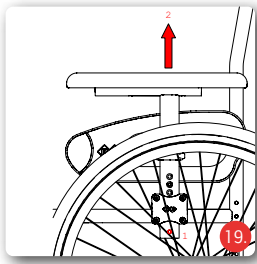- Press the push-button at the lower end of the tube and adjust the armrest to the desired height (Fig.19).
- Release the push button so that the armrest locks into the next possible position.

Removing the side panels

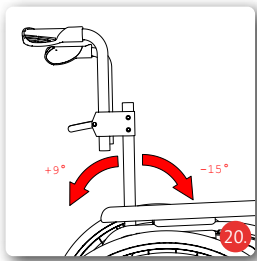
- Press the push-button at the lower end of the tube and pull the section completely out of the lock.

To assemble, push the side part into the lock and adjust the armrest to the desired height. The side part must engage audibly.

4.4. Backrest angle

4.4.1. Backrest (standard)

The backrest of your wheelchair is not adjustable. The backrest angle is 90° to the seat surface.

4.4.2. Angle-adjustable Backrest (optional)

The optional backrest of the wheelchair can be adjusted in angle (Fig. 20).

Your dealer will tell you how to adjust the backrest in the assembly instructions.

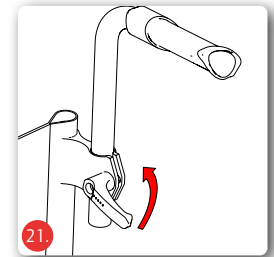
4.5. Push handle height

4.5.1. Push handles (standard)

The push handles can be adjusted to a comfortable position for the accompanying person. The backrest height and the back sag must then be adjusted.

For further adjustment options, please consult the installation instructions of your dealer.

4.5.2. Push handles (optional)

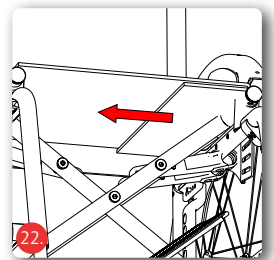
- The optional push handles can be adjusted to the individual height of the accompanying person, independently of other adjustments.
- Release the lock (Fig. 21).
- Set a comfortable height for the accompanying person.
- Tighten the lock again.

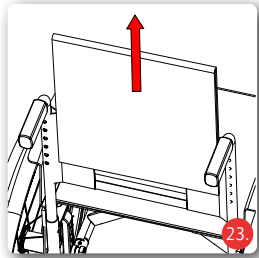
4.6. Fitting seat and back

4.6.1. Seat sag

The seat of your wheelchair can be retightened. Always tighten it sufficiently. Nevertheless, make sure that there is enough slack in the pelvic area.

- If present, first remove your seat cushion.
- Undo the Velcro Fastener under the seat
- Pull the seat upholstery as tight as desired and tighten with the Velcro Fastener (Fig. 22).
- Put your seat cushion back on the seat.


4.6.2. Back sag

A well-fitted back allows you to sit more relaxed in the long term and reduces the risk of pressure points. Make sure that there is enough sag in the pelvic area!

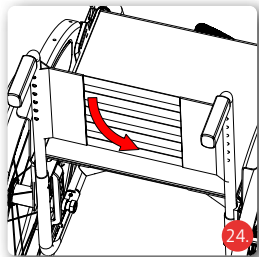- First, completely remove the upholstery of the backrest covering (Fig. 23).
- Loosen the Velcro Fasteners of the tensioning straps (Fig 24).
- Adjust the tapes in order, starting from the bottom.
- Put the upholstery back over the tensioning straps.

4.7. Brakes

The wheelchair is equipped as standard with parking brakes for operation by equipped for the wheelchair user. Optionally, you can extend the equipment with a drum brake, which is used by an attendant.

Your specialist dealer can adjust the drum brake using the installation instructions.

To adjust the brake, contact your specialist dealer. Remember that incorrectly adjusted brakes can have life-threatening consequences.

The parking brake must not be used as a service brake, as there is a risk of fatal injury due to the risk of falling!

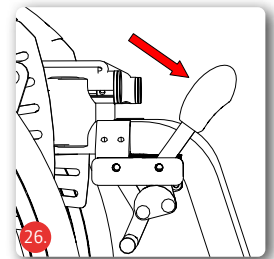
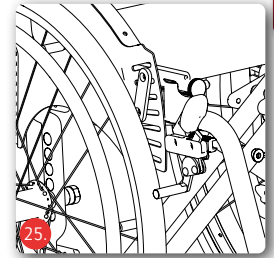
Be aware of the risk of burns if your hands heat up when braking over the push rims!

4.7.1. Actuation of the locking brake

The parking brake may only be used for parking. Only the hand rims on the drive wheels (Fig. 25) or the optional drum brakes for escorts are used to brake the drive.


- When braking, grasp the highest point of the handrims. First let the handrims slide between your hands.
- Slowly apply the braking force via the holding force that you apply to the gripping.
- To increase the braking force, slowly increase the holding force on the handrims until the wheelchair stops.
- As soon as the wheelchair is stationary, push both brake levers forward (Fig. 26). The wheelchair is safely braked when it can no longer be pushed away and both brake levers are pushed forward as far as they will go.
- Pull the lever back towards you to release the brake.


4.7.2. Operation with brake lever extension (optional)

You can optionally equip your wheelchair with a brake lever extension. The extended lever reduces the necessary effort and is therefore particularly recommended for people with reduced arm and hand strength.

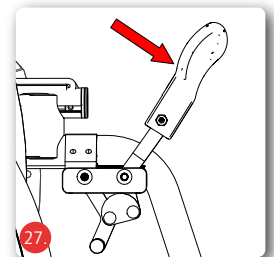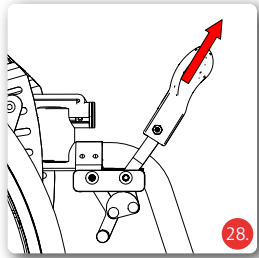- Bring the wheelchair to a standstill (see chapter 4.7).
- Push both brake levers forward. The wheelchair is safely braked when it can no longer be pushed away and both brake levers are pushed forward as far as they will go (fig. 27).


- Pull the lever back towards you to release the brake. Operating position: Extend the brake lever to its longest position and fold it down before driving off (Fig. 28, Fig. 29).

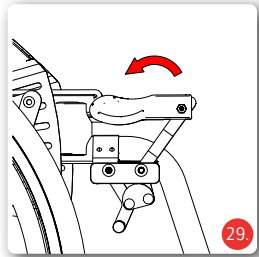Fold the brake lever extension backwards during side transfer (Abb. 29).


For brake adjustment and installation of the brake lever extension, please consult the installation instructions provided by your dealer.

The parking brake - with and without brake lever extension - only acts on one drive wheel and must therefore always be used on both sides!

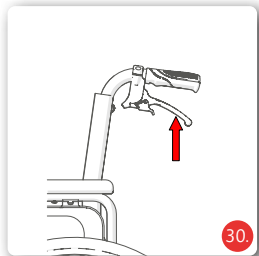Never use the brake lever as a support - e.g. when transferring or standing up - otherwise there is a risk of tipping and falling! The lever could break off!

4.7.3. Drum brake (optional)

The wheelchair can be equipped with a brake system that can be operated by an attendant via the brake levers on the push handles. (Fig. 30).

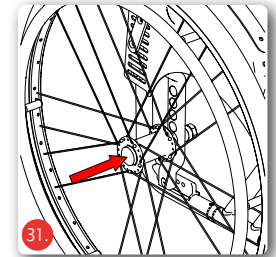
4.8. Wheels and tyres

4.8.1. Thru axle

The wheelchair has quick-release axles as standard. Therefore you can easily mount and dismount the wheels.

- Press the locking button of the quick-release axle with the thumb and pull off the wheel (Fig. 31).
- When refitting, simply place the wheel on the holder. When doing so, press in the locking button.

The locking sound of the quick-release axle tells you that the wheel is securely mounted. After each assembly, make sure that the wheels are firmly seated!

4.8.2. Drive wheels

Check the drive wheels regularly for sufficient tyre pressure and tyre tread.

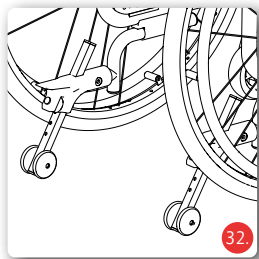
Set the tyre pressure as indicated on the tyre casing.

4.8.3. Castor wheels

If the castor wheels are worn or defective, ask your specialist dealer to repair them!

4.9. Anti-tip rollers and tipping aid (optional)

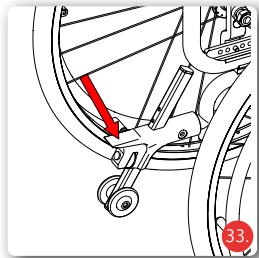
The combination of anti-tip wheels and tipping aid (Fig. 32) can be used for more stability on slopes. It also helps attendants to overcome obstacles such as kerbs.

Your dealer can find the setting in the assembly instructions.

4.9.1. Tipping aid (optional)

- The tipping aid can be mounted on one or both sides.
- Hold the wheelchair by the push handles with both hands.
- Step with one foot on one of the tipping aids (Fig. 33) and push the wheelchair downwards by the push handles at the same time.

4.9.2. Anti-tip rollers (optional)

The anti-tip castors prevent the wheelchair from tipping over backwards. The anti-tip castors are particularly recommended for inexperienced wheelchair users or wheelchair users with leg amputations.

Contact your dealer for adjustment

For adjusting please contact your dealer

4.9.3. Lap belt

If required, a lap belt is available as an accessory. This is fitted to the side frame of the wheelchair when the side section is temporarily removed or swung down. Fold the open end of the strap once around the side frame tube and then fix it with the strap buckle.

The harness length is also adjusted via the harness buckle. Adjust the belt length so that the belt is tight but not pressing.

4.10. Wheel camber

If your wheelchair is not equipped with a drum brake, it is possible to adjust the wheel camber. The greater the wheel camber, the greater the stability when turning and twisting the wheelchair.

Contact your dealer for adjustment

4.11. Spoke protector (optional)

The spoke protector is attached to the spokes with cable ties.

4.12. Attach pockets (optional)

If you want to attach a bag to your wheelchair, hang the loops of your bag over the back tubes of your wheelchair. Please note that the maximum user weight is reduced by the attached weight.

Please note that the reflectors must not be covered.

Caution: Increasing the load of the bag also increases the risk of your wheelchair tipping over backwards. The use of anti-tip wheels is strongly recommended.

5. Accessories

5.1. Legrest horizontally adjustable

A horizontally adjustable legrest is available for the BX 11S wheelchair.

5.2. Amputation support adjustable

An adjustable amputation leg support can also be attached to the BX 11S wheelchair.

5.3. Tipping aid

A tipping aid is available to make it easier to tip the wheelchair.

5.4. Anti-tip rollers

To prevent the wheelchair from tipping over backwards, it is also possible to equip the wheelchair with anti-tip castors.

5.5. Angle-adjustable back with adjustable sliding tubes

On request, the wheelchair can also be supplied with an angle-adjustable back. In this case, the push handles can be adjusted to the individual height of the accompanying person, independently of other adjustments.

5.6. Tyre casing

In addition to the different sizes of the wheels, it is also possible to choose a different variant for the casing of the drive wheel.

5.7. Spoke protector

To prevent fingers from getting caught in the spokes, your wheelchair can also be equipped with a spoke guard.

5.8. Drum brake

For the equipment with a 24" drive wheel, there is the option to equip with a drum brake.

5.9. Brake lever extension

It is also possible to equip your wheelchair with a brake lever extension. The extended lever reduces the amount of force required and is therefore particularly recommended for people with reduced arm and hand strength.

5.10. Plug-in side panels with height adjustment

Plug-in side panels with height-adjustable armrests (PU foam) are also available as accessories.

5.11. Lap belt

A Lap belt can be supplied on request.

5.12. Cane holder

To be able to carry a walking stick on the wheelchair, a cane holder is available as an accessory.

6. Use

6.1. Entering and exiting from the side

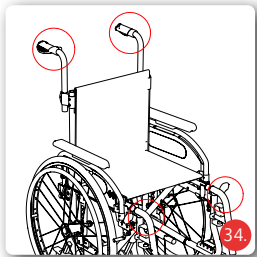
- Bring the wheelchair and the current or future seat as close together as possible.
- Secure the wheelchair with the parking brake (see chapter 4.7).
- If necessary, swing the armrest backwards or remove the side part (see chapter 4.3).
- Fold both foot plates upwards, swing them to the side and remove them or fold the single foot plate upwards (see chapter 4.2).
- Slide sideways onto the other seat. Sit as far back as possible.
- Move the leg supports back to the starting position.
- If necessary, lock the side parts again.

6.2. Entering and exiting from the front

Never climb onto the foot plate(s), there is a danger of tipping over!


- Fold up the foot plate or remove the leg supports (see chapter 4.2).
- Bring the wheelchair and the current or future seat as close as possible to each other and apply the parking brakes on both sides (see chapter 4.7).
- Should you wish to transfer from another wheelchair or room undercarriage, these must also be secured by applying the parking brakes.
- Now slide onto the seat by turning the body.
- Finally, hang the leg supports back in the starting position and fold down the foot plates.

6.3. Climbing over stairs or a high step

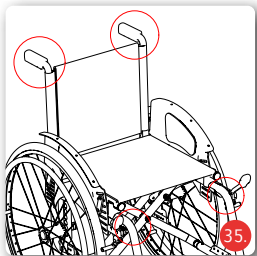Only the frame and the push handles serve as holding points.

The helpers only lift firmly mounted frame parts. The lifting points depend on the model equipment.

BX 11 (with detachable foot plate) (Fig. 34).

BX 11S (with swing-away legrests) (Fig. 35).

To drive up a staircase, the rear helper pulls the wheelchair backwards up the steps using the fixed handles. The lower helper grips the front frame tubes and stabilises the wheelchair. In doing so, he presses the rear wheels of the wheelchair against the steps.

When driving down, the lower helper brakes by pushing the wheelchair

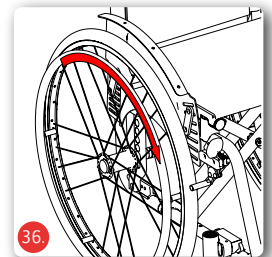
straight against the steps. The upper helper holds the wheelchair by the push handles, secures it and keeps it in the correct position. The wheelchair should roll down one step at a time.

For obstacles and stairs with more than three steps, two accompanying persons must help.

6.4. Driving with the wheelchair

- Sit in your Bischoff & Bischoff wheelchair. Make sure that you sit as far back as possible.
- Locomotion is via the handrims on the drive wheels.
- Place your hands with thumb and bent index finger on the handrims (Fig. 36).
- From this position push the wheelchair with both hands.
- Drive to the right: hold the right handrim and continue to push only the left handrim.
- Drive to the left: perform the push in reverse.
- Turning on the spot: push the handrims against each other with both hands at the same time.
- Braking: reduce the speed of the handrim with your hands.
- Carry out the first driving attempts carefully until you have become accustomed to the wheelchair and its driving behaviour. Wheelchairs have only limited tipping and sliding safety. Take special care when braking, starting or turning on inclines or declines. Extreme leaning out of the wheelchair also increases the risk of tipping over. An anti-tipper reduces this danger. To overcome obstacles more easily, the attendant can tilt the wheelchair (see chapter 4.9.1).

6.5. Check before setting off

6.5.1. Checking the brakes

Before each journey, check that the brake system is functioning properly.

When the brakes are applied, the wheelchair cannot be pushed away on a dry surface. When the brakes are released, the wheelchair moves without grinding noises and with good directional stability.

If this behaviour does not apply, inform your specialist dealer immediately and remove the wheelchair from service until it has been repaired!

6.5.2. Checking the tyres

Before each journey, check the tyres for integrity and correct air pressure (see chapter 4.8.2).

7. Technical data

Dimensions	BX 11	BX 11S
Total length with leg supports (mm):	Min. 910 / Max. 1130	Min. 990 / Max. 1135
Total length without leg supports (mm):	Min. 910 / Max. 1030	Min. 950 / Max. 1100
Width (mm) (with wheel camber 0°):	Min. 550 / Max. 750	Min. 550 / Max. 750
Height (mm):	Min. 900 / Max. 1025	Min. 835 / Max. 1020
Width folded (mm):	Min. 320 / Max. 340	Min. 320 / Max. 340
Height folded (mm):	Min. 835 / Max. 875	Min. 835 / Max. 875
Length folded (mm):	Min. 910 / Max. 1030	Min. 950 / Max. 1100
Handle height (mm):	800 - 950	800 - 950
Lower leg length (mm):	430 - 535 (BX 11)	430 - 510 (BX 11S)
Footboard angle (°):	+/- 25 (BX 11) +/- 10 (BX 11S)	+/- 25 (BX 11) +/- 10 (BX 11S)
Seat width (mm):	360 - 520 with 20 mm steps	360 - 520 with 20 mm steps
Seat depth (mm):	380 - 540	380 - 540
Front seat height (mm):	490 - 560 with 10 mm steps	490 - 560 with 10 mm steps
Rear seat height (mm):	420 - 520 with 20 mm steps	420 - 520 with 20 mm steps
Angle:		
Seat level (°):	0 - 15	0 - 15
Leg to seat surface (°):	75	75
Back rest angle (°):	-9 bis 15	-9 bis 15
Back height without seat cushion (mm):	300 - 450 with 25 mm steps	300 - 450 with 25 mm steps

Height of armrest without seat cushion (mm):	180 - 220	180 - 220
Armrest length (mm):	340	340
Armrest width (mm):	40	40
Handwheel diameter (mm):	515 - 530	515 - 530
Horizontal position of the axis Achse:	250	350

Weights

Max. Load capacity (kg):	136	136
Unladen weight (kg):	13,4 - 15	14 - 18
Weight without attachments (kg):	Min. 9,4 / Max 13	
Weight of the heaviest component (kg):	10 - 10,2	9,5 - 10
Weight attachments		
- Legrest (kg):	2,2	
- Side panel (kg):	2,6	
- Seat panel (kg):	2,8	

Driving characteristics

Steering range (mm):	950 - 1120	1010 - 1200
----------------------	------------	-------------

Stability

Static uphill/downhill (°):	11 / 15	11 / 15
Static lateral (°):	15	15

Environmental conditions

Temperature (°C):	max. 40/-20
Humidity (%):	max. 80

Storage conditions

Temperature (°C):	max. 40/-20
Humidity (%):	max. 60

Tyres

Drive wheels (inch):	22" / 24", pneumatic tyres, max.7,5bar 24" PU (optional)
Steering wheels (inch):	4" - 7" PU
Wheel camber (°):	0 - 2

Equipment

Folding mechanism	Double cross brace
Side panels	Swivelling and removable (optionally height-adjustable)
Legrests	removable/swivelling with continuous footrest, the lower leg length is adjustable, (optionally horizontal adjustment).
Brakes	Knee lever tyre brake (compact brake), Drum brake for attendant.
Drive wheels:	Thru axle
Frame:	Aluminium, lacquered
Clothes guard:	Active Side panel: Aluminium Removable side panel: plastic (ABS)
Seat and back upholstery:	Nylon, Textile
Plug in armrest :	PU foam (optional)
The dimensions given are subject to manufacturing tolerances of +/- 10 mm.	

8. Care instructions for the user

8.1. Cleaning and disinfection

The seat and back materials can be cleaned with warm water using a sponge and/or a soft brush. In case of stubborn dirt, a commercial cleaning agent can be added to the water.

Note: Never use aggressive cleaning agents such as solvents / scouring agents or hard brushes.

Plastic parts are best cleaned with a commercially available plastic cleaner. Please observe the product information of the cleaning agent.

- Plastic claddings are attacked by non-ionic surfactants and solvents, especially alcohols!
- Wipe the frame parts of the wheelchair with a damp cloth. For heavier soiling, also use a mild cleaning agent.
- Clean the wheels with a damp brush with plastic bristles (do not use a wire brush!).
- Wash the push handles, arm pads, seat and back pads with a mild detergent.
- Upholstery parts can be machine washed at 40°C. Do not machine dry, only dry at room temperature.

8.2. Disinfection

The wheelchair can be disinfected with a household disinfectant. Use approved spray disinfectants or alcohol-based wipe-on disinfectants (z.B. Bacillol AF oder Kohrsolin FF); See manufacturer's recommendations for application time and concentration.

- In the case of multiple product users, a commercial disinfectant is prescribed.
- Use only wipe disinfectants approved by the Robert Koch Institute (RKI) and the Association for Applied Hygiene (Association for Applied Hygiene e.V.) (e.g. Bacillol AF or Kohrsolin FF); see manufacturer's recommendations for exposure time and concentration.

- Caution: Damaged upholstery parts cannot be sufficiently disinfected. These should be replaced.
- Under certain circumstances, the surfaces can be affected by unsuitable disinfectants. Longer-term restrictions of the functionality can be the result.

Please note our enclosed overview of the most important care, hygiene and maintenance tasks.

9. Maintenance note

We recommend that the wheelchair is inspected by a specialist dealer at least once a year.

In the event of faults or defects on the wheelchair, it must be handed over immediately to the specialist dealer for repair.

The repair and replacement of parts is carried out by the specialist dealer.

Repair and adjustment work requires special technical training and may therefore only be carried out by specialist dealers authorised by Bischoff & Bischoff GmbH.

The necessary information and documents for repair and maintenance will be made available to the specialist dealer on request.

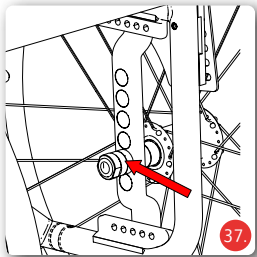
If you have any questions, please contact the Bischoff & Bischoff customer service.

You must check your wheelchair regularly according to the maintenance schedule and find it to be in perfect working order.

9.1. Maintenance advice for the trade

Maintenance must be carried out in accordance with the Bischoff & Bischoff maintenance plan. A maintenance schedule will be provided upon request. A training programme for the specialised trade is also available.


9.2. Adjustment work for the trade

The settings explained in this chapter affect safety-relevant criteria of your product and may therefore only be carried out by trained specialist personnel.

9.3. Centre of gravity, seat angle and Seat height


9.3.1. Adjusting the drive wheel

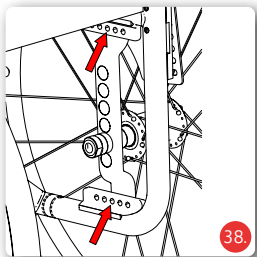- The position of the drive wheel influences the driving behaviour of your wheel chair.
- Change the position of the drive wheel:
 - Vertical: Adjusting the seat height and the seat angle (Fig. 37).
 - Horizontal: Adjustment of the centre of gravity position (Fig. 38).

Seat slope

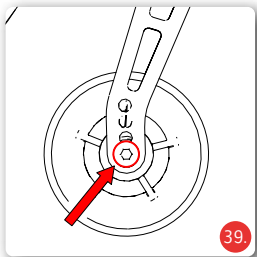- Horizontal or forward-sloping seat: mount the thru-axle mount in the lowest position. However, the seat angle sloping forwards also depends on the position of the front wheel.
- If the seat slopes steeply backwards: fit the thru axle mount in the top position. In this position, the seat angle and centre of gravity are shifted backwards and the wheelchair can tip backwards more easily

Adjustments to the drive wheels may increase the risk of tipping over. Use the anti-tip castors for safety!

By changing the seat angle, there is a risk of tipping over and thus an increased risk of injury.

9.3.2. Height of the front wheel

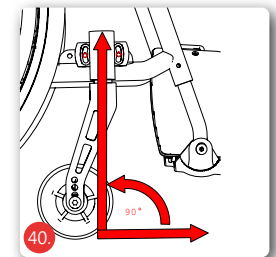
- After you have adjusted the rear wheel position, now check the position of the steering wheels and adjust them if necessary.
- Loosen the axle and remove it completely (Fig. 39).
- Position the steering wheel for the desired seat height.
- Put the axle back into the caster fork and tighten it.
- Set the steering axle perpendicular to the ground to achieve straight running and safe driving behaviour.

9.3.3. Angle of the steering axle

By changing the drive wheel position or that of the front wheel, the angle of the steering axle can also change. However, this must always be at a right angle (90°) to the ground (Fig. 40).

- Release the castor bushing lock by loosening the two screws on the outside (Fig. 40). Fine adjustment and fixing in the specified position is done by the pin screws inside the castor bushing.
- Move the front wheel to the desired position and then tighten the fastening screws.

Further details can be found in the assembly instructions.


9.3.4. Wheelbase

Extending the wheelbase improves the stability and sturdiness of the wheelchair. Reducing the wheelbase, on the other hand, improves manoeuvrability. Reducing the wheelbase is only recommended for experienced drivers.

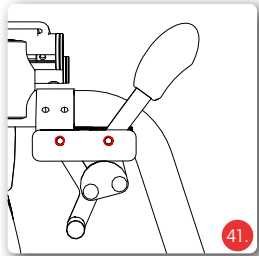If the wheelbase is changed, subsequent adjustment of the knee lever brake is absolutely necessary!

9.4. Adjusting the locking brakes

To adjust the brake, contact your specialist dealer. Remember that incorrectly adjusted brakes can have life-threatening consequences.

If the brakes only work insufficiently or unevenly, contact your specialist dealer. He will check the brakes as follows:

- If there is uneven wear of the tyre tread, or if the tread is no longer visible, change the tyre.
- If the profile is no longer visible on the brake pin, change the brake pin.
- Move the brake system into position that provides safe braking in all driving situations and is operable.
- To adjust the distance, loosen the two hexagon socket screws on the brake bracket (Fig. 41).
- Then slide the brake system along the bracket into the correct position.
- Tighten the two screws again.
- Check that the brake fully engages when the lever is operated and does not snap back.
- Then tighten the screws again and carry out a brake test!

Check that both drive wheels have an air pressure of 7,5 bar.

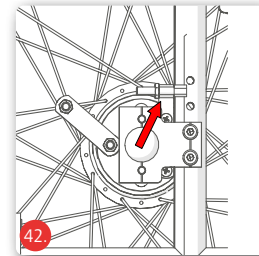
9.5. Adjusting the drum board

To adjust the brake, contact your specialist dealer. Remember that incorrectly adjusted brakes can have life-threatening consequences.

To achieve an optimal braking effect, the braking force is adjusted at the adjusting screw. The braking force is increased by unscrewing the adjusting screw and decreased by screwing it in (Fig. 42).

- Please turn out the adjusting screw until you hear grinding noises on the turning wheel.
- Then turn the adjusting screw in until the grinding of the wheel disappears. The wheel then runs freely.
- After finishing the adjustment, fix the adjusting screw by tightening the lock nut.


9.6. Adjusting the anti-tip rollers

Anti-tip castors prevent the wheelchair from tipping over backwards.


- adjust the distance between the wheel and the ground to approx. 30 - 50mm.
- pull back the disc and move the anti-tip roller along the guide (Fig. 43).

The anti-tip castors are particularly recommended for inexperienced wheelchair users or those with leg amputations.

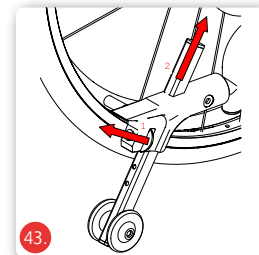If the user does not need anti-tip wheels, you can push it into the top position or pull it out completely (see chapter 4.9.2).

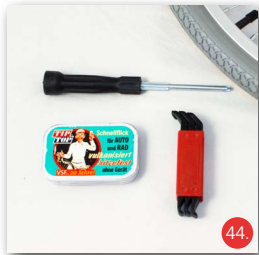For safety reasons, always use the anti-tip rollers in pairs!

9.7. Changing the tyre casing and tyre tube

Regardless of whether you only want to replace the tyre casing or repair/replace the inner tube, in both cases the inner tube must always be removed completely.

You will need the following tools to change the tyre casing or repair the inner tube:

- A small screwdriver
- A commercially available hose repair kit
- 1 set of commercial coat lifters with at least 3 coat lifters.

Disassembly of the jacket:

1. Remove the wheel, see chap. 4.8.1.
2. Place the wheel with the handrim downwards on a non-scratching surface.
3. Release the air from the tyre by pushing down the pin in the centre of the valve with the screwdriver until you hear the air escape.

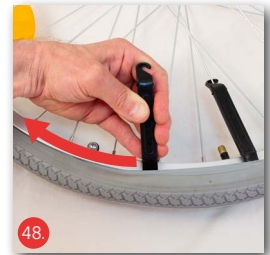
4. Slide 2 mantle lifters under the bead of the rim with the underside facing upwards, tilt the mantle lifters towards the spokes and thus pull the mantle upwards with both mantle lifters at the same time.


5. Hang the coat lifters in the spokes, see Fig. 47.

6. Pull a casing lifter along the rim all around the tyre.

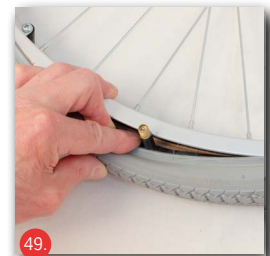7. Then push out the valve. If there is still too much air in the tyre, deflate it as described.


8. Pull the inner tube out of the entire tyre.

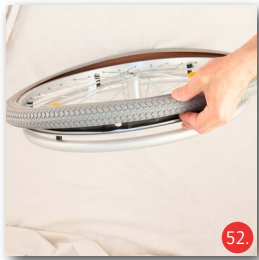9. If you want to change the tyre casing, simply pull it off to the side.

Mounting the tyre casing and the tube:

1. First put one half of the casing over the rim flange on the side opposite the push rim.

2. At the point provided for fitting the valve, press down the casing bead, making sure that the casing remains above the rim around the rest of the wheel. Then insert the valve through the hole provided.

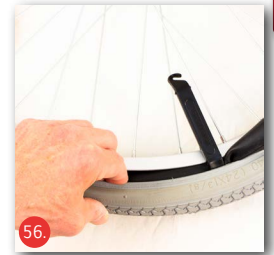
3. Place the casing lifter behind the casing bead and carefully lever the casing over the hose. Be careful not to crush the hose.


4. Hook the casing lifter into the spoke. Be careful not to damage the tube.


5. Now stuff the tube all the way around the tyre under the casing.

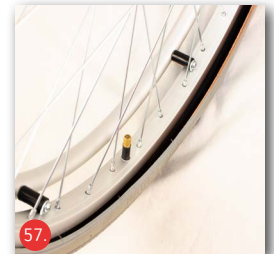6. Ready mounted hose.

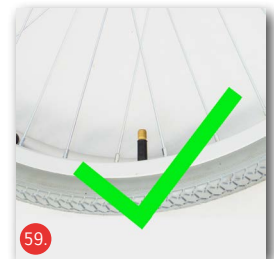7. If the valve is not straight, bring it into a straight position by pushing the valve in the desired direction. You can support this by rolling the jacket.

8. Roll the casing for support to move the valve into the correct position. Do this also before inflating the tyre so that the tube is correctly positioned in the rim shoulder.

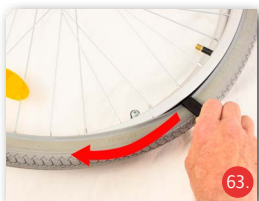
Jacket assembly:

1. The undersides of the casing lifters are to be pushed between the casing and the rim with the side facing upwards.

2. Now use the mantle lifters to throw the mantle bead over the rim.

3. Now pull one of the two casing lifters along the rim flange all the way around the tyre, thus placing the casing bead behind the rim flange.

4. Pull the casing lift as far as possible around the tyre.

5. If the casing lifter cannot be pulled any further, place the third casing lifter in the middle of the other two and place it between the casing and the rim flange.

6. Now throw the mantle over the rim.

7. Inflate the tyre a little, then waltz, then finish inflating.

8. Screw the dust cap onto the valve. Now the wheel can be put back on the wheelchair, see chapter 4.8.1.

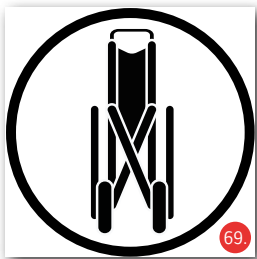
10. Repair

Repairs and maintenance may only be carried out by authorised partners (medical supply stores or specialist dealers). Please contact our customer service by telephone to find a partner near you.

11. Transport

11.1. Transport of the wheelchair

Your wheelchair should only be transported when folded (Fig. 68). To make the wheelchair suitable for transport in a vehicle or aircraft as manageable as possible, remove all detachable parts (leg supports, drive wheels, armrests, plug-in accessories) and fold the wheelchair (see chapter 4.1).

When folding, take care not to trap any limbs.

When folded, the wheelchair should be secured to fixed frame parts in the vehicle.

11.2. Carriage in motor vehicles

You can tell that your wheelchair is not approved as a seat in motor vehicles for the transport of mobility-impaired persons (KMP) by the symbol on the type plate (Fig. 69):

The wheelchair is not crash tested and must therefore never be used as a seat in a KMP. There is a danger to life!

12. Passing on the wheelchair

The wheelchair is suitable for reuse. Before it is passed on to another user, it must be serviced and hygienically prepared by a specialist dealer.

When passing on the wheelchair, please remember to give this user manual and the address of your specialist dealer to the new user.

To adjust the brake, contact your specialist dealer. Remember that incorrectly adjusted brakes can have life-threatening consequences.

13. Storage / Shipping

If the wheelchair is to be stored or shipped, all insertable and unattached parts, except the drive wheels, must be removed and packed individually in suitable cardboard boxes. The individually packed parts can then be packed together in a larger carton. It is recommended to keep the original packaging and store it for these purposes so that it is available in case of need. Then your wheelchair is optimally protected against environmental influences during storage or transport.

For the storage of your wheelchair, please observe the information in the technical data.

To avoid mould and damage to the upholstered parts, the storage location should be as dry as possible and not exposed to direct sunlight.

14. Disposal

If your wheelchair is no longer used and is to be disposed of, contact your specialist dealer.

If you want to dispose of the waste yourself, ask local recycling companies about the regulations in your area.

15. Warranty

1. For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from production and material defects. In case of evidence of material or manufacturing defect, defective parts will be replaced free of charge.

2. This warranty excludes claims, if a repair or replacement of a product or a part is required for the following reasons:

- a) normal wear and tear, which include but is not limited to the following parts where fitted; Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.
- b) overloading the product such as, for example, by exceeding the maximum user weight or load.
- c) the product or part has not been maintained or serviced in accordance with the Instruction manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.
- d) Accessories have been used which are not original accessories.
- e) The product or part has been damaged through negligence, accident or incorrect use.
- f) changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
- g) Repairs have been carried out, before our customer service has been informed of the circumstances.

3. To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.

4. For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1).

5. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.

6. If the repair fails after a reasonable time, the customer can, at his discretion, request a reduction, compensation or the termination of the purchase contract.

7. The guarantee is subject to the law of the country in which the product was purchased from Bischoff & Bischoff & Bischoff.

Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your dealer will be happy to provide these to you.

For replacement devices and repairs, the warranty period is one year.

Observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your dealer will provide these to you.

Warranty certificate

Produkt: BX 11 / BX 11S

Model number:

Serial-number:*

* (to be entered by the dealer)

Specialist retailer:

Date & Stamp

(Pay attention to our general terms and conditions))

*Instruction plates are located on the wheelchair frame

Non-observance of the operating instructions and improperly performed maintenance work, as well as in particular technical modifications and additions (attachments) without the consent of Bischoff & Bischoff GmbH will invalidate both the warranty and product liability in general.

Notice!

Despite careful research and editing, incorrect information may have crept into our operating instructions. The latest version of the operating instructions can be found in the download area on our website: www.bischoff-bischoff.com

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