Bedienungsanleitung | Instruction manual
Manual de instrucciones | Mode d'emploi
Manuale di istruzioni

Centuro S7
Centuro S7 XXL
Centuro S7 MAX

DE


EN

ES

FR

IT

Enjoy mobility.

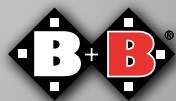Inhalt

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Contents

1. Preface	52	6.4.2. Initiating emergency stop	75
1.1. Signs and symbols	53	6.5. Overload switch	75
2. Important safety instructions	53	6.6. Preparing the scooter for transportation	76
2.1. General safety instructions	53	6.7. Disassembling for storage	77
2.2. Safety in vehicles	55	6.8. Taking on public transport	77
2.3. Use on Public Roads	55	7. Technical Data	78
2.4. Information on EMC interference	55	8. Care instruction for the user	81
2.5. Brakes	56	8.1. Cleaning	81
2.6. Purpose	57	8.2. Disinfection	82
2.7. Indications	57	9. Check Before Use	83
2.8. Contraindications	57	9.1. Brake Inspection	83
2.9. Declaration of conformity	58	9.2. Checking the lights, indicators and horn	83
2.10. Service Life	58	9.3. Inspection of Tyres	83
2.11. Information about range	58	9.4. Batteries	84
3. Description of Product and Contents	59	9.5. Putting into operation	84
3.1. Inspection on Delivery	59	9.6. Daily use	85
3.2. Type plate and serial number	60	9.7. Charging procedure	86
3.3. Summary	61	9.8. Replacing the batteries	86
3.4. Contents	62	9.9. Fuses	87
4. Basic adjustments	62	10. Diagnostic codes	88
4.1. Steering column	62	11. Maintenance	89
4.2. Seat	63	11.1. Maintenance instructions	89
4.3. XL seat (optional)	64	11.2. Maintenance tips for retailers	89
4.4. Backrest	64	12. Repairs	90
4.5. Armrests	64	13. Transport	90
5. Accessories	66	14. Parking	90
6. Using the scooter	68	15. Hand over the scooter	91
6.1. Control Functions	68	16. Use by others	91
6.1.1. Changes to the controller	71	17. Storage/Shipping	91
6.2. Getting in and out	71	18. Disposal	91
6.3. Driving the scooter	72	19. Warranty	92
6.4. Pushing mode	73		
6.4.1. Brakes in pushing mode	74		

1. Preface

Dear user,

You have decided on a high-quality Bischoff & Bischoff scooter, and we would like to thank you for your trust.

The scooter was designed to offer you many advantages and meet your needs. Thanks to the various adjustment options and optimum adaptability, the scooter can be adjusted according to your individual body size. The scooter is designed for outdoor use only. Please read and follow the operating manual carefully before using your new scooter for the first time. The operating manual is an integral and necessary component of the scooter. Keep the operating manual where you can refer to it and if you transfer the scooter, give the manual to the new owner. The Scooter S7 is available with a max. speed of 15 km/h (S7 MAX / XXL 10 km/h).

It is available in three versions:

- Centuro S7
- Centuro S7 XXL higher maximum load of 250kg and XL seat than default
- Centuro S7 MAX better driving characteristics in mountainous surroundings

The scooter complies with the requirements of ISO 7176-14:2008 (drive and controller systems for scooters and scooters requirements and testing). For visually impaired users, this document is available as a PDF file at our website www.bischoff-bischoff.com. Any repairs and adjustments require special technical training and may therefore only be carried out by authorized Bischoff & Bischoff dealers.

Note!

In spite of careful research and editing, incorrect information may have crept into our operating manuals. Rights to technical changes reserved. Illustrations may include accessories. The current version of each instruction manual is ready for download at the Bischoff & Bischoff website:

www.bischoff-bischoff.com

1.1. Signs and symbols

Important! This is especially useful information on a given subject.

Warning! These are especially safety-relevant instructions. Follow the directions in the manual!

2. Important safety instructions

2.1. General safety instructions

- To avoid falls and dangerous situations, you should first practice driving your new scooter on level, unobstructed ground. We recommend having an assistant with you.
- Before operating the scooter, please check that all attached parts are correctly fastened.
- Practice driving on level, unobstructed ground before first use. Get a good understanding for the way the scooter brakes and accelerates when driving straight ahead or a curve.
- Please note that displacing the centre of gravity by shifting the body or by adding loads may increase the risk of tipping over the scooter to the side or backwards.
- The scooter may not be used for transport of more than one person or any additional load. Make sure to only use it for the intended purpose.
- Be aware of the risk of trapping body parts when making settings and adjustments to the scooter.
- Never use the scooter under the influence of alcohol or other substances that may impair your attention and/or your physical and mental capacity.
- To overcome obstacles, please be sure to use ramps. Should it be necessary to lift the Centuro S7 over an obstacle, then only do so

without the occupant. Only lift by the solid parts of the frame.

- Avoid hitting obstacles (steps, curbs) at full speed.
- The control unit must always be switched off when getting in or out of the scooter.
- All road traffic regulations must be observed when driving on public roads.
- The rear lights on the scooter must always be visible to other road users.
- If possible, wear bright, conspicuous clothing. This will make it easier for other road users to see you.
- Please note that the upholstery will heat up if directly exposed to sunlight and may cause injuries to your skin when touched. You should therefore cover these parts or protect the scooter from exposure to direct sunlight.
- The seat materials comply with the flammability resistance requirements of EN 1021-2 and ISO 8191-2. Nevertheless, you should exercise the necessary caution if you are smoking and refrain from putting any cigarettes down on the scooter.
- Reporting of incidents:

The operator or user must immediately report

1. any malfunction
2. any change to the features or performance as well as
3. Immediately report any improper labelling or instructions for use of a medical device that have or could have led to the death or serious deterioration of the health of a patient, an employee or a third party to your competent regional authority for medical devices. This institute will immediately forward the notification to the operator's competent authority and also inform the manufacturer and the manufacturer's competent authority.

Please note that operation in pushing mode by disconnecting the drives must only take place on a level surface. The electromechanical brake is thereby disabled. There is a danger of fatal injury if this instruction is not complied with.

2.2. Safety in vehicles

Your scooter is not suitable for use on the designated buses in the public transport network. For further information please refer to Section 6.

The Centuro S7 is not suitable for use as a car seat. Under no circumstances may it be used as such. It does not meet the requirements stipulated by ISO 7176-19. Failure to observe this instruction can lead to extremely serious or fatal injuries in the event of an accident.

2.3. Use on Public Roads

The scooter is designed for outdoor use. Please keep in mind that you are using public roads and therefore have to observe all road traffic regulations. Do not endanger other road users through inconsiderate driving, especially on walkways (.

2.4. Information on EMC interference

Despite compliance with all applicable EMC directives and standards, it is possible that the scooter may affect or be affected by other electrical devices (e.g. mobile phones, alarm systems in department stores). If you become aware of any such interference, either turn off the other device or turn off your scooter and move it outside the range of the interference.

It is possible that your scooter will interfere with other devices which emit electromagnetic fields and that they will interfere with your scooter.

2.5. Brakes

Make sure you brake your power wheelchair in good time. Especially in front of people and when journeys on slopes.

Service brake:

There is a service brake for braking while in pushing mode with the drive disconnected or in an emergency (Fig. 02).

Parking brake:

Switch off the wheelchair at a standstill using the on/off switch (see chapter 6.). If the drives are engaged, the scooter cannot be pushed or cannot be pushed or moved, as the motors brake automatically (see chapter 6.4).

The function of the service and parking brake complies with the provisions of the StVZO.

Pushing mode:

In order to brake in pushing mode with decoupled drives or in an emergency a service brake is provided (Fig. 2). Braking the the scooter when the drive is engaged is performed by the automatic brake after the drive lever is released. If the drives are engaged, the scooter cannot be pushed or moved as the motors brake automatically.

Please note that sliding operation by decoupling the drives only on level ground. This disables the electromechanical brake out of operation. There is a danger to life if this is not observed

The braking distance is significantly longer on a downhill gradient than on level ground. Keep this in mind when driving on sloping terrain and brake in time.

Emergency brake:

To make an emergency stop while driving, turn the on/off switch.

This results in very heavy braking.

This can lead to injuries and should therefore only be used in case of danger.

An emergency stop may only be performed in the event of a dangerous situation, as this can lead to injuries.

2.6. Purpose

The scooter Centuro S7 is designed for outdoor use by persons with impaired mobility. It is therefore capable of travelling long distances and overcoming small obstacles. It is classified according to EN 12184 in category C. The maximum load capacity is 160 kg or 250 kg for Centuro S7 XXL.

2.7. Indications

Inability or severely impaired ability of adult persons to walk due to

- paralysis
- loss of limb
- defect/deformity of limb
- joint contracture
- joint damage (not in both arms)
- other disease

Provision of a scooter is advisable if the use of manually operated scooters is no longer possible because of the disability but the person can still competently operate a vehicle driven by an electric motor.

2.8. Contraindications

The provision of a scooter is unsuitable for adult persons

- with severe balance disorders
- with impaired or insufficient vision
- with strongly impaired cognitive abilities

2.9. Declaration of conformity

We Bischoff & Bischoff GmbH declare under our sole responsibility, that the Centuro S7 / Centuro S7 MAX / Centuro S7 XXL wheelchair meets all the requirements of MDR 2017/745 that are applicable.

2.10. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life is up to 5 years. Beyond this time the scooter can be used continuously, provided it is in a safe condition.

2.11. Information about range

The range of the Centuro S7 / Centuro S7 MAX is approx. 45 km (Centuro S7 XXL approx. 35 km). The range that can be achieved by electric scooters depends on the following factors:

- Composition of the carriageway
- Way of driving
- Battery condition
- Vehicle load weight
- Ambient temperature
- The use of accessories

Please note that the range specified by us was determined under test conditions:

- batteries as new and fully charged
- ambient temperature of 21 °C
- max. user weight of 160 kg or 250 kg for Centuro S7 XXL
- level, firm surface
- constant speed
- deactivated lighting system

We wish to advise you that the following factors severely restrict the range:

- poor state of charge of batteries
- low ambient temperature (below 10 °C)
- frequent starting and braking

- Driving uphill
- Driving on surfaces that are not firm (field or forest paths)
- tyre pressure incorrect or too low (see Section 7)

If there is a combination of several factors, the range can be reduced by up to 50 %.

Never use your scooter with flat or deep discharged batteries. You could put yourself and others at great risk through an unforeseen halt.

3. Description of Product and Contents

3.1. Inspection on Delivery

All Bischoff & Bischoff products undergo a proper final inspection in our company and are labelled with the CE mark.

The scooter is delivered in a special carton. Keep the carton after unpacking, if possible. It is useful for later storage of the product or for sending it back.

Delivery and first instructions are taken care of by your qualified medical supply dealer.


If the scooter is shipped by rail or road, the goods should be checked immediately for transport damages (carton) in the presence of the bearer.

Should there be any damage, please contact our customer service immediately.

Check the contents for completeness and make sure nothing is damaged. Should you find any irregularities or damage, please contact our customer service.

3.2. Type plate and serial number

The type plate (Fig. 03) and serial number are located on seat's column.

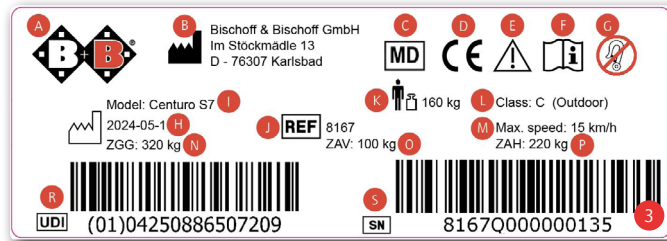- A** Manufacturer logo
- B** Manufacturer information
- C** Medical device
- D** CE mark
- E** Warning! Follow instructions for use
- F** Important - Follow instructions for use
- G** Not suitable for use as a seat in a vehicle
- H** Date of manufacture
- I** Model designation
- J** Model number
- K** Maximum user weight
- L** Application class
- M** Maximum speed
- N** Maximum weight allowed
- O** Permissible axle load front
- P** Permissible axle load rear
- R** UDI number
- S** Serial number

3.3. Summary

3.4. Contents

After receipt of the goods, please check the contents for completeness right away. The contents consist of:

- Outer packaging
- Scooter (ready for use incl. 2 batteries)
- Charger
- Operating manual
- Seat
- Armrests
- Headrest
- Basket

4. Basic adjustments

Before using the scooter for the first time, it is necessary to adjust the scooter according to your height. This is essential to ensure that your posture is physiologically correct for driving for extended periods of time without excessive fatigue. The basic adjustments of the scooter are described below. You should carry out these adjustments with the help of your medical supply dealer.

4.1. Steering column

Adjusting the steering column of your scooter enables you to set the driving position so that it is comfortable. You can make the adjustment while seated in the driver's seat.

- Hold the steering bar at the upper position with the right hand and pull the lever towards you using the left hand (Fig. 05).
- Move the steering column into the required position and release the lever. The steering column is now fixed in the new position.

4.2. Seat

The seat height can be adjusted and the seat can also be moved forwards and backwards as well as rotated to the side.

Rotating the seat

- Push down the lever (located on the right underneath the seat surface, Fig. 06). The seat can now be rotated in both directions.
- There is a fixed lock-in position for the seat every 45°.
- Release the lever and make sure that the seat is locked in position again.


Moving the seat

- Pull the lever (on the right below the seat surface, Fig. 07) towards you. The seat can now be moved forwards and backwards.
- You can now adjust the seat to the required position.
- Release the lever and ensure that the seat is locked in one of the closest fixed positions.

Seat height

You will firstly need to remove the seat.

- Fold the backrest onto the seat by holding the backrest with your right hand and using your left hand to lift up the lever on the left next to the seat surface (Fig. 10). The backrest will then fold forwards automatically.
- Operate the seat rotation lever (Fig. 06) and simply remove the seat by pulling it upwards (Fig. 8).
- Undo the screw connection (Fig. 9) and position the seatpost in the required position inside the tube guide.
- Then securely tighten the screw connection again (Fig. 9) using 40 Nm in the next hole and secure them with screw locking fluid (medium strength).
- Now place the seat back onto the mounting.
- Make sure that it is locked in position again.

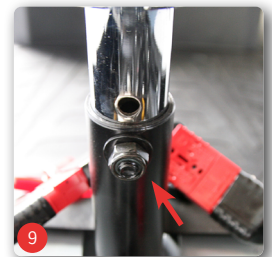4.3. XL seat (optional)

As an option, an XL seat is available for the Centuro S7 (see Section 5). This is the standard configuration of the Centuro S7 XXL.

4.4. Backrest

The angle of the backrest on your scooter can be adjusted forwards and backwards.

Adjusting the angle

- Pull the lever (on the left on the seat, Fig. 10) upwards. The back now pushes forwards.
- Push the backrest into the required position.
- Release the lever and make sure that the backrest is locked in position again.
- Please note that the adjustment of the seat to the rear and/or the angle adjustment to the rear can increase the overall length of the scooter.

4.5. Armrests

The armrests can be swivelled upwards in order to make it easier to mount and dismount. In addition, the height, distance and angle of the armrests can be adjusted.

Height

- Loosen the screw until you can remove it, hold the screw in its position. Now you can move the armrest vertically up and down (Fig. 11).
- Move the armrest to its preferred height and tighten the screw.

Side distance

- Undo the hand screw (Fig. 12) on the back of the backrest frame while holding onto the armrest.
- Adjust to the required distance and tighten the screw again.
- The armrests can also be removed (e.g. for replacement, storage or transportation) after undoing the screw on the back of the frame (Fig. 12).
- Please note that pulling the armrests out to the side can increase the vehicle width.

Angle of support

- You can adjust the angle of support of the armrests by screwing the knurled nut (Fig. 13) on the relevant armrest further in or by unscrewing it.

5. Accessories

5.1. Lockable rear box (optional)


For the safe transport of objects (Fig. 14).

- Can be locked and removed
- Lid can be flipped up to enable easy access to the luggage.
- You can open the box and also release it from the support using a key.
- Dimensions (approx.): height x width x depth 423 x 567 x 280 mm, 40 litres storage space.

5.2. Rollator holder (optional)

- Can be fitted to the seat frame on all scooters using the multifunction holder (Fig. 15).
- For the safe transport of rollators.
- With folding rollators, the rollator holder can be adjusted to the size of the rollator.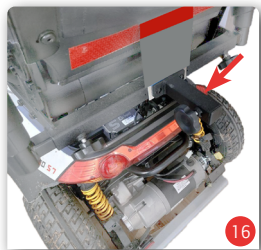

5.3. Multifunction holder (optional)

- The multifunction holder is used to fit other brackets for transportation (Fig. 16).

5.4. Crutch holder (optional)

The crutch holder is fitted either to the seat frame or the multifunction holder (Fig. 17).

- When fitted to the seat frame, the crutch holder is held in place with a retaining strap.
- Never climb on the crutch holder and never use it as a tipping aid.

5.5. XL-Seat

An XL seat is available for a larger seating area.

5.6. Rain cover (optional)

The rain cover is used to protect your scooter from rain. It should be used when you park your scooter outdoors (Fig. 18).

- In order to ensure that your scooter is fully covered by the rain cover, the backrest should be folded down and the handlebars should be facing backwards.

5.7. Oxygen bottle holder (optional)

Fit the oxygen bottle holder on the multifunction holder of the scooter for the safe transport of an oxygen bottle when driving.

 Where accessories are fitted, these extend the scooter and will rotate when you rotate the seat. When doing this you must pay close attention to your surroundings.

5.8. Licence plate holder (optional)

- For affixing an insurance registration number on the carrying bar at the back of your scooter (Fig. 19).
- This position is recommended so that the reflectors and lights are not covered.

 Never use your scooter without an insurance registration number on public roads (relevance only in Germany).

 Important! Please note that reflectors must not be covered.

6. Using the scooter

After you have adjusted the scooter according to your height, familiarise yourself with the functions of the controller.

6.1. Control Functions

The controller is a component of the steering column of your scooter. It contains the LED displays, lights and functional elements.

Hazard warning lights

Use this button (Fig. 21) to switch on the hazard warning lights in emergency situations where the scooter is malfunctioning. All four indicator lights then flash and signal to other road users that your scooter has broken down. A warning beep will be sounded as well.

Speed setting

You can use these three buttons (Fig. 22) to set the maximum speed.

- To set to the slowest level, keep the left switch (turtle) pressed until the display (Fig. 20) only shows one bar next to the turtle.
- To set to the maximum speed, keep the right switch (rabbit) pressed.
- The H/L switch allows a default setting for slow (low-L) and/or fast (high-H). Set at low, the speed is approx. 50% of the max. speed. Set at High, the max. speed is 15 km/h or 10 km/h for Centuro S7 MAX and XXL.
- To switch between km/h and mph, press the MODE button until the speed indicator flashes. Use the H/L key to switch between units. To confirm your selection, hold down the SET button for at least 3 seconds.

Headlight

Use this button (Fig. 23) to switch on the lights on your scooter when dusk begins to fall or when passing through dark routes such as tunnels. This will make you more easily visible to other road users.

Battery Status Display

The battery status display (Fig. 24) shows to what extent the batteries are charged. If you can see all nine bars, the battery is fully charged. As you use the scooter, the battery capacity decreases and fewer bars are displayed.

If only three bars are displayed, this means that only short journeys are possible and the battery should be recharged (see Section 9.4).

Indicators (left/right)

Use the arrow buttons (Fig. 20/25) to switch on the left or right indicator on your scooter to signal to other road users that you wish to turn left/right. Pressing the button again turns the indicator off again. After flashing 15 times the signal turns off automatically.

Horn

The horn (Fig. 26) enables you to draw attention to yourself in difficult situations.

Ignition key

Switch the scooter off and on using the ignition key (Fig. 27).


USB Port

There is a USB port on the inside of the steering column, below the handlebar (Fig. 28). You can use this to conveniently charge your smartphone or other mobile terminals when your scooter is switched on. The max. current supply is 1 A/5 V.

Mileage

You can display your mileage in kilometres (km) or in miles (mi). To do this press the MODE button (Fig. 20) twice until the mileage indicator flashes. Use the H/L key to switch between units. To confirm your selection, hold down the SET button (Fig. 20) for at least 3 seconds.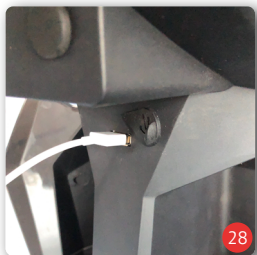

Time

To set the time, press the MODE button three times until the time flashes. You can switch between „AM/PM“ and a 24-hour display by pressing the H/L key.

Set the hours by pressing the left arrow key (indicator). Set the minutes using the right arrow key (indicators).

To confirm your selection, hold down the SET button (Fig. 20) for at least 3 seconds.

Temperature

You can choose to display the ambient temperature in degrees Celsius (°C) or Fahrenheit (F). Press the MODE button four times until the temperature indicator flashes. Use the H/L key to switch between °C and F. To confirm your selection, hold down the SET button for at least 3 seconds.

After pressing the MODE button, the selected display flashes three times. If you do not make any changes during this time, you will leave the menu automatically.

6.1.1. Changes to the controller

Changes to the programming of the controller must not be carried out and will mean that the conformity and public road licence is void. Changes to the programming have an effect on the speed, acceleration and deceleration. This can lead to life-threatening injuries of the user and / or third persons if improperly carried out.

6.2. Getting in and out

Ensure that the scooter is switched off before sitting on it.

- Move the steering column, (see Section 4.1), into the upright position and swivel back the armrest on the side from which you intend to sit down so that the armrest is out of the way.
- It is easier for you to sit down if you rotate the seat 90° to the side from which you intend to sit down (see Section 4.2).
- Now sit down.
- Rotate the seat in the driving direction or swivel the armrest forwards again.
- Adjust the steering column so that you can comfortably reach all the controls.
- To get out, follow the instructions accordingly in reverse order. However, take out the key beforehand so that you can ensure that no unintended actions or movements occur.

Ensure that the seat firmly locks in position again

6.3. Driving the scooter

You should make your first journey on a level site with ample space.

- Ensure that you are sitting correctly on your scooter. Set the speed as low as possible. You can increase the speed when you are more familiar with handling the scooter.
- To switch your scooter on, insert the ignition key into the ignition lock on the control panel and turn the key a quarter turn to the right (Fig. 27).
- To switch off the scooter, turn the key a quarter turn to the left. After getting out of the scooter, always ensure that you have removed the ignition key and do not leave the scooter unattended with the ignition key inserted.
- Please make sure that you do not have your hand on the forwards / backwards lever when you switch on your scooter. This would cause your scooter to display an error message. In this case, turn it off and then on again to rectify this error.
- Pull back the speed selection lever on the right to move forwards (Fig 29). The further you pull back the lever, the higher the speed. The speed can go up to the maximum speed that has been preset using the speed control: 15 km/h (S7 MAX / XXL 10 km/h). Release the lever to stop gently. Practice these two procedures in order to get used to them.
- Steering is easy and works exactly like a bicycle in that you turn the handlebars in the direction in which you wish to move. Leave plenty of distance when you travel around a curve so that the rear wheels are able to follow the path of the front wheels and so that you do not cut across objects.

- It is necessary to pay attention when reversing (Fig. 30). Ensure that your speed is low before reversing. This requires practice. Practice this also on a level site with ample space.

Please note that, for safety reasons, the reversing speed is only half the speed going forward.

- If you turn sharply, turn the handlebars before you accelerate. Always drive in a wide arc around corners and obstacles and ensure that you drive slowly and carefully.

Depending on the model, your scooter is equipped with a function to reduce the cornering speed, nevertheless, you must always be able to adjust the driving speed correctly at all times.

- When the drive is connected, the brake is automatically applied when the throttle lever is released. The scooter then cannot be pushed or displaced because the brake is automatically applied to the motor.

6.4. Pushing mode

If you want to push the scooter in certain situations, you must unlock the brakes. It is easy to push the scooter in free wheel mode.

The electromechanical brake will then not operate.

- Push down the brake lever (Fig. 31), it is positioned on the rear left side of the scooter.

- A small jerk will indicate that the drives are now unlocked.
- To lock, simply push the lever up and locking then takes place automatically.
- The engine has a centrifugal clutch, the brake blocks automatically should you accelerate the scooter too hard in overrun cutoff. Only after a coming to a complete standstill will it be released.

For safety reasons, the motor drive is locked when the scooter is being operated in pushing mode.

Warning! Danger of fatal injury! The drives must only be disconnected on a level surface.

6.4.1. Brakes in pushing mode

Your scooter is equipped with a manual service brake (see Section 2.5). This can be used for support with braking when the drives are connected; it can also be used as a service brake when the drive is disconnected.

To brake or reduce the speed, pull the brake lever towards you (Fig. 32/33).

Parking and locking the brake is performed by locking the drive. For this purpose, the scooter must be brought to a standstill by a helper and/ or by using the manual service brake. When at a standstill, the propulsion can be re-engaged (Fig. 31).

6.4.2. Initiating emergency stop

Should the automatic brake not work as usual while driving, you can initiate an emergency stop. Proceed as follows:

Turn the key a quarter turn to the left to stop the engine. The rear wheels will block immediately. Note that you can be thrown forward by the abrupt stop. Hold on to the handlebars adequately and perform this emergency stop only in an absolute emergency.

An emergency stop may only be carried out in the event of danger, as this can lead to injuries.

6.5. Overload switch

The controller will cut out if the motor is overloaded, e.g. by driving uphill for too long. In this case, the ignition just needs to be switched off and on again in order to start moving again. If this does not work, this means that the controller has overheated.

In this case, please wait a few minutes. In the event of an electrical overload, the fuse will shut off the power to the scooter. The overload switch (Fig. 34) is located on the front of the seat construction (Fig. 35). After pressing the switch, the scooter should function normally again.

If it does not function as expected, please contact your specialist retailer or Bischoff & Bischoff customer service.

6.6. Preparing the scooter for transportation

RISK OF ENTRAPMENT! Ensure that you are aware of the increased risk of entrapment when stowing your scooter.

You can fold up the scooter in order to save space during transportation.

- Switch off the scooter.
- Remove the armrests (Fig. 36) and then fold the backrest forwards onto the seat as described in Section 4.9.
- Remove the basket (Fig. 37).
- Fold the steering column towards the foot area (Fig. 38).
- Depending on the available space, it is sufficient to fold down the backrest and steering column and secure the scooter using securing straps on the frame (Fig. 39).
- Carefully protect the scooter against slipping by attaching the securing straps at the designated holding points.
- The scooter, including the batteries, is generally suitable for transportation on an aircraft. You can find out the transportation conditions from your airline. The batteries may need to be removed. Information can be found in Section 9.4.

6.7. Disassembling for storage

For storage, you must also remove the batteries, be extremely careful. Avoid handling metal tools, touching the battery terminals and observe the instructions of the battery manufacturer.


This work should only be done by a trained technician. There is danger of fatal injury!

For your safety, the seat must be lifted by two people, as it is the heaviest individual part.

- You will firstly need to remove the seat and the panel.
- Remove the seat by pulling it upwards, see Section 4.2.
- The panel is fixed in place with two hand screws (Fig. 40).
- Undo these screws and remove the panel (Fig. 41).
- The batteries should then be removed. To make this easier, the cables are fitted with plug connections.
- Disconnect the plug connections (Fig. 42) for both batteries.
- Then undo the screw on the mounting bracket. You need to completely remove the screw.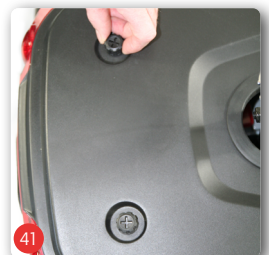

6.8. Taking on public transport

In accordance with a German decree from the year 2017, your scooter is not suitable to be taken on public transport and is not authorised to bear the corresponding sticker (Fig. 43).

7. Technical Data

● Basic equipment ○ Optional			S7	S7MAX	S7 XXL
Dimensions and weights					
Max. Load:	160 kg		●	●	
	250 kg				●
Seat width:	490 mm		●	●	
Seat width XL seat:	520 mm		○	○	●
Max. effective seat width:	Seat width + 100 mm:		●	●	●
Seat depth:	420 mm		●	●	
Seat depth XL seat:	450 mm		○	○	●
Seat angle:	2°		●	●	●
Seat height, from ground to front seat edge:	465 / 490 / 515 mm		●	●	●
Distance armrest to seat:	230 / 280 mm		●	●	●
Backrest height/Backrest height including headrest:	480 / 665 mm		●	●	
Backrest height including XL seat/Backrest height including XL seat and headrest:	545 / 720 mm		○	○	●
Backrest angle:	-12° to +39°		●	●	●
Overall length/with accessories:	1360 / 1450 mm		●	●	●
Overall width:	745 mm		●	●	●
Packing size length/width/height:	1400 / 745 / 720 mm		●	●	●
Curb weight	approx. 142 kg		●	●	
	approx. 157.2 kg				●

● Basic equipment ○ Optional			S7	S7MAX	S7 XXL
Weight of heaviest parts:	117 kg, incl. batteries		●	●	
Seat:	approx. 16.2 kg		●	●	
XL seat:	approx. 18.2 kg		○	○	●
Headrest:	0.8 kg		●	●	●
Armrests (2 pcs):	4.2 kg		●	●	●
Electric					
Fuses:	70 A (readjustable)		●	●	●
Batteries:	2 x 12 V, 90 Ah (maintenance-free, leak-proof)		●	●	●
Charger:	24 VDC / 10 A		●	●	●
Motor:	1100 W		●		
	1000 W			●	●
Controller:	PG 140 (S-DRIVE)		●	●	●
Max. current output of the USB port:	approx. 1 A/5 V		●	●	●
Lights and indicators in accordance with the German StVZO					
Tyres					
Driving and steering wheels:	13" (D320 mm), 300 kPa (3.0 bar)		●	●	●
Driving characteristics					
Speed (forwards):	15 km/h		●		
	10 km/h			●	●
Climbing capacity:	10°/17%		●		●
Climbing capacity:	12°/21%			●	
Max. Obstacle height:	100 mm		●	●	●

● Basic equipment					
○ Optional			S7	S7MAX	S7 XXL
Range:	approx. 45 km	●	●		
	approx. 35 km				●
Reversing width:	2300 mm	●	●	●	
Braking distance (from max. speed):	< 1600 mm	●	●	●	
Static stability:	25° uphill 9° downhill 15° sideways	●	●	●	
Operating forces					
Throttle lever:	< 13.5 N	●	●	●	
Buttons on the controller:	< 5 N	●	●	●	
Disconnecting the drives:	< 60 N	●	●	●	
Operating conditions					
Temperature:	-0 - +50° C	●	●	●	
Air humidity:	20 - 80%	●	●	●	
Storage conditions					
Temperature:	-20 - +60° C	●	●	●	
Air humidity:	45 - 60 %	●	●	●	
Equipment					
Frame:	Powder-coated steel	●	●	●	
Panel:	Plastic	●	●	●	
Seat:	Artificial leather	●	●	●	
2 independent braking systems in accordance with the German StVZO					
Accessories: XL seat, crutch holder, holder for rollators, holder for oxygen bottle, holder for transport box, lockable transport box, licence plate holder, rain cover.					

The stated measurements are subject to the manufacturing tolerances of +/- 10 mm

8. Care instruction for the user

8.1. Cleaning

The seat and back materials can be cleaned with warm water using a sponge and/or a soft brush. In case of stubborn dirt, a commercial cleaning agent can be added to the water.

Note: Never use aggressive cleaning agents such as solvents / scouring agents or hard brushes.

Plastic parts are best cleaned with a commercially available plastic cleaner. Please observe the product information of the cleaning agent.

- Plastic claddings are attacked by non-ionic surfactants and solvents, especially alcohols!
- Wipe the frame parts of the wheelchair with a damp cloth. For heavier soiling, also use a mild cleaning agent.
- Clean the wheels with a damp brush with plastic bristles (do not use a wire brush!).
- Do not use a high-pressure cleaner.
- Wash the push handles, arm pads, seat and back pads with a mild detergent.
- Upholstery parts can be machine washed at 40°C. Do not machine dry, only dry at room temperature.

Make sure that the electrics or electrical components do not come into contact with water!

8.2. Disinfection

- Before disinfection, the rollator must be thoroughly cleaned. The wheelchair can be disinfected with a household disinfectant. Use approved spray disinfectants or alcohol-based wipe-on disinfectants.
- If the scooter is used by several people, a commercially available disinfectant must be used.
- Use only wipe-on disinfectants approved by the Robert Koch Institute (RKI) and the Association for Applied Hygiene (Verbund für Angewandte Hygiene e.V.) (e.g. Bacillol AF or Kohrsolin FF); see manufacturer's recommendations for exposure time and concentration.
- Caution: Damaged upholstery parts cannot be sufficiently disinfected. These should be replaced.
- Under certain circumstances, the surfaces can be affected by disinfectants. Longer-term restrictions of the functionality can be the result.

Please note our enclosed overview of the most important care, hygiene and maintenance tasks.

Attention: Danger of infection! If there are several users, the wheelchair must be cleaned after each use.

Do not use a high-pressure or steam cleaner.

9. Check Before Use

9.1. Brake Inspection

Make sure the brake system is functioning properly before each use.

- Start off carefully and let go of the throttle lever. The scooter must come to a complete stop as usual.
- When stopped, the mechanical brakes must engage with an audible click. Should you notice any unusual braking behaviour, please notify your dealer right away and ensure the scooter is not being used.
- The motor brake is functioning correctly if you can't push the scooter when it is turned off. Should you nevertheless be able to do so, then the electro-magnetic brakes are defective. In this case you may no longer use the scooter. Contact your dealer immediately.

9.2. Checking the lights, indicators and horn

Make sure your lights, indicators and horn are functioning properly before each use.

- If a unit under inspection does not react when the corresponding button on the control panel is pushed, please notify your dealer immediately and ensure the scooter is not being used.

9.3. Inspection of Tyres

Before embarking on any journey, check that the tyres are in good condition.

- Check the tyre pressure (see Section 7).
- Carry out a regular check of the tread depth of the tyres once a month. This should be at least 1 mm.
- Any faulty or worn out tyres must be changed through your mobility dealer.

9.4. Batteries

Your scooter is equipped with maintenance-free batteries that do not require a fluid level check. If the scooter is not used for a longer period of time, the charger should be connected every 4 weeks, otherwise the batteries could be destroyed by deep discharge.

Always check the charge status of the batteries by looking at the battery status display on your control unit before setting off. If only three bars are shown on the battery status display, you should stop travelling long distances and connect the battery charger to the scooter immediately.

We recommend connecting the charger to the scooter before and after every journey.

Only use authorised chargers to charge the batteries (see technical data). Please also observe the instruction manual of the charger!

9.5. Putting into operation

The batteries of your scooter must be fully charged before first use. To do this, connect the charger supplied to the scooter (please refer to the instruction manual supplied separately with the charger for details).

The charging process can take up to 12 hours, depending on the size and charge status of the batteries. As with all future charging processes, the charge status must be read on the charger and not on the battery display of the scooter.

Depending on the lead battery technology (AGM or Gel), the maximum usable battery capacity is only reached after several charging processes (up to 10 complete charging processes). During this time, take particular care not to run the batteries completely flat and to recharge them as fully as possible after each ride.

The charge status must be read on the charger and not on the display of the scooter.

9.6. Daily use

If possible, fully charge the scooter batteries before every journey. Familiarise yourself with the function of the battery indicator and check it regularly during the journey. Plan your journeys so, that there is always enough battery charge to enable you to return to the starting point or, if necessary, to the next charging point.

If the battery charge level is very low, depending on the scooter model, an acoustic warning, e.g. a repetitive double beep, is emitted in addition to the flashing battery indicator (see chapter 10). From this point onwards, the remaining range of the wheelchair is very short - a few hundred metres to a few kilometres - please charge the batteries fully immediately! The warning tone is only deactivated after the batteries have been fully charged (see display on the charger). If the warning tone is still active despite full charging, acknowledge the warning tone as follows: With the charger still connected to the wheelchair and switched on, switch the wheelchair on using the on/off button or the driving key and switch it off again after a few seconds.


Please also observe the instructions for use of the supplied charger, which are described on the charger itself and in the operating instructions for the charger. Only use the charger supplied and approved by Bischoff & Bischoff! (Fig. 44).

During subsequent use, you can recharge the batteries at any battery level.

You can also interrupt the charging process if the batteries are not yet full, e.g. because you want to continue driving.

If possible, however, always charge the batteries to 100 % so that you have as much range as possible.


Do not charge frozen batteries.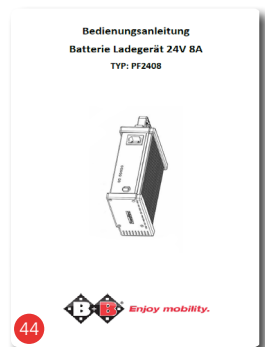

9.7. Charging procedure

The batteries must be charged between 0° and 40° C

- First switch off the scooter.
- Before starting up the charger, check that the power supply is 230 V.
- Connect the charger to the charging socket on the scooter and then to the mains socket (230 V).
- The charging socket is located on the side below the ignition lock (Fig. 45).
- Once the charging process is complete, disconnect the mains plug and then disconnect the scooter from the charger.
- When not in use, the battery should be charged once a month to prevent deep discharge.
- Deep discharging considerably shortens the service life of the battery and can also damage it.
- Batteries should be stored upright, dry and frost-free.
- As soon as the range decreases noticeably with fully charged batteries, you should replace them.

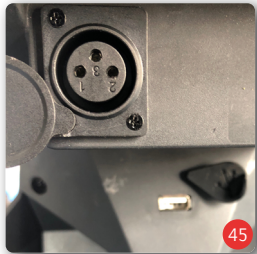Never use your scooter with empty or almost empty batteries. You could put yourself and others in great danger if you stop unexpectedly.

9.8. Replacing the batteries

Be very careful when working on the batteries. Avoid handling metal tools, touching the battery terminals and observe the instructions of the battery manufacturer.

This work should only be done by a trained technician. There is danger of fatal injury!

Only use batteries that are approved for this scooter (see Section 7). Make sure that the scooter is switched off before starting to replace the batteries. (see Section 5.7).

The batteries are located underneath the seat of the scooter and are protected by the housing cover (Fig. 46).


- Start by removing the housing cover.
- Disconnect the plug connections carefully and without using force (Fig. 47).
- Undo the screw on the mounting bracket.
- Now remove the batteries.
- Lift the new batteries into the space provided and connect the contacts. Make sure you observe the correct colour-coding (red and black, Fig. 48).
- Reattach the housing cover and the seat.
- The declaration of conformity from Bischoff & Bischoff GmbH as well as the public road licence will only be maintained by using authorised parts.

When the batteries have reached the end of their life, i.e. cannot be sufficiently charged any more, they must not be disposed of with the household waste. When new batteries are purchased, the mobility dealer takes the old batteries and disposes of them correctly.

9.9. Fuses

In addition to the overload switch (see Chap. 6.5), there is also a fuse in the battery cable (Fig. 49).

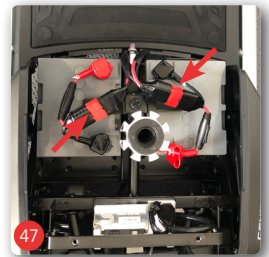10. Diagnostic codes

The following diagnostic code is only displayed when the scooter is in operation.

2x		Battery empty
-----------	---	---------------

The following diagnostic codes sound only once when switching on the scooter.

Please note: Depending on the vehicle and configuration, a 9-fold (faster) beep may sound before the actual diagnostic code.

2x		TSupply voltage too low / Batteries defective
3x		Supply voltage too high
4x		Maximum current exceeded / control unit too warm
5x		Motor brake unlocked / Motor brake faulty / Wiring faulty
6x		Rocker actuated
7x		Driving rocker faulty / Driving rocker without connection
8x		Motor faulty / Wiring of motor faulty
9x		Control unit defective

11. Maintenance

We recommend having the entire scooter inspected by a specialist dealer at least once a year. In the event of faults or defects on the scooter, it must be returned immediately to the specialist dealer for repair. On request, the specialist dealer will be provided with the necessary information, documents and circuit diagrams for repair and maintenance. A training programme for specialist dealers is available. Please contact Bischoff & Bischoff customer service if you have any questions.

Check your scooter regularly according to the maintenance schedule.

11.1. Maintenance instructions

Typical wearing parts are brakes and tyres. Contact your specialist dealer for spare parts. We recommend that you return your scooter to your specialist dealer for inspection at least once a year. In the event of faults or defects, return the scooter to your specialist dealer immediately for repair. The specialist dealer will be provided with the necessary information and documents for repair and maintenance on request.

11.2. Maintenance tips for retailers

Maintenance must be carried out in accordance with the Bischoff & Bischoff maintenance schedule. A maintenance schedule is available on request. A training programme for specialist dealers is also available.

The scooter must be checked according to the maintenance schedule and be in perfect working order.

12. Repairs

Repairs and maintenance work may only be carried out by authorised partners (medical supply stores or specialist dealers). Please call our customer service to find a partner in your area. The customer service contact details can be found on the back of these operating instructions.

13. Transport

Your scooter is not suitable for transport on local public transport in the buses designed for this purpose (Fig. 50).

The Centuro S7 is not suitable for use as a vehicle seat. It must not be used as such under any circumstances. It does not fulfil the requirements of ISO 7176-19 and failure to do so may result in serious injury or death in the event of an accident.

14. Parking

Do not park your electric scooter in damp or humid conditions, as this will lead to mould growth and rapid deterioration of the upholstery parts. Also ensure that the drives are engaged (see chapter 6.5).

15. Hand over the scooter

The scooter is suitable for reuse. Before it is passed on to another user, it must be serviced and sanitised by a specialist dealer and hygienically reconditioned. When passing on the scooter please remember to pass on these operating instructions and the address of your specialist dealer to the new user.

16. Use by others

The scooter is designed for reuse by other persons. Before it is passed onto another user, it must be serviced by a dealer and hygienically cleaned. When passing on the scooter, don't forget to hand over this operating manual and the address of your dealer to the new user.

17. Storage/Shipping

If the scooter is to be stored or shipped, the motor brake must be disengaged (see Section 6.5). All detachable and unfastened parts must be removed and packed individually in suitable cardboard boxes. The individually packed parts can then be packaged together in a larger cardboard box.

We recommend keeping the original packaging and storing it with these purposes in mind so that it is available if required. Your scooter then has optimum protection against environmental influences during storage or transport.

18. Disposal

If your scooter is no longer being used and is to be disposed of, please contact your mobility dealer.

If you wish to dispose of the product yourself, contact local waste management companies to enquire about the waste disposal regulations that apply to your place of residence.

19. Warranty

1. For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from production and material defects. In case of evidence of material or manufacturing defect, defective parts will be replaced free of charge.
2. This warranty excludes claims for the following reasons:
 - a) normal wear and tear, which include but is not limited to the

following parts where fitted; Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.

- b) overloading the product such as, for example, by exceeding the maximum user weight or load.
 - c) the product or part has not been maintained or serviced in accordance with the operating manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.
 - d) Accessories have been used which are not original accessories.
 - e) The product or part has been damaged through negligence, accident or incorrect use.
 - f) changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
 - g) Repairs have been carried out, before our customer service has been informed of the circumstances.
3. To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.
4. For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1).
5. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.
6. If the repair fails after a reasonable time, the customer can, at his discretion, request a reduction, compensation or the termination of the purchase contract.
7. The guarantee is subject to the law of the country in which the product was purchased from Bischoff & Bischoff & Bischoff.

Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your qualified mobility dealer will be happy to provide these to you.

Warranty certificate

Product: Centuro S7 /XXL / MAX

Serial number:*

* (must be inscribed by the mobility dealer)

Dealer:

Date and seal

(Please note our general business terms and conditions)

Non-observance of the operating instructions as well as improper use, maintenance jobs and technical changes or adaptations without the permission of the Bischoff & Bischoff company, lead to guarantee loss and the loss of product liability in general.

Note!

In spite of careful research and editing, incorrect information may have crept into our operating manuals. Rights to technical changes reserved. Illustrations may include accessories. The current version of each instruction manual is ready for download at the Bischoff & Bischoff website:

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