Bedienungsanleitung
Instruction manual
Manual de instrucciones

Ejoy RD
Ejoy RD XL
Ejoy RD XXL

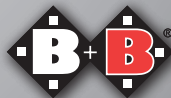
DE

EN

ES

Enjoy mobility.

Inhalt

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Contents

1.	Preface	64
1.1.	Signs and symbols	65
2.	Important safety instructions	65
2.1.	General safety instructions	65
2.2.	Safety in vehicles	67
2.3.	Use on public roads	67
2.4.	Notes on electromagnetic interference	67
2.5.	Brakes	68
2.6.	Purpose	69
2.7.	Indications	69
2.8.	Contraindications	70
2.9.	Declaration of conformity	70
2.10.	Service life	70
2.11.	Notes on range	71
3.	Description of Product and Contents	72
3.1.	Inspection on Delivery	72
3.2.	Contents	72
3.3.	Overview	73
3.4.	Type plate and serial number	74
3.5.	Basic setup	75
3.5.1.	Mechanical legrests	75
3.5.2.	Horizontally adjustable legrests (optional)	75
3.5.3.	Electronic legrests (optional)	77
3.5.4.	Seat	77
3.5.5.	Electronic adjustment options	80
3.5.6.	Headrest (optional)	80
3.5.7.	Slack in the backrest	80
3.5.8.	Sideguards	81
3.5.9.	Control unit	82
3.5.10.	Lap belt	83
4.	Using the wheelchair	83
4.1.	Functional description of the VR2 control unit	84
4.2.	Functions of the control panel	86
4.3.	Functional description of the VR2 control unit for electronic adjustment options (optional)	88
4.4.	Functional description of the VR2 attendant control unit	90

4.5.	Locking and unlocking of the wheelchair	92
4.6.	Functional description of the R-NET control unit	93
4.7.	Functional description of the R-NET control unit for electronic adjustment options (optional)	94
4.8.	Functional description of the R-NET attendant control unit	95
4.9.	Locking and unlocking of the wheelchair	97
4.10.	Adjustments to the Control unit	98
4.11.	Getting in and out of the chair	98
4.12.	Driving the wheelchair	99
4.13.	Pushing mode	100
4.14.	Legrests	102
4.15.	Lifting and holding points	102
5.	Accessories	103
6.	Technical Data	104
7.	Maintenance instructions for the user	107
7.1.	Cleaning and disinfection	107
7.2.	Check Before Use	108
7.2.1.	Checking the brakes	108
7.2.2.	Checking the lighting, indicators and horn	108
7.2.3.	Checking the tyres	109
7.3.	Batteries	109
7.3.1.	Charging batteries	110
7.3.2.	Replacing the batteries	111
7.3.3.	Battery, fuses	112
8.	Maintenance instructions	113
9.	Repairs	113
10.	Transport	114
11.	Storage and shipping	114
12.	Use by others	115
13.	Service life	115
14.	Disposal	115
15.	Warranty	116

1. Preface

Dear user,

You have chosen a high-quality electric wheelchair from Bischoff & Bischoff and we would like to thank you for your trust.

The electric wheelchair has been designed to offer many advantages and meet your needs. Through the wide range of adjustment options and the very best adaptability with regards to lower leg length, seat and backrest angle as well as the control unit which can be fitted on the right or left, the wheelchair can be adapted to your individual body measurements. The Ejoy can be delivered with two different control units. In the standard configuration, the Ejoy is supplied with the VR2 control unit. As an option, this offers you the opportunity to adjust up to two components electronically. The R-NET control unit allows for electronic adjustment of the legrests, seat angle and backrest angle. In this way, the Ejoy can be adapted easily to your particular needs.

The electric wheelchair is designed for use indoors or for driving outdoors.

Please read and follow the Instruction manual carefully before using your new electric wheelchair for the first time. The Instruction manual is an integral and necessary component of this product. Keep the Instruction manual where you can refer to it and if you transfer the wheelchair, give the manual to the new owner.

For visually impaired users, this document is available as a PDF file from our website www.bischoff-bischoff.com.

Note!

In spite of careful research and editing, incorrect information may have crept into our Instruction manuals. We reserve the right to make technical modifications. Illustrations may include accessories. The current version of each Instruction manual is ready for download on our website:

www.bischoff-bischoff.com

1.1. Signs and symbols

Important! This is especially useful information on a given subject.

Warning! These are especially safety-relevant instructions. Observe the instruction manual!

2. Important safety instructions

2.1. General safety instructions

Before setting off, check that all attachments are properly fixed in place.

- To avoid falls and dangerous situations, you should first practice driving your new electric wheelchair on level, unobstructed ground. An attendant is recommended in this case.
- Familiarise yourself thoroughly with the braking and acceleration behaviour when driving in a straight line and into and out of curves.
- Please note that the shifting of weight through body movements or loading can increase the risk of tipping over to the side, to the front or back.
- The electric wheelchair must only be used to transport one person. Please respect the intended use (see Chapter 2.6).
- When setting up or adjusting the product, be aware of the risk of body parts being pinched.
- Never use the electric wheelchair under the influence of alcohol or other agents which affect your awareness or your physical and mental capacities.
- It is essential that you use ramps to get over obstacles. If it should be necessary to lift the Ejoy over an obstacle, then only do this

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

without an occupant. Only use fixed frame parts to lift the chair.

- Avoid driving into obstacles without braking (furniture, steps or similar items).
- The control unit must always be switched off whenever you are getting in or out of the chair.
- When on roads, observe the road traffic regulations.
- The lighting and reflectors on the wheelchair must always be visible to other road users.
- Where possible, wear bright conspicuous clothes. You will then be seen better by other road users.
- Please note that upholstery parts which are subject to sunlight will heat up and could cause skin injury when touched. You should therefore cover these parts or protect the wheelchair from direct sunlight.
- The seating upholstery and pads' materials meet the flammability resistance requirements in accordance with EN 1021-2. Therefore do not expose the upholstery or pads or any other parts to ignition sources - such as, for example, cigarettes.
- Always put the seat and backrest into an upright position when you are driving the wheelchair.
- Use on slopes is only permitted when the seat and backrest are in an upright position.
- At low ambient temperatures, the metal parts of the wheelchair can get very cold, so please avoid skin contact if this is the case.
- If you should notice your electric wheelchair behaving unusually or if it is damaged, stop using it and contact your dealer.
- Reporting incidents.

If you, as the operator, user or their relatives, detect possible defects or functional limitations, please contact an authorised dealer immediately. Operators, users or their relatives should inform the dealer who supplied the dealer who provided you with the product of any suspected serious incidents which directly or indirectly have had or could have one of the following and by which they are affected. You can also report this to the competent higher federal authority report - the death of a patient, user or other person,

Electric wheelchair | **Ejoy RD**
Electric wheelchair | **Ejoy RD XL**
Electric wheelchair | **Ejoy RD XXL**

- the temporary or permanent serious deterioration of the health of the state of health of a patient, user or other persons,
- a serious risk to public health.

2.2. Safety in vehicles

The Ejoy is not designed to be used as a vehicle seat. Under no circumstances may it be used as such. It does not fulfil the requirements in compliance with ISO 7176-19. Violation can cause the most serious injuries resulting in death in the case of an accident (Fig. A).

2.3. Use on public roads

The wheelchair is designed for indoor and outdoor use. Please keep in mind that you are using public roads and therefore have to observe all road traffic regulations.

Do not endanger other road users through inconsiderate driving, especially on pavements.

2.4. Notes on electromagnetic interference

Despite compliance with all applicable EMC Directives and standards, it is possible that the electric wheelchair may interfere with other electrical devices (e.g. mobile phones, alarm systems) or that its driving performance will be reduced by those devices.

If you notice such behaviour, switch off either the third-party device or your electric wheelchair and move it beyond the range of interference.

It is possible that your electric wheelchair will interfere with other devices which emit electromagnetic fields and that they will interfere with your chair.

2.5. Brakes

Operating brake:

When in motion the motors act as an operating brake. To brake the wheelchair slowly, move the steering and driving lever (joystick) slowly back to the neutral position (initial position).

For more forceful braking, release the steering and driving lever.

The braking distance is significantly longer on slopes than on flat ground. Bear this in mind when going down ramps and brake in time.

Make sure that you brake your electric wheelchair in good time. Particularly when approaching people or when driving downhill.

Parking brake/parking:

Switch the wheelchair off using the on/off switch (see Chapter 4.1).

If the drives are engaged, you cannot push or move the electric wheelchair (see Chapter 4.6).

Pushing mode:

In order to be able to push the wheelchair, pull the release levers on both sides of the wheelchair upwards (see Chapter 4.7), if the wheelchair is switched on, then the battery status display will flash. For pushing mode you can switch the wheelchair off.

Please note that push mode by disconnecting the electromagnetic brakes must only be used on flat ground. The electromechanical brake is then disabled. Non-observance means danger to life.

Emergency brake:

Press the on/off switch to initiate an emergency stop when driving.

This will lead to intense deceleration. This can lead to injuries and is therefore only to be used in emergencies.

An emergency stop must only be carried out in an emergency as it can lead to injuries.

2.6. Purpose

The Ejoy electric wheelchair is designed for indoor and outdoor use for adults who cannot walk. Unless otherwise described, the instructions in this instruction manual are directed at the occupant of the chair as the operator. The Ejoy electric wheelchair is able to handle longer distances. Suspension is provided to overcome small obstacles. It is classified as Category B in accordance with DIN EN 12184.

The maximum payload is:

Ejoy RD 140 kg
Ejoy RD XL 170 kg
Ejoy RD XXL 200 kg
Ejoy RD XXL 250 kg

Please note: We can only provide a warranty if the product is used under the conditions specified and for the intended purposes.

2.7. Indications

Provision of this electric wheelchair is suitable for adults with significant through to fully-developed impairment of mobility/walking with structural and/or functional damage of the lower extremities, e.g. through

- amputation

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

- paralysis
- loss of limb
- limb defects/deformity
- joint contractions
- joint damage
- musculoskeletal/neuromusculoskeletal movement disorders
- other illnesses

Provision of electric wheelchairs is generally only advised where use of manual wheelchairs is no longer possible because of the disability and the electric wheelchair can be used properly.

2.8. Contraindications

Provision of this electric wheelchair is not suitable for children, and for adults with:

- severe equilibrium disorders
- impaired and insufficient vision
- strongly impaired cognitive abilities

2.9. Declaration of conformity

We Bischoff & Bischoff GmbH declare under our sole responsibility, that the Ejoy wheelchair meets all the requirements of MDR 2017/745 that are applicable.

2.10. Service life

The expected service life of the electric wheelchair is up to five years, if it is used correctly, and the safety, care and maintenance instructions are followed. The electric wheelchair can be used beyond this period if it is in a condition allowing its safe usage.

Electric wheelchair | **Ejoy RD**
Electric wheelchair | **Ejoy RD XL**
Electric wheelchair | **Ejoy RD XXL**

NE

2.11. Notes on range

The range of the Ejoy is up to 35 km. The achievable range depends on different factors:

- condition of the road surface
- way of driving
- battery condition and charge
- payload weight
- ambient temperature
- use of accessories

Please note that the range stated by us was calculated under test conditions:

- batteries which are as new and fully charged
- ambient temperature of 21 °C
- max. user weight
- level, firm ground
- constant driving speed
- deactivated lighting system

We would like to point out that the following factors will mean that the range is severely reduced.

- poor battery charge level
- low ambient temperature (below 10 °C)
- frequent starting and braking
- climbing hills
- driving on ground which is not firm (field or forest paths)
- incorrect or low tyre pressure (see Chapter 6)
- electromagnetic fields (see Chapter 2.4)

With a combination of several factors the range may be reduced by up to 50%.

Never use your electric wheelchair with empty or almost empty batteries. You may endanger yourself and others by stopping unexpectedly.

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

3. Description of Product and Contents

3.1. Inspection on Delivery

All Bischoff & Bischoff GmbH products undergo a proper final inspection in our company and are labelled with the CE mark (see Chapter 3.3).

The wheelchair will be delivered in a special cardboard box. Keep the box after unpacking, if possible. It is useful for later storage of the product or for sending it back.

Delivery and first instructions are taken care of by your qualified medical supply dealer.

If the product is shipped by rail or road, the goods should be checked immediately for transport damages in the presence of the bearer.

Check that the delivery is complete and undamaged (see Chapter 3.2). Should you find any irregularities or damage, please contact our customer service. You can find the contact details for Customer Service on the back cover of this instruction manual.

3.2. Contents

After receipt of the goods, please check the contents for completeness right away. The contents consist of:

- packaging
- wheelchair
- legrests
- charger
- any accessories
- this Instruction manual

Electric wheelchair | **Ejoy RD**
 Electric wheelchair | **Ejoy RD XL**
 Electric wheelchair | **Ejoy RD XXL**

NE

3.3. Overview


- A Backrest tube
- B Back cushion
- C Control unit
- D Armrest
- E Sideguard
- F Seat cushion
- G Headlight
- H Indicators
- I Drive wheels
- J Castors
- K Legrests
- L Footplate

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

3.4. Type plate and serial number

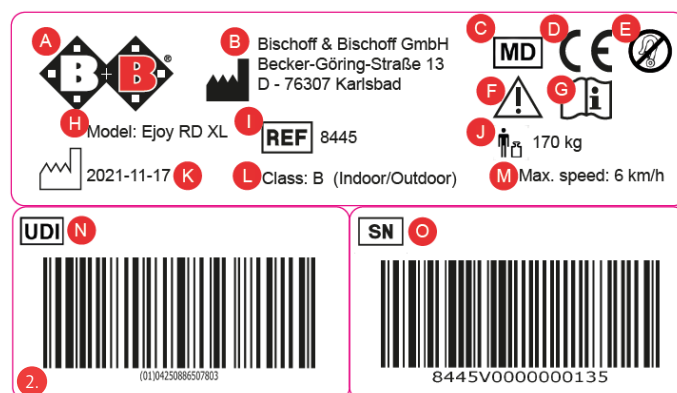The type plate (Fig. 2) and serial number are located under the seat surface. The values given here serve as an example only. Please see the type plate on your wheelchair for the exact information.

- A Manufacturer logo
- B Manufacturer information
- C Medical device
- D CE mark
- E Vehicle fixing
- F Warning! - read the User manual
- G Important! - read the User manual
- H Model designation
- I Model number
- J max. user weight
- K Date of manufacture
- L Classification in accordance with DIN EN 12184
- M maximum speed
- N UDI number
- O Serial number

Electric wheelchair | **Ejoy RD**
 Electric wheelchair | **Ejoy RD XL**
 Electric wheelchair | **Ejoy RD XXL**

NE

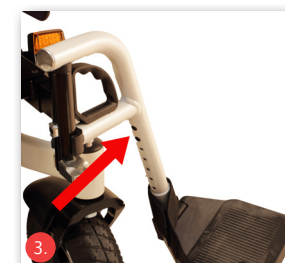
3.5. Basic setup

Before using it for the first time, adjust the wheelchair to suit your body size. Adjusting the wheelchair allows long periods of fatigueless driving in a physiologically correct position. The basic setup of the wheelchair is described in this section. You should carry out this work in collaboration with the medical equipment retailer.

3.5.1. Mechanical legrests

Lower leg length


- The lower leg length can be adjusted.
- To do this loosen the two Allen screws (Fig. 3).
- Adjust to the required height.
- Re-tighten the screws.

For secure fixing, make sure that two screws are always used!

Footplate angle

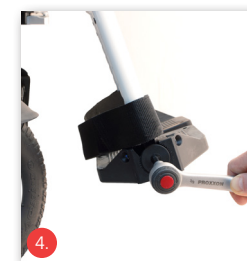
- The angle of the footplate can be adjusted.
- Loosen the screw fixing on the side of the footplate (Fig. 4).
- Adjust the footplate to the required angle.
- Re-tighten the screw.

3.5.2. Horizontally adjustable legrests (optional)

Assemble, disassemble and swing them away as for the standard legrests.

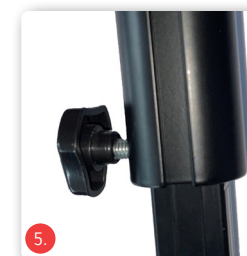To adjust, loosen the star clamping screw, then adjust the legrest to the required length and then re-tighten the star clamping screw.

The horizontal adjustment works as follows:

- Push the locking lever forwards
- by lifting or lowering the footplate receiver tube you can move the legrest to the required position
- Release the locking lever; legrest locks into place automatically.

Never stand on the footplate and when swinging it away, watch your fingers as there is a risk of pinching.

To change seat, the legrest must be swung away or removed. Never sit on the legrest. Risk of tipping over!

Footplate angle

- Adjusting the footplate is exactly the same as for the mechanical legrests (see Chapter 3.5.1).

Height of the calf pad

- The height of the calf pad is variable.
- Loosen the screw fixing on the side of the legrest.
- Adjust the calf pad to the required height (Fig. 6).
- Re-tighten the screws.

Depth of the calf pad

- The depth of the calf pad is variable.
- Loosen the screw fixing on the side of the calf pad.
- Adjust the calf pad to the required depth (Fig. 7).
- Re-tighten the screws.

3.5.3. Electronic legrests (optional)

Lower leg length

The lower leg length can be adjusted in eight steps.

- To do this, loosen the Allen screw and then pull it out,
- adjust to the required height,
- re-tighten the screw.

Footplate angle

- Adjusting the footplate is exactly the same as for the mechanical legrests (see Chapter 3.5.1).

Make sure there is a reasonable gap between footrests and the floor.

Height of the calf pad

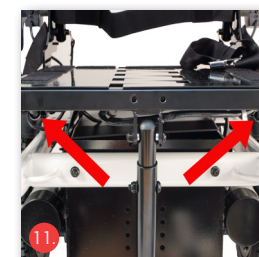
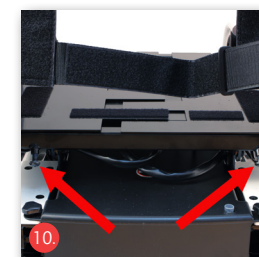
- Adjusting the calf pad is the same as for the horizontally adjustable leg support (see Chapter 3.5.2).

3.5.4. Seat


Seat width

Adjusting the seat width:

- Make sure that the wheelchair is switched off.
- Remove the attachments such as the back cushion and seat pad. The back cushion and seat pad are secured with hook and loop fastenings to prevent them slipping.
- Loosen the hook and loop fastener of the backrest sling and remove it (Fig. 8/9).
- Remove both fixing screws on the sleeve of the push handle (Fig. 9).
- Then loosen the front and rear screws on the underside of the seat surface (Fig. 10/11).
- First adjust one of the seat halves (left or right) and then the other one after that. When moving the seat halves, make sure that the sleeve on the push handle (Fig. 9) does not tilt during the proce-


Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

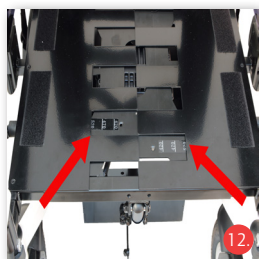- sure, if necessary, rotate the sleeve slightly so that the push handle tubes become loose again. Adjust both sides to the same level.
- First move just one seat half to the required width, you can use the number scale on the seat surface as a guide (Fig. 12).
 - Then re-tighten the front and rear fixing screws for this side under the seat.
 - Then push the other seat half to the required width.
 - Re-fix the screws under the seat and then the screws of the sleeve on the push handle. Use medium-strength screw lock liquid to do this.
 - You can re-tighten the hook and loop fasteners (Fig. 9) and refit all other attachments.
 - You should then use a readjusted seat cushion and back cushion.

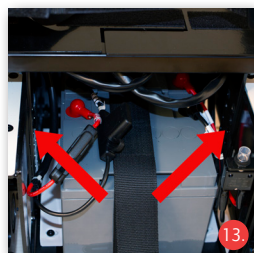Make sure that both halves of the seat are adjusted evenly.

Seat height

To adjust the seat height, loosen the screws in the right and left brackets and adjust to the height required. Please use medium-strength lock liquid to fix the screws in place (Fig. 13/14).

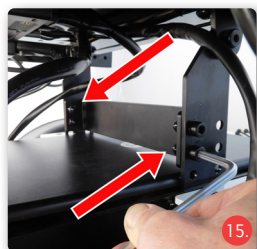
Seat height adjustment Reinforcement strut

If it is necessary to adjust the seat height, the supplied reinforcement strut must be used.

This is to be fitted when the seat height is raised to such an extent, that there are 2 holes on top of each other on the left and right side of the seat brackets above the battery cover, so that the reinforcement strut can be fitted.

If the seat height is to set even higher, always use the 2 holes on top of each other in the seat brackets, so that the reinforcement brace is directly above the battery cover (Fig. 15).

For these adjustments please contact your specialist dealer.

Electric wheelchair | **Ejoy RD**
 Electric wheelchair | **Ejoy RD XL**
 Electric wheelchair | **Ejoy RD XXL**

NE

Seat depth

- The seat depth can be adjusted.
- To adjust the seat depth, loosen the backrest fixings, push the backrest forwards or backwards and then re-tighten the screws (Fig. 16).

Seat angle - mechanical

This description only applies if you have a configuration with a mechanically-adjustable seat angle.

- The default setting for the seat angle is 0° to the back.
- The seat angle can be adjusted backwards in steps.
- To do this loosen the fixing screws on the telescopic profile under the front of the seat (Fig. 17).
- Pull the screws out and adjust to the required seat angle.
- Put the screws into the next possible position and re-tighten them. Use medium-strength screw lock liquid to do this.

Backrest angle - mechanical

This description only applies if you have a configuration with a mechanically-adjustable backrest angle.

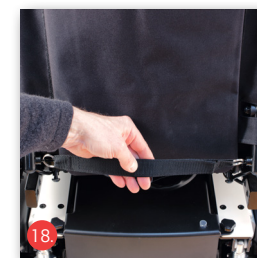
- The backrest angle of your electric wheelchair can be adjusted easily. You also have the option of folding the backrest fully forwards.
- In order to fold the backrest, pull the locking pins using the strap on the backrest and then fold the backrest down (Fig. 18).
- To adjust to the required backrest angle, pull the strap backwards. When doing this, hold the backrest firmly with your other hand.
- Adjust the backrest to the required angle. When you release the wire, the bolts will click into the holes.

Make sure that both bolts, right and left are completely in place.

Please note that after these adjustments the tipping stability may deteriorate.


Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

3.5.5. Electronic adjustment options

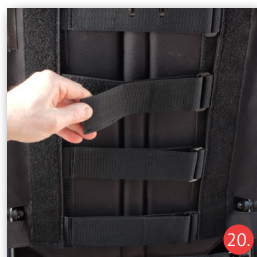Depending on the configuration, it is also possible to adjust the seat angle, backrest angle or angle settings on the footrests electronically. Please refer to Chapter 4, "Using the wheelchair" (adjustment options) for information.

Bear in mind the increased risk of pinching with electronic adjustment of seat functions for the user, attendant and third party.

3.5.6. Headrest (optional)

As an option, a headrest is available which is angle, height and depth adjustable (Fig. 19).

- Loosen the screws on the stabilising bar
- Position the headrest at the required height
- Re-tighten the screw fixing. You can adjust the depth and angle of the headrest using the quick-release device in the neck area of the headrest
- Position the headrest to the required depth
- Re-tighten the screw fixing.

3.5.7. Slack in the backrest

A well-adjusted backrest will make it easier to sit for long periods of time more comfortably and effectively reduces the risk of pressure points. To achieve this, make sure there is sufficient slack in the pelvic area.

Electric wheelchair | **Ejoy RD**
 Electric wheelchair | **Ejoy RD XL**
 Electric wheelchair | **Ejoy RD XXL**

NE

- Remove the cushion support of the backrest sling.
- Loosen the hook and loop fasteners on the belt straps (Fig. 20).
- Adjust the straps in sequence, starting from the bottom.
- Put the cushion support of the backrest sling back in place.

3.5.8. Sideguards

Make sure that the wheelchair is switched off before removing the sideguards.

The sideguards can be completely removed for side transfers. Note that the sideguards can be removed but not completely separated without disconnecting the cable for the control units. In addition the height of the armrests can be adjusted. You can also adjust sideways.

Removing sideguards

- To remove the sideguards, loosen the clamp lever screw (Fig. 21) and pull the armrest out upwards.
- When fitting the sideguards make sure that they are pushed in to the limit and fixed back in place.

Adjusting the height

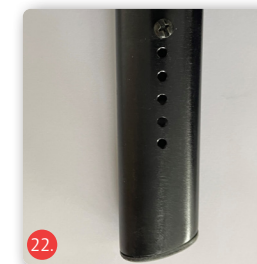
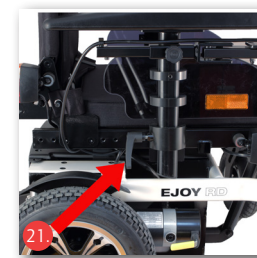
- First take the sideguards out of the bracket.
- Remove the check screw (Fig. 22) and screw it back into the required position in the armrest.
- The lowest hole is the maximum height setting for the armrest.

Moving sideways

- Loosen the fixing (Fig. 23) under the seat and push the sideguard to the required position.
- Re-tighten the fixing.

Length adjustment of sideguard

Loosen the fastening screw (Fig. 24) underneath the seat. Pull out the side part and insert it into the next guide tube. Tighten the fastening screws firmly again.

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

Length adjustment of armrest

- First take the sideguards out of the bracket.
- Loosen the armrest screws (Fig. 25).
- Move these to the required position
- Screw the armrest firmly back onto the sideguard.

Make sure that the adjustment is only within the marked area!

3.5.9. Control unit

The control unit can be fitted below the armrest, either on the right or left. The adjustment must be carried out by your dealer wheelchair, to guarantee that the wheelchair operates safely.

Note that the cable to the control unit must be relocated. Consult a specialist to do this.

Armrest gap

You can adjust the position of the control unit to your forearm length

- Loosen the hand screw on the outside of the sideguard
- Move the control unit to the required position and re-tighten the screw (Fig. 26).

Swivel

- The control unit can be swivelled to the side (Fig. 27). It clicks into the armrest extension and can be swung fully outwards.

Using the control unit is explained in Chapter 4, "Using the wheelchair".

Electric wheelchair | **Ejoy RD**
 Electric wheelchair | **Ejoy RD XL**
 Electric wheelchair | **Ejoy RD XXL**

3.5.10. Lap belt

The lap belt is fitted to the seat frame.

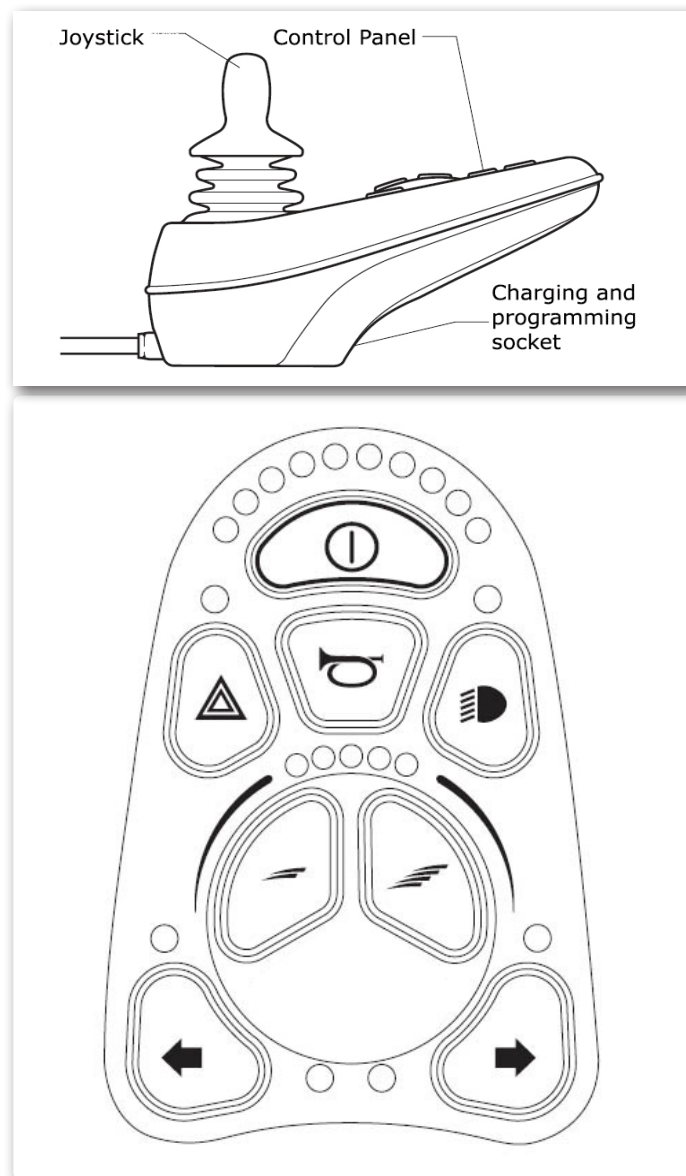
- Adjust the belt length to suit your requirements, by positioning the buckle accordingly.
- Fasten the belt with the buckle (Fig. 28).
- Open the belt by pressing the red button on the buckle.

4. Using the wheelchair

Once you have adjusted the wheelchair to your personal needs, familiarise yourself with the functions of the control unit. This is described in Chapter 4.1.

4.1. Functional description of the VR2 control unit

Using for the first time

Sit on the seat of your Bischoff & Bischoff wheelchair and place your feet on the footplates.

Make sure that you are sitting as far back in the seat as possible. Make sure that loose clothing such as, for example, scarves, shirts, etc. do not hang out of the wheelchair to the side where they could get caught in the wheels.

Switch your wheelchair on using the on/off button (Fig. 27). You can now operate the control unit as described in this chapter.

Joystick

The main function of the joystick is to control the speed and direction of the wheelchair. The further you push the joystick from the central position, the faster the wheelchair will go. When you move the joystick back, the brakes are applied automatically (see Chapter 2.5).

If the wheelchair is fitted with electronic adjustment options, the joystick can also be used for selection and adjustment, see Section 4.3 for further details.

Control panel

All electronic functions for the wheelchair are operated using the control panel. The on/off button (Fig. 27) powers the electronics for the control system, which in turn powers the motors of the wheelchair. Do not use the on/off button to stop the wheelchair, unless it is an emergency. Doing this reduces the service life of the drive components. The battery display indicates that the wheelchair is switched on.

The functions of the control panel are described in Chapter 4.2.

Charging and programming socket

Using the charging and programming socket is described in Chapter 7.

4.2. Functions of the control panel

ON/OFF button

Use this button (Fig. 29) to switch your wheelchair into drive mode. The button must not be pressed when driving, as this will trigger an emergency stop (see Chapter 2.5).

Hazard lights

Use this button (Fig. 30) to switch the hazard lights on in emergency situations when the wheelchair is not fit to drive. All four hazard lights will then flash and indicate to other road users that your wheelchair is not fit to drive. When switching the control unit off, the hazard lights will remain on and can only be switched off when the control unit is reactivated.

Indicators (left/right)

Use this button to switch on the left or right indicator of your wheelchair (Fig. 31), to indicate to other road users that you would like to change your direction to the left/right. By pressing this button again, you will switch the indicators back off.

Battery status display

You can check the battery's charge level on the battery status display (Fig. 32). If the display is in the green area, then sufficient charge is available. If the display is in the red area, then only short journeys are possible and the battery should be recharged (see Charging batteries). If the battery display is constantly lit, then the electric wheelchair is fully functional. If there is a fault, this is shown by rapid flashing. First check

- the battery charge level and
- whether the drives are engaged.

If the battery charger continues to flash, contact your dealer.

Headlight

Use this button (Fig. 33) to switch on the active lighting of your wheelchair at dusk or when going through dim areas such as, for example, tunnels. You are then more visible to other road users.

Speed level display

The speed level display (Fig. 34) shows the pre-set speed level. The possible settings are from one to five LED lamps. The speed levels indicate the maximum achievable speed when deflecting the joystick, i.e. in speed level five, the highest maximum speed is achievable and in level one, the lowest.

Adjusting the speed level

Use these buttons (Fig. 35) to increase (left) or decrease (right) the speed level, i.e. the maximum achievable speed. Speed regulation is through operation of the joystick.

Horn

The horn (Fig. 36) enables you to draw attention to yourself in difficult situations.

Immobiliser

In order to protect your wheelchair against unauthorised use or theft, park the wheelchair in an enclosed space or activate the immobiliser.

- With the control unit switched off, press the ON/ON/OFF button until you hear a beep. Then release the button.
- Push the joystick in the direction of travel, then against the direction of travel and then into the neutral position. A beep will sound each time. The control unit is now disabled.
- To re-enable the control unit
- Switch the control unit on using the ON/OFF button.
- Push the joystick in the direction of travel, then against the direction of travel and then into the neutral position. A beep will sound each time.

29.

30.

31.

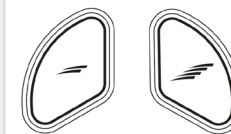
32.

33.

34.


35.

36.

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

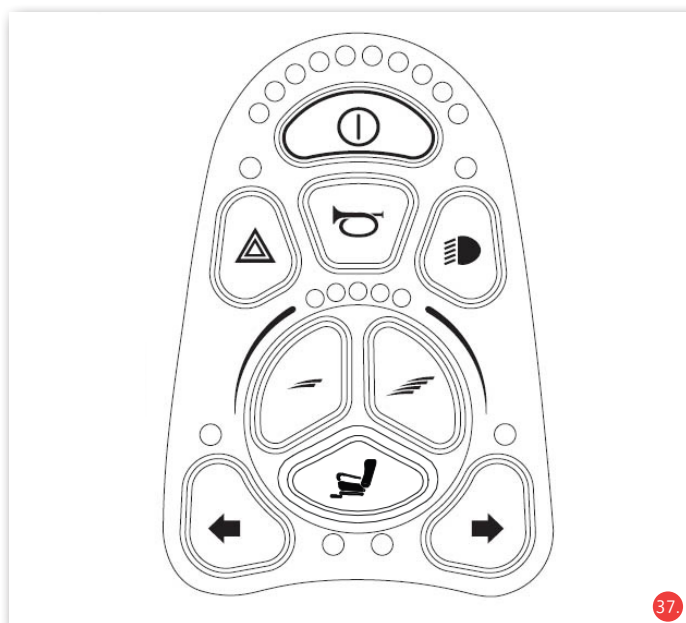4.3. Functional description of the VR2 control unit for electronic adjustment options (optional)

Electric wheelchair | **Ejoy RD**
 Electric wheelchair | **Ejoy RD XL**
 Electric wheelchair | **Ejoy RD XXL**

Controlling the adjustment options

By pressing the adjustment function button (Fig. 38) the electronic adjustment options are activated. By pressing the button once the LED display below the button is activated.

Wheelchairs with one adjustment option

Pressing the adjustment function button (Fig. 38) once activates the electronic adjustment options. This is indicated by both LEDs below the adjustment function button lighting up.

By deflecting the joystick forwards or backwards, you can control the adjustment direction for this channel.

To return to drive mode, either press the adjustment function button (Fig. 36) or one of the two the speed level buttons (Fig. 35).


Wheelchairs with two adjustment options

By pressing the adjustment function button (Fig. 38) the electronic adjustment options are activated. By pressing the button once, the left LED lights up, and then by deflecting the joystick forwards or backwards you can control the adjustment direction for this channel.

Select between the two adjustment options by deflecting the joystick to the left or right. If the other adjustment option is selected, the relevant LED will light up. LEFT for adjustment option 1 and RIGHT for adjustment option 2.


To return to drive mode, press the adjustment function button (Fig. 38) again or press one of the two speed level buttons. (Fig. 35).

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

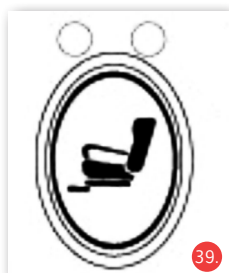4.4. Functional description of the VR2 attendant control unit

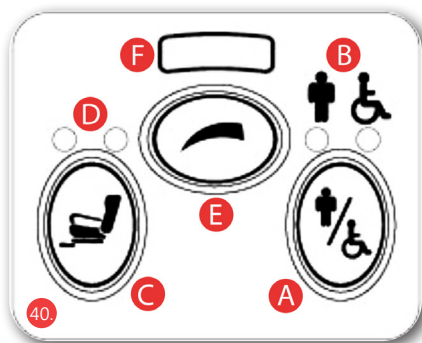As an option the electric wheelchair can be fitted with an attendant control unit.

- A** Toggle switch for operating mode
- B** Operating display
- C** Adjustment function button
- D** Display for activated adjustment option
- E** Speed level adjustment button
- F** Speed level display

Toggle switch for operating mode

Use the toggle switch (A) to select the operating mode, whether the wheelchair will be controlled by the attendant control unit or by the user. The selected operating mode will be shown through the LEDs (B). The red LED lights up when the wheelchair is being controlled by the user. The green LED lights up when the wheelchair can be controlled by the attendant.

Electric wheelchair | **Ejoy RD**
 Electric wheelchair | **Ejoy RD XL**
 Electric wheelchair | **Ejoy RD XXL**

Adjustment function button

All VR2 attendant control units are fitted with an adjustment function button (C) as standard. If the VR2 control unit is not programmed with any adjustment options, this button has no function. However if the VR2 control unit is programmed with 1 or 2 adjustment options, the functions for this button are described in the relevant section below.

Wheelchairs with one adjustment option

Pressing the adjustment function button (C) once will activate the electronic adjustment options. This is indicated by both LEDs above the adjustment function button lighting up (D). By deflecting the joystick forwards or backwards, you can control the adjustment direction for this channel. To return to drive mode, either press the adjustment function button (C) or the speed level button (E).

Wheelchairs with two adjustment options

By pressing the adjustment function button (C) the electronic adjustment options are activated. By pressing the button once, the left LED lights up, and then by deflecting the joystick forwards or backwards you can control the adjustment direction for this channel. Select between the two adjustment options by deflecting the joystick to the left or right. If the other adjustment option is selected, the relevant LED will light up. LEFT for adjustment option 1 and RIGHT for adjustment option 2. To return to drive mode, press the adjustment function button (C) again or press one of the two speed level buttons. (Fig. 33).

Speed levels display

Indicates the adjustment of the speed level for the wheelchair, if the attendant control unit is activated. There are five settings - Setting 1 is the lowest speed and setting 5 is the highest possible speed.

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

Button for speed level adjustment

Use this button (E) to adjust the speed level of the wheelchair, if the attendant control unit is activated. By pressing this button, the maximum speed of the relevant drive speed will be increased by one level, until you reach level 5. Pressing again will reset the speed back to level 1. If the control unit is programmed for operation with drive profiles, only the speed level within the selected profile will be adjusted using the button for speed level adjustment of the positioning system.

4.5. Locking and unlocking of the wheelchair

The VR2 control system can be locked to prevent unauthorised use. The locking method is by a sequence of button presses and joystick movements as described below.

To lock the wheelchair

- With the control system switched on, press and hold the on/off button.
- After 1 second, the control system will beep. Now release the on/off button.
- Move the joystick forward until the control system beeps.
- Move the joystick backwards until the control system beeps.
- Release the joystick, you will hear a long beep.
- The wheelchair is now locked.



To unlock the wheelchair;

- Switch on the control system with the on/off button. The maximum speed/profile display moves up and down.
- Move the joystick forward until the control system beeps.
- Move the joystick backwards until the control system beeps.
- Release the joystick, you will hear a long beep.
- The wheelchair is now unlocked.

Electric wheelchair | **Ejoy RD**
 Electric wheelchair | **Ejoy RD XL**
 Electric wheelchair | **Ejoy RD XXL**

NE

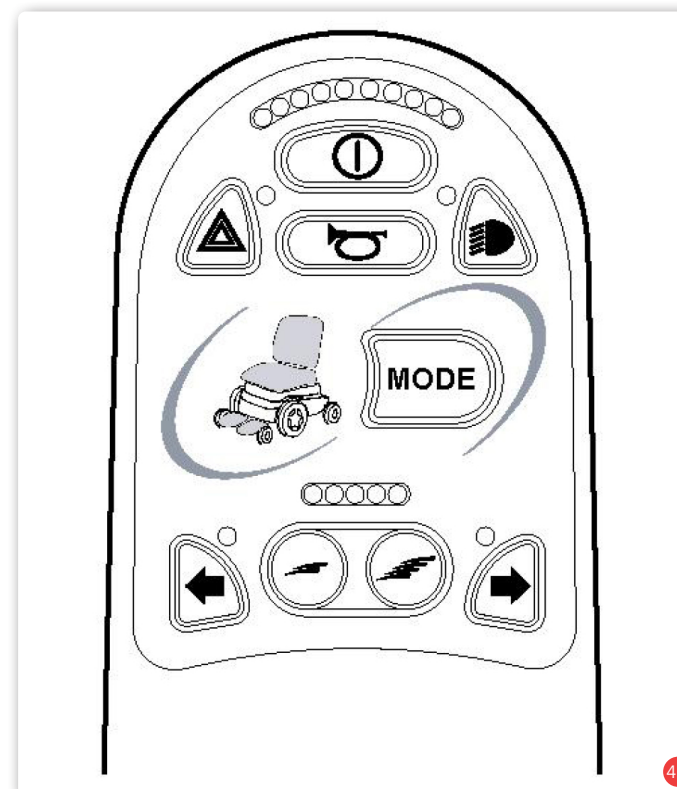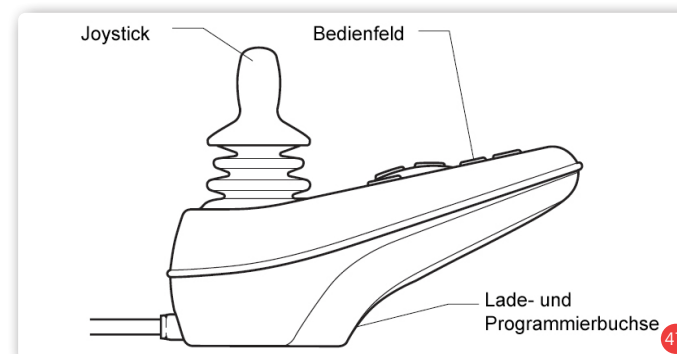4.6. Functional description of the R-NET control unit

The functions of the control panel are the same as for the VR2 control unit.

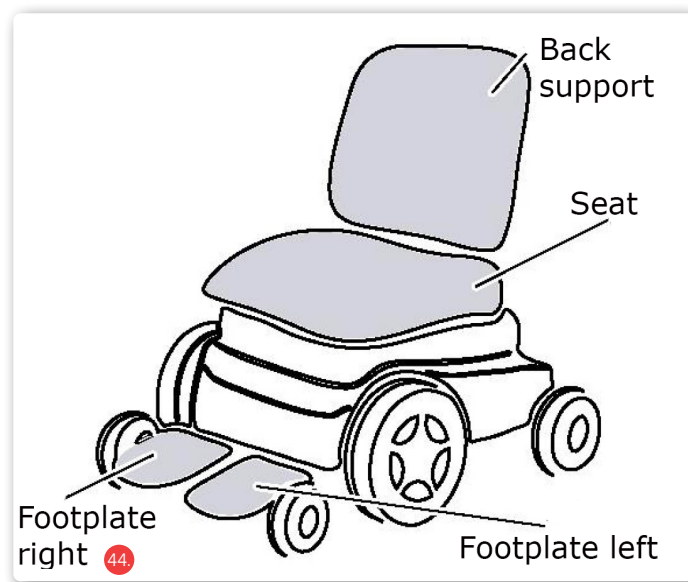
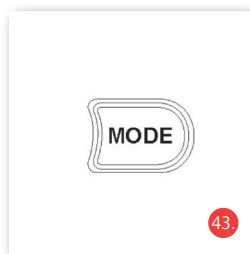
4.7. Functional description of the R-NET control unit for electronic adjustment options (optional)

Using for the first time

Sit on the seat of your Bischoff & Bischoff wheelchair and place your feet on the footplates.

Make sure that you are sitting as far back in the seat as possible. Make sure that loose clothing such as, for example, scarves, shirts, etc. do not hang out of the wheelchair to the side where they could get caught in the wheels. Switch your wheelchair on using the on/off switch (Fig. 29). By pressing the Mode button (Fig. 42/43) you can now activate the controls for the adjustment options.

Display panel for adjustment options

The user can activate the adjustment functions with the Mode button. The lamps for the relevant LED for an adjustment option (left footrest, right footrest, seat, backrest) show which adjustment function is active at that time.

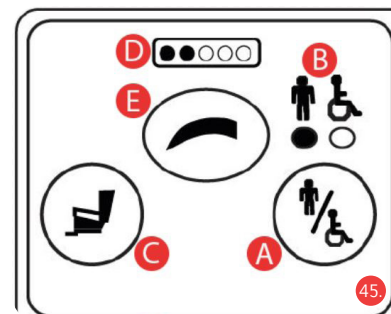
Adjustment functions

Once the adjustment functions have been activated using the Mode button, move the joystick (Fig. 41) left or right, to select between the possible adjustment functions. These are shown through the LEDs on the display panel (Fig. 44).

By deflecting the joystick forwards or backwards, you can control the relevant adjustment direction for the individual channels. In order to return to drive mode, press the Mode button again or one of the buttons for speed level adjustment.

4.8. Functional description of the R-NET attendant control unit

As an option, an attendant control unit is available so that an attendant is able to control the wheelchair.

- A** Toggle switch for operating mode
- B** Operating display
- C** Adjustment function button
- D** Display for active drive and speed level
- E** Speed level adjustment button

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

Toggle switch for operating mode

Use the toggle switch (A) (Fig. 45) select the operating mode, whether the wheelchair will be controlled by the attendant control unit or by the user. The selected operating mode will be shown through the LEDs (B). The red LED lights up when the wheelchair is being controlled by the user. The green LED lights up when the wheelchair can be controlled by the attendant.

Adjustment function button

All R-NET attendant control units are fitted with an adjustment function button as standard. If the R-NET control unit is not programmed with any adjustment options, this button has no function. However if the R-NET control unit is programmed with 1 or up to 4 adjustment options, the functions for this button are described in the relevant section below.

Wheelchairs with up to 4 adjustment options

Pressing the adjustment function button (C) once will activate the set-up mode for the electronic adjustment options.

By deflecting the joystick forwards or backwards, you can control the relevant adjustment direction for the individual channels. Select between the adjustment options by deflecting the joystick to the left or right. When you change the selected adjustment option, this is shown on the control panel of the user's control unit.

To return to drive mode, either press the adjustment function button (C) or the speed level button (E).

Speed levels display

Indicates the speed level adjustment for the wheelchair, if the attendant control unit is activated. There are five settings - setting 1 is the lowest speed and setting 5 is the highest speed. The speed level is adjusting using the (E) button.

Electric wheelchair | **Ejoy RD**
Electric wheelchair | **Ejoy RD XL**
Electric wheelchair | **Ejoy RD XXL**

NE

Button for speed level adjustment

Use this button (E) to adjust the speed level of the wheelchair, if the attendant control unit is activated. By pressing this button, the maximum possible speed of the relevant drive speed will be increased by one level, until you reach level 5. Pressing again will reset the speed back to level 1. If the control unit is programmed for operation with drive profiles, only the speed level within the selected profile will be adjusted using the button for speed level adjustment of the positioning system.

4.9. Locking and unlocking of the wheelchair

The R-Net control system can be locked to prevent unauthorised use. The locking method is by a sequence of button presses and joystick movements as described below.

To lock the wheelchair

- With the control system switched on, press and hold the on/off button.
- After 1 second, the control system will beep. Now release the on/off button.
- Move the joystick forward until the control system beeps.
- Move the joystick backwards until the control system beeps.
- Release the joystick, you will hear a long beep.
- The wheelchair is now locked.

To unlock the wheelchair;

- Switch on the control system with the on/off button. The maximum speed/profile display moves up and down.
- Move the joystick forward until the control system beeps.
- Move the joystick backwards until the control system beeps.
- Release the joystick, you will hear a long beep.
- The wheelchair is now unlocked.

4.10. Adjustments to the Control unit

Changes to the programming of the control unit must not be carried out and voids the product conformity and road approval. Changes to the programming have an impact on speed, acceleration and brake delay. If carried out incorrectly, this can lead to fatal injuries to the user and/or a third party.

4.11. Getting in and out of the chair

Side transfer

- Position the wheelchair and the current or future seat surface side by side, as close as possible to one another.
- If you are transferring from another wheelchair or indoor frame, secure these by applying the parking brakes.
- The wheelchair must be secured so that it cannot roll away unintentionally. To do this, switch the control unit off by pressing the ON/OFF button (when the control unit is switched off, all displays are unlit).
- Remove the armrest/sideguard.
- Flip the footplates to the side or remove the legrests.
- Now slide sideways onto the other seat surface.
- Please make sure that you are sitting as far back in the seat as possible.
- Re-fit the armrests and legrests back in their initial positions.

Frontal transfer

- First swing the footplates upwards or remove the legrests (Fig. 52).
- Position the wheelchair and the current or future seat as close as possible to one another, facing each other.
- If you are transferring from another wheelchair or indoor frame, secure these by applying the parking brakes.
- Now slide onto the seat surface by rotating your body.
- Then hang the legrests back in their initial position.

4.12. Driving the wheelchair

Before setting off, check that the electromagnetic brake is properly locked!

- Make sure that the drives are engaged (see Chapter 4.13)
- Make sure that you are sitting comfortably and securely.
- Press the ON/OFF button to switch the wheelchair into drive mode (Fig. 29).
- All electronic drive functions are now available to you.

Driving functions

Your dealer can adjust the values for speed, acceleration and deceleration individually to your requirements, taking the legal requirements into account.

At the start, choose a lower speed level and increase it during the journey as required.

You can start driving using the joystick. The joystick is used both for increasing the speed and changing the direction of travel. When the joystick is in the central position, the wheelchair is braked mechanically and secured against rolling away.

Driving in a straight line: To drive in a straight line, push the joystick forwards carefully. The further you push the joystick forwards, the faster your wheelchair will go. When you let go of the joystick, the drive will be braked automatically.

Driving backwards: To drive backwards, push the joystick backwards carefully. The further you push the joystick backwards, the faster your wheelchair will go. When you let go of the joystick, the drive will be braked automatically.

Cornering: To change the direction of travel, tilt the joystick to the left or right. This is also possible with the joystick deflected forwards or backwards, i.e. when driving forwards and backwards.

Warning! Use on slopes is only permitted when the seat and backrest are in an upright position.

Attention: risk of falling! Always drive up to the kerb head on. Driving up at an angle can lead to accidents.

Warning! Always put the seat and backrest into an upright position when you are driving the wheelchair.

4.13. Pushing mode

If you want to push the wheelchair in specific situations, you must release the brakes. In freewheel mode, the wheelchair is easy to push.

Warning! The electromagnetic brake is then disabled.

Wheelchairs with decentralised release on the motor:

- The drives can be disconnected and then engaged again using the yellow lever (Fig. 46).
- To push the electric wheelchair, pull the lever upwards. The switch will click into place automatically.
- Repeat the procedure with the drive on the other side. You can now use the wheelchair in push mode. To engage the drives pull the lever down, it will then click into place automatically. Repeat the procedure on the other side here too.
- If the drives are disconnected, then the battery status display flashes if the wheelchair is switched on.

Wheelchairs with central lever for unlocking:

On wheelchairs with a central lever for unlocking, the operation for the control for engaging and disengaging the drives is on the left side of the wheelchair (Fig. 47).

Push mode:

For pushing operation, pull the red lever outwards until you feel it clicks into place (Fig. 48).

The red lever is then in the correct position for pushing operation if it remains in the outward position (Fig. 49).

- If the drives are unlocked, the battery status indicator flashes when switched on.

Drive mode:

To enter travel mode, pull the black actuating knob outwards until the red lever is released from its external lock. The red lever then moves back to its original position for driving (Fig. 50).

On the side of the operating levers there is a notice for the operation of the unlocking and locking of the motors (Fig. 51).

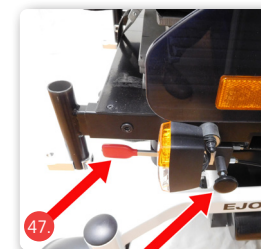
Please note that push mode by disconnecting the electromagnetic brakes must only be used on flat ground. The electromechanical brake is then disabled. Non-observance means danger to life.

Warning! Always use both drives at the same position (risk of accident).

Attention: danger to life! The electromagnetic brakes must only be disconnected when on flat ground.

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

4.14. Legrests

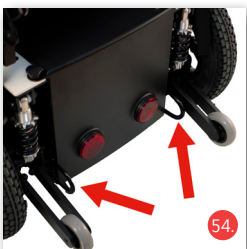
In order to make it as easy as possible to get on and off, you can flip the footplates upwards or swing the legrests out or remove them by lifting them upwards.

- To swing the legrest out, release the catch (Fig. 52).
- In the "out position" the legrest can be completely removed.
- With electronically-adjustable legrests, you must first disconnect the connecting cable under the seat.
- To fit the legrests, repeat the procedure in reverse order.
- Swing the legrest back to the front until you hear it clearly click into place.

4.15. Lifting and holding points

To load the electric wheelchair you should ideally use ramps due to the weight. If you still need to lift the wheelchair, grip it by firmly-fixed frame parts or the hold points. Do not hold it by the casing, electronic parts and other attachments. For transportation you can secure the electric wheelchair using the two eyelets (Fig. 53/54). The wheelchair must not be used as a vehicle seat (see Chapter 2.2).

When driving, respect other road users.

Electric wheelchair | **Ejoy RD**
 Electric wheelchair | **Ejoy RD XL**
 Electric wheelchair | **Ejoy RD XXL**

NE

5. Accessories

5.1. Rearview mirror (optional)

With the optional rearview mirror, which is fitted to the armpad, you have better all-round visibility. You can simply push or pull it to the required position.

5.2. Crutch holder (optional)

Put the crutch in the crutch holder and then fix it to the backrest. Never step on the crutch holder and never use it as a tipping aid.

Please note that the reflectors must not be covered.

5.3. Calf strap

A calf strap is also available for a more relaxed support of the legs.

5.4. Headrest

A height and depth adjustable headrest can also be mounted on your Ejoy.

5.5. Number plate holder

To be able to attach a number plate to your Ejoy, there is also a number plate holder.

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

6. Technical Data

	Ejoy RD	Ejoy RD XL	Ejoy RD XXL
Dimensions and weights			
Maximal payload (kg):	239.6	269.6	351,6
Max. user weight (kg):	140	170	250
Total length with legrests (mm):	1070		1125
Total length without legrests (mm):	880		930
Frame width (mm):	540		580
Total width min./max. (mm):	620/830		760/980
Height without headrest (mm):	960 - 1090		950
Seat width (mm):	420 - 520		580 - 800
Effective seat width	570	640	800
Seat depth (mm):	400 - 550		510 - 560
Front seat height (mm):	425 - 545		435 - 515
Rear seat height (mm):	430 - 550		420 - 500
Seat angle (mechanical) (°):	0° - 20°		0° - 20°
Seat angle (electronic) (°):	-0° - 20°		0° - 20°
Backrest height (standard seat) (mm):	480		570
Backrest height (Comfort seat) (mm):	570 (option)		
Backrest angle (mechanical) (°):	90° - 120°		
Backrest angle (electronic) (°):	100° - 142° (option)		
Kerb weight (kg) incl. batteries:	104		108
Weight of heaviest part (kg):	65.60 (frame)		
Armrest height to seat surface (without cushion) (mm):	255 - 305		260 - 310
Armrest length (mm):	400		405
Armrest width (mm):	85		85
Front armpad length to backrest tube (mm):	400		400
Lower leg length (seat surface to footplate) (mm):	390 - 540		390 - 540

Electric wheelchair | **Ejoy RD**
Electric wheelchair | **Ejoy RD XL**
Electric wheelchair | **Ejoy RD XXL**

	<i>Ejoy RD</i>	<i>Ejoy RD XL</i>	<i>Ejoy RD XXL</i>
Drive characteristics			
Turning radius (mm)	585		830
Turning circle (mm):	1170		1660
Static stability (°):	9° uphill 20° downhill 20° sideways		
Maximum speed forwards (kph):	6 / 10 optional		
Min. braking distance from max. speed* (mm):	1100 (at 6 kph)		1150 (at 6 kph)
Range (km)	approx. 35 (with 55 Ah battery)		approx. 35
Climbing capacity (°)	8°/14%		
max. obstacle height (mm):	50		
Operating forces			
Control lever (N):	< 13.5		
Buttons (N):	< 5		
Engaging/disconnecting the motor brake (N):	< 60		
Operating conditions			
Temperature (°C):	-20 - +50		
Humidity (%):	20 - 80		
Storage conditions			
Temperature (°C):	-20 - +65		
Humidity (%):	45 - 60		
Tyres			
Drive wheels (inch):	14		
Castors (inch):	9		
Tyre pressure (bar/PSI):	3.5 / 50		
Drive			
Type of drive:	Rear wheel drive		
Nominal motor power (W):	350		450

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

	<i>Ejoy RD</i>	<i>Ejoy RD XL</i>	<i>Ejoy RD XXL</i>
Rotational speed (RPM) 6 kph/10 kph	3000/5100		
Battery:	2 x 12 V / 56 Ah 2 x 12 V / 80 Ah (optional)		2 x 12 V / 80 Ah
Fuses	80 A		
Control unit	VR2 R-NET (optional)		
<i>Material</i>			
Lighting	In accordance with StVZO		
Frame:	Steel, corrosion-resistant double-coat paint		
Seat and back upholstery:	Nylon		
Sideguard, legrest	Aluminium		
<i>Accessories</i>			
Contour seat cushion, Contour backrest cushion, adjustable legrest, amputee legrest, headrest, VR2 control unit, R-NET control unit, calf pad, rearview mirror, PU tyres, crutch holder.			
* Please note that the braking distance may increase on uneven, steep ground. The dimensions stated are below the manufacturing tolerances of +/- 10 mm.			

Electric wheelchair | **Ejoy RD**
Electric wheelchair | **Ejoy RD XL**
Electric wheelchair | **Ejoy RD XXL**

7. Maintenance instructions for the user

7.1. Cleaning and disinfection

Cleaning

Make sure that the electric wheelchair is switched off before you start cleaning.

- The frame parts and casings can be wiped with a damp cloth. If there is heavy contamination, you can use a mild household cleaner.
- The wheels can be cleaned with a damp brush with plastic bristles (do not use a wire brush!).
- You can clean dust and minor contamination from your seat, backrest and armrests with a damp cloth. Do not use any aggressive cleaners as these could damage the surface of the material. The upholstery can be washed with 40 °C warm water.
- Make sure that electrical or electronic components do not come into contact with water!

Disinfection

Before disinfection clean the foldable electric wheelchair thoroughly in accordance with the instructions:

- Use a household disinfectant for wipe-down disinfection.
- Spray the upholstery/pads and handles as appropriate.
- Leave the cleaner to work in accordance with the manufacturer's specifications, before you wipe it off.

When doing this, make sure that the electrics and electronics do not come into contact with liquid.

Warning! Do not use high-pressure or steam cleaners.

Please observe the enclosed overview on care, hygiene and maintenance work

7.2. Check Before Use

7.2.1. Checking the brakes

Make sure the brake system is functioning properly before each use.

- Start up carefully and then allow the driving lever to come back to its initial position. The electric wheelchair must come to a halt as usual (see Chapter 2.5).
- When stationary, the electromechanical brake must be locked in place with an audible click. Notify your dealer of defects immediately and discontinue use of the electric wheelchair!
- The motor brake is working properly if you cannot push the electric wheelchair when it is switched off. However if it is possible to do this, the electromagnetic brake is defective. In this case, you must stop using the electric wheelchair. Contact your dealer immediately.

7.2.2. Checking the lighting, indicators and horn

Before setting off each time, check that the horn works properly.

- If, after pressing the appropriate button on the control panel, there is no response, notify your dealer immediately and take the electric wheelchair out of use.

7.2.3. Checking the tyres

The tyres will wear over time.

- Therefore check they are intact before setting off each time.
- Check the tread depth of the tyres. It should be at least one millimetre. The air pressure must match the values specified in the technical data (see Chapter 6).

Contact your dealer if the tyres are defective or worn.

7.3. Batteries

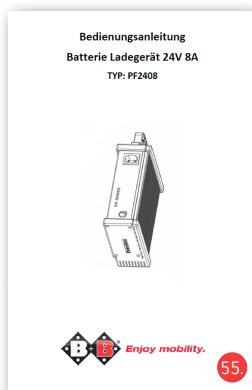
Your wheelchair is fitted with maintenance-free batteries, so you do not need to check the liquid level.

If your wheelchair is not used for a longer period of time, you must connect the charger every 4 weeks, otherwise the batteries could be damaged by deep discharge.

Before setting off each time, check the charge status of your battery by looking at the battery status display on your control unit. If the battery status display is only showing red LED's, you should not travel longer journeys and connect the battery charger to the wheelchair immediately. We recommend that you connect the charger to the wheelchair after every trip.

Only use an approved charger to charge the batteries (see technical data). Please also follow the operating instructions for the charger!

7.3.1. Charging batteries

Only use approved chargers to charge the batteries (type in accordance with the technical data or as delivered by the Bischoff & Bischoff dealer Fig. 48).

Please also follow the operating instructions for the charger!

Do not charge frozen batteries.

The batteries must be charged when the temperature is between 0° and 40°C.

When using the scooter for the very first time, we recommend driving until just 3 bars are visible in the display (see Chapter 6.1) and recharge the batteries at that point.

In the event that this charge level is displayed, you should recharge the batteries as soon as possible in order to avoid deep discharge.

Recharging requires approx. 12 - 16 hours.

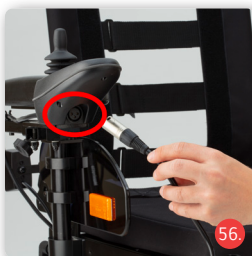
When using the product after that, you can recharge the batteries at any battery charge level.

You can interrupt the charging procedure even if the batteries are not fully charged, if you need to continue driving, for example.

But where possible, always charge to 100% so that you have the greatest range.

Charging batteries:

- First switch the scooter off.
- Before you start using the charger, check that the power supply is 230 V.
- Connect the charger to the power socket (230 V) and then with the charging socket of the scooter (Fig. 56).
- The charging socket is located on the side below the ignition switch (Fig. 57).
- Once the charging procedure is complete, unplug the mains plug and disconnect the scooter from the charger.
- When not in use, the battery should be charged once a month in order to avoid deep discharge.
- Deep discharge shortens the service life of the battery significantly, or the battery may also be damaged by the deep discharge.
- Batteries should be stored upright, in dry, frost-free conditions.
- As soon as the range with fully-charged batteries starts to decrease significantly, you should replace the batteries.

Never use your scooter with empty or almost empty batteries. You may endanger yourself and others by stopping unexpectedly.

7.3.2. Replacing the batteries

When working on the batteries be extremely careful, avoid using metal tools, touching the battery terminals and follow the manufacturer's instructions.

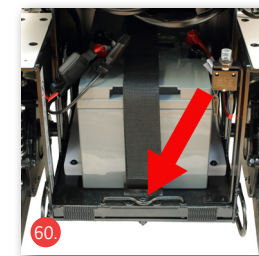
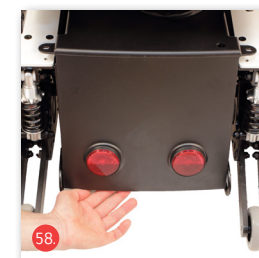
Warning! This work should only be carried out by experts. There is danger of fatal injury!

Only use batteries which are approved for this wheelchair (see Chapter 6).

Make sure that the wheelchair is switched off, before you start to replace the batteries.

- First loosen the rear part of the cover of the battery compartment by pulling the cover to the back (Fig. 58).
- The push the cover of the battery compartment below the seat upwards (Fig. 59). Now pull the cover backwards out of the wheelchair.
- Remove the screw on the front of the bar and then pull the batteries out using the grab bar (Fig. 60)
- Loosen the belt strap (Fig. 60)
- First loosen the red, then the black battery cable.
- Lift the batteries out carefully.
- Lift the new batteries into the designated space and connect the contacts. Note the correct colour coding (red and black).


Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

- Secure the batteries with the belt strap and push the battery compartment back into its initial position, it will click back into place automatically. Fix the screw back onto the grab bar. Put the battery cover back on.


If the service life of the batteries has expired, i.e. they can no longer be charged sufficiently, you must not dispose of the batteries with household waste. When you buy new batteries, old batteries will be taken back by the dealer and disposed of properly!

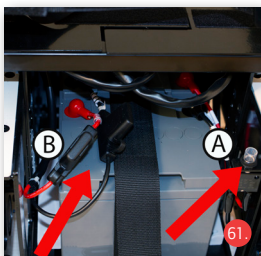7.3.3. Battery, fuses

The power circuit is protected by 2 types of fuse.

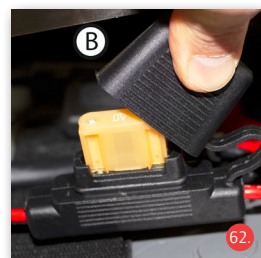- The overload protection (Fig. 61A) is triggered when the wheelchair is overloaded. Wait until the motor system of the wheelchair cools down a bit and press the button of overload protection back down. Before continuing to drive, make sure that the wheelchair is not overloaded.
- The safety fuses in the power circuit of the wheelchair (Fig. 62B) serve as further protection of the power circuit. If the wheelchair does not work when the overload protection is pressed, please

You must check your wheelchair regularly in accordance with the maintenance plan and make sure that it is in perfect working order.

check the safety fuses.

- To do this remove the protective cap of the fuse (Fig. 62B).
- Before continuing to drive, make sure that the wheelchair is not overloaded. Ask your dealer to try to establish the cause.

Electric wheelchair | **Ejoy RD**
 Electric wheelchair | **Ejoy RD XL**
 Electric wheelchair | **Ejoy RD XXL**

NE

8. Maintenance instructions

Typical wear parts are brakes and tyres. Contact your dealer for spare parts.

We recommend that you take your wheelchair to your dealer for inspection at least once a year.

If there are faults or defects, take the wheelchair to your dealer immediately for repair.

The information and documentation required for repair and maintenance shall be supplied to the mobility dealer on request.

If you have questions, you can contact the Bischoff & Bischoff customer service. You can find the contact details for Customer Service on the back cover of this instruction manual.

Check your wheelchair regularly in accordance with the maintenance plan.

9. Repairs

Repairs and servicing work must only be carried out by authorised partners (medical supply stores or dealers). To find a partner in your area, please contact our Customer Service. You can find the contact details for Customer Service on the back cover of this instruction manual.

10. Transport

To load the electric wheelchair you should ideally use ramps due to the weight. If you still need to lift the wheelchair, grip it by firmly-fixed frame parts or the hold points. Do not hold it by the casing, electronic parts and other attachments. For transportation you can secure the electric wheelchair using the two eyelets (Fig. 53/54). The wheelchair must not be used as a vehicle seat.

Under no circumstances may it be used as such. It does not fulfil the requirements in compliance with ISO 7176-19. Failure to observe this instruction can lead to extremely serious or fatal injuries in the event of an accident.

You can see that your wheelchair is not approved for use as a seat in a vehicle for transporting individuals with impaired mobility (KMP) from the symbols on the type plate of your product (Chapter 3.2):

Wheelchair is not designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. 63)

Wheelchair is designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. 64)

11. Storage and shipping

If the wheelchair is to be stored or shipped, all loose parts which are not fixed, except for the drive wheels, must be removed and packed individually in an appropriate cardboard box. The power circuit must be open when stored for long periods of time. The individually packed parts can then be packaged together in a larger cardboard box. We recommend keeping the original packaging and storing it with this purpose in mind, so that it is available if required. Your wheelchair then has optimum protection against environmental influences during storage or transport.

For storage of your commode chair, please observe the information in

the technical data. To avoid mould growth and damage to the upholstery parts, the storage location should be as dry as possible and not be exposed to direct sunlight.

12. Use by others

The wheelchair is designed for re-usage by other persons. Before the product is passed to another user, it must be serviced by a dealer and hygienically cleaned. When passing the wheelchair onto another user, please remember to hand over this user manual and the address of your dealer to the new user.

The wheelchair must be checked in accordance with the maintenance plan and be in proper technical condition.

13. Service life

The expected service life of the Ejoy RD is up to 5 years, if it is used correctly, and safely, following the care and maintenance instructions. Beyond this period the wheelchair can be used if it is in a condition allowing its safe usage.

14. Disposal

If your wheelchair is no longer in use and needs to be disposed of, contact your dealer.

If you wish to dispose of the product yourself, contact local recycling companies to enquire about the regulations that apply to your place of residence.

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

15. Warranty

1. For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from production and material defects. In case of evidence of material or manufacturing defect, defective parts will be replaced free of charge.
2. This warranty excludes claims, if a repair or replacement of a product or a part is required for the following reasons:
 - a) normal wear and tear, which include but is not limited to the following parts where fitted; Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.
 - b) overloading the product such as, for example, by exceeding the maximum user weight or load.
 - c) the product or part has not been maintained or serviced in accordance with the Instruction manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.
 - d) Accessories have been used which are not original accessories.
 - e) The product or part has been damaged through negligence, accident or incorrect use.
 - f) changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
 - g) Repairs have been carried out, before our customer service has been informed of the circumstances.
3. To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.
4. For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1).
5. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.
6. If the repair fails after a reasonable time, the customer can, at his discretion, request a reduction, compensation or the termination of the purchase contract.

Electric wheelchair | **Ejoy RD**
Electric wheelchair | **Ejoy RD XL**
Electric wheelchair | **Ejoy RD XXL**

7. The guarantee is subject to the law of the country in which the product was purchased from Bischoff & Bischoff & Bischoff. Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your dealer will be happy to provide these to you.
For replacement devices and repairs, the warranty period is one year.

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

Warranty certificate

Product: Ejoy RD

Serial number:*

* (To be filled in by the dealer)

Dealer:

Date and seal

(Pay attention to our general terms and conditions.)

*Signs are located on the seat bracket

Note!

Print errors, product and colour changes and technical information are excepted. The colours of the products shown may be slightly different in real life. The images may show accessories. The current version of each Instruction manual is ready for download on our website:

www.bischoff-bischoff.com

Electric wheelchair | **Ejoy RD**
Electric wheelchair | **Ejoy RD XL**
Electric wheelchair | **Ejoy RD XXL**

Non-observance of the operating instructions as well as improper use, maintenance jobs and technical changes or adaptations without the permission of the Bischoff & Bischoff company, lead to guarantee loss and the loss of product liability in general.

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

Ejoy RD | Electric wheelchair
Ejoy RD XL | Electric wheelchair
Ejoy RD XXL | Electric wheelchair

Notes

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

Electric wheelchair | **Ejoy RD**
Electric wheelchair | **Ejoy RD XL**
Electric wheelchair | **Ejoy RD XXL**

Electric wheelchair | **Ejoy RD**
Electric wheelchair | **Ejoy RD XL**
Electric wheelchair | **Ejoy RD XXL**

Electric wheelchair | **Ejoy RD**
Electric wheelchair | **Ejoy RD XL**
Electric wheelchair | **Ejoy RD XXL**

Notes

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

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