
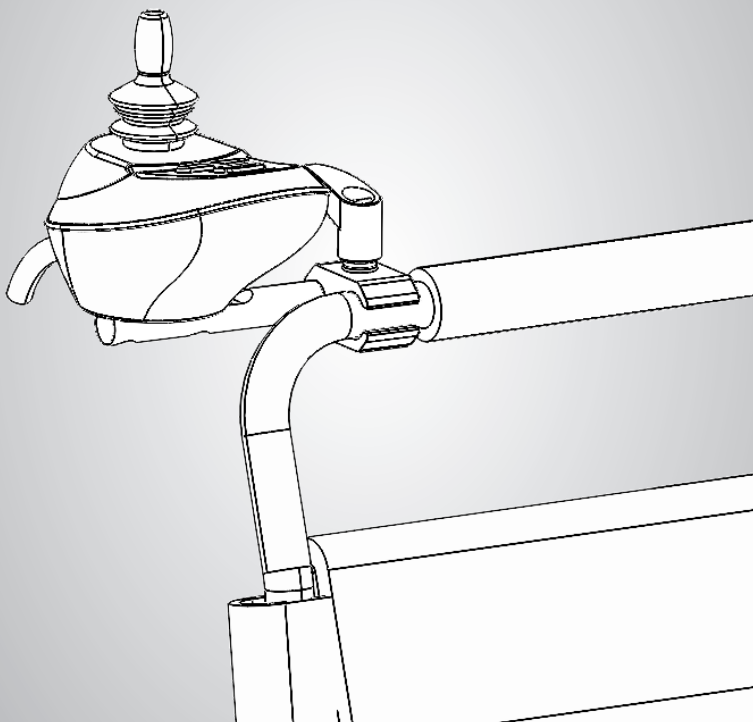
Montageanleitung / Instruction Manual

Montage Begleitersteuerung
Assembly Instructon for the Attendant Conrol
Montaje del control de asistentes

Via

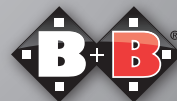Enjoy mobility.

Montageschritte

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

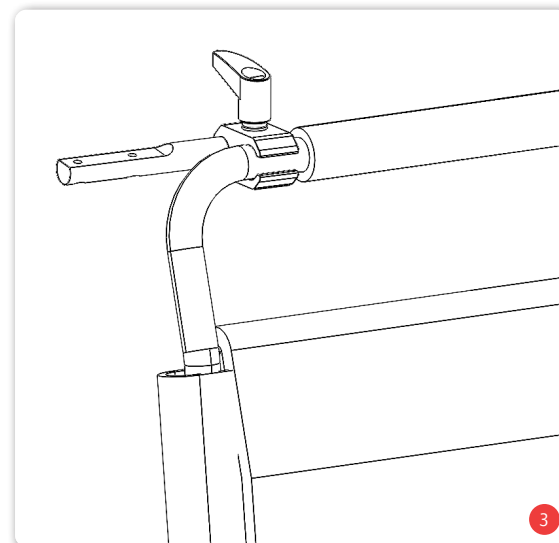
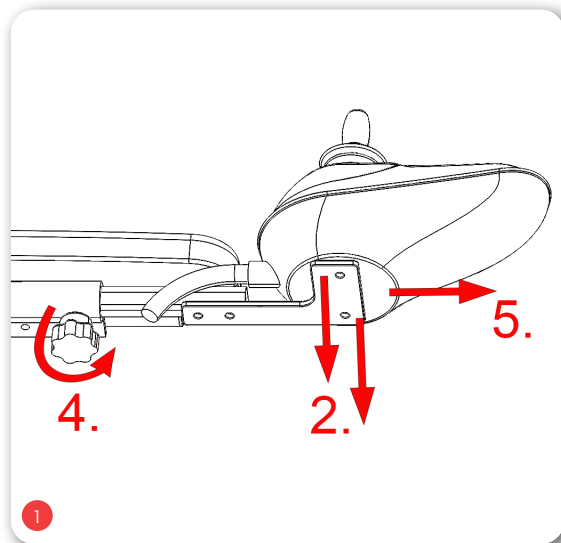
Assembly steps

1. Loosen the Velcro strap for attaching the cable to the control unit.
2. Remove the screws under the control panel and
3. Put the control unit to one side.
4. Unscrew the star knob screw.
5. Pull out the holder for the control panel to the front.

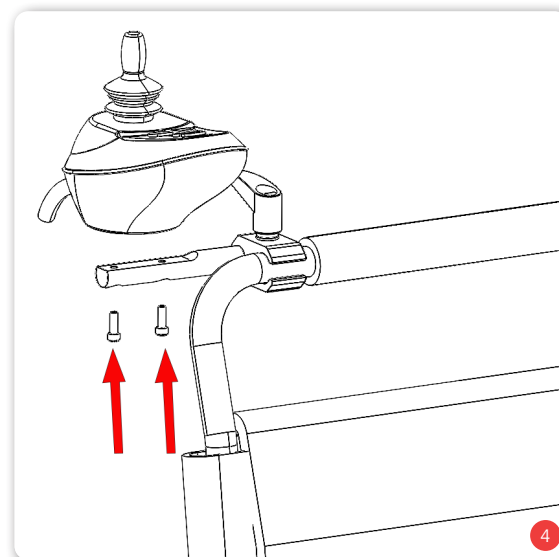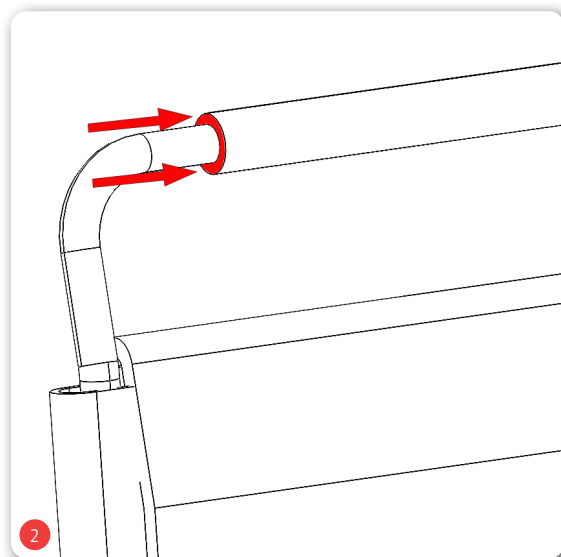6. Push the rubber grip on the edge (red) to the side (escort control can be mounted on the right and left).

7. Handle rubber displaced.
8. Mount the bracket for the attendant control unit in the straight section of the handlebar.
Note: The height can be adjusted individually by turning.
Note: The position of the clamp lever can be rotated after tightening by pushing it up - choose a position that does not disturb the attendant or the user.

9. Mount the control panel on the bracket of the slider control.
Note: Make sure that the cables are not crushed or stretched when folding them up
10. Attach the control unit to the back tube using the Velcro cable fastener.
11. Ready mounted.
Note: When folding, watch out for collisions with other components; if necessary, loosen the clamping lever and tighten the twisted operating part.

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Bischoff & Bischoff GmbH