Bedienungsanleitung  
Instruction manual  
Manual de instrucciones  
Mode d'emploi

## Terra Terra Sx

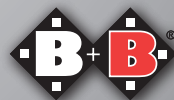
DE

EN

ES

FR

*Enjoy mobility.*

**Inhalt**

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## Contents

1.	Preface	4	5.2.	Headrest (optional)	28
1.1.	Signs and symbols	5	5.3.	Elevating, width-adjustable legrest (optional)	28
2.	Important safety instructions	5	5.4.	Width-adjustment guide (optional)	29
2.1.	General safety instructions	5	5.5.	VR2 attendant control unit (optional)	29
2.2.	Safety in vehicles	7	5.6.	Pneumatic tyres (optional)	29
2.3.	Use on public roads	8	5.7.	Kerb climber (optional)	29
2.4.	Notes on electromagnetic interference	8	5.8.	Crutch holder (optional)	30
2.5.	Brakes	8	5.9.	Rearview mirror (optional)	30
2.6.	Purpose	10	5.10.	Fixing a bag (optional)	31
2.7.	Indications	10	6.	Technical data	32
2.8.	Contraindications	11	7.	Maintenance instructions for the user	35
2.9.	Declaration of conformity	11	7.1.	Cleaning and disinfection	35
2.10.	Service life	11	7.2.	Check Before Use	36
2.11.	Notes on range	11	7.2.1.	Checking the brakes	36
3.	Description of Product and Contents	13	7.2.2.	Checking the lighting, indicators and horn	36
3.1.	Inspection on Delivery	13	7.2.3.	Checking the tyres	37
3.2.	Contents	13	7.3.	Batteries and fuses	37
3.3.	Type plate and serial number	14	7.3.1.	Charging batteries	37
3.4.	Overview	15	7.3.2.	Replacing the batteries	38
3.5.	Basic setup	15	8.	Maintenance information for specialised retailers	39
3.5.1.	Mechanical legrests	16	9.	Maintenance instructions	40
3.5.2.	Elevating legrests (optional)	16	10.	Repairs	40
3.5.3.	Seat	17	11.	Use by others	40
3.5.4.	Slack in the backrest	18	12.	Transport	41
3.5.5.	Sideguards	18	13.	Storage/Shipping	41
3.5.6.	Control unit	20	14.	Disposal	41
3.5.7.	Lap belt	20	15.	Warranty	42
4.	Using the wheelchair	21			
4.1.	Functional description of the VR2 control unit	21			
4.2.	Getting in and out of the chair	24			
4.3.	Driving the wheelchair	25			
4.4.	Pushing mode	26			
4.5.	Legrests	27			
4.6.	Lifting and holding points	28			
5.	Accessories	28			
5.1.	Comfort seat and backrest (optional)	28			

## 1. Preface

Dear user,

You have chosen a high-quality power wheelchair from Bischoff & Bischoff and we would like to thank you for your trust.

The power wheelchair has been designed to offer many advantages and meet your needs. Through the wide range of adjustment options and the very best adaptability with regards to lower leg length, seat and backrest angle as well as the control unit which can be fitted on the right or left the wheelchair can be adapted to your individual body measurements. The Terra can be delivered with two different control units. In the standard configuration, the Terra is supplied with the VR2 control unit, this offers you the option of adjusting up to two components electronically. The optional R-NET control unit allows for electronic adjustment of the legrests, seat angle and backrest angle. In this way, the Terra can be adapted easily to your particular needs.

The power wheelchair is designed for use indoors or for driving outdoors.

Please read and follow the Instruction manual carefully before using your new power wheelchair for the first time. The Instruction manual is an integral and necessary component of this product. Keep the Instruction manual where you can refer to it and if you transfer the wheelchair, give the manual to the new owner.

For visually impaired users, this document is available as a PDF file from our website [www.bischoff-bischoff.com](http://www.bischoff-bischoff.com).

### Note!

In spite of careful research and editing, incorrect information may have crept into our Instruction manuals. We reserve the right to make technical modifications. Illustrations may include accessories. The current version of each Instruction manual is ready for download on our website:

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## 1.1. Signs and symbols

Important! This is especially useful information on a given subject.

Warning! These are especially safety-relevant instructions. Observe the instruction manual!

## 2. Important safety instructions

### 2.1. General safety instructions

- Before setting off, check that all attachments are properly fixed in place.
- Before using it for the first time, practice on level, unobstructed ground. Familiarise yourself thoroughly with the braking and acceleration behaviour when driving in a straight line and into and out of curves.
- Please note that through the shifting of weight body movements or loading, the risk of tipping over to the side, to the front or back can increase.
- The power wheelchair must only be used to transport one person. Please respect the intended use (see Chapter 2.6).
- When setting up or adjusting the product, be aware of the risk of body parts being pinched.
- Never use the power wheelchair under the influence of alcohol or other agents which affect your awareness or your physical and mental capacities.
- It is essential that you use ramps to get over obstacles.
- When on roads, observe the road traffic regulations.
- The rear lights and reflectors on the wheelchair must always be visible to other road users.

- Always put the seat and backrest into an upright position when you are driving the wheelchair.
- Use on slopes is only permitted when the seat and backrest are in an upright position.
- At low ambient temperatures, the metal parts of the wheelchair can get very cold, so please avoid skin contact if this is the case.
- Avoid driving into obstacles without braking (furniture, steps or similar items).
- Always switch the control unit off whenever you are getting on or off the chair.
- Upholstery parts may heat up in direct sunlight. Therefore protect your power wheelchair from direct sunlight, in order to avoid skin injuries.
- The seating upholstery and pads' materials meet the flammability resistance requirements in accordance with EN 1021-2. Therefore do not expose the upholstery or pads or any other parts to ignition sources - such as, for example, cigarettes.

Protect the wheelchair from sunlight to avoid burns caused by heated components

- If you should notice your power wheelchair behaving unusually or if it is damaged, stop using it and contact your dealer.
- Reporting incidents. The operator or user must immediately report
1. any malfunction
  2. any change to the features or performance as well as
  3. Immediately report any improper labelling or instructions for use of a medical device that have or could have led to the death or serious deterioration of the health of a patient, an employee or a third party to your competent regional authority for medical devices. This institute will immediately forward the notification to the operator's competent authority and also inform the manufacturer and the manufacturer's competent authority.

## 2.2. Safety in vehicles

To see whether your wheelchair is permitted for use as a seat in a vehicle for transporting individuals with impaired mobility or not, you can refer to the sticker next to the type plate on the wheelchair:

- Wheelchair is designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. A)
- Wheelchair is not designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. B)

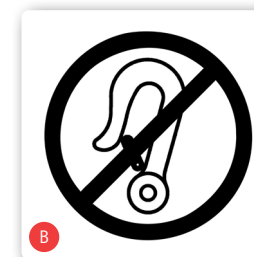
If your wheelchair is not designed to be used as a seat in a vehicle, under no circumstances must it be used as such. It does not fulfil the requirements in compliance with ISO 7176-19. Failure to observe this instruction can lead to extremely serious or fatal injuries in the event of an accident.

(Fig. B).

All Bischoff & Bischoff products which have been approved as seating in vehicles have been tested in compliance with ISO 7176-19. Tests in compliance with ISO 7176-19 have been carried out with AMF-Bruns restraint systems.

By fitting a restraint system, the wheelchair can be used as a vehicle seat.

The correct fitting of the restraint system to this wheelchair is a prerequisite for safe transport of products in a vehicle for transporting individuals with impaired mobility (KMP) and is the basis for the approval. You can see the fixing points on the restraint system in Fig. A. AMF-Bruns restraint systems provide tie down points (Fig. A) for fixing the products in the vehicle for transporting individuals with impaired mobility. The products must only be fixed to these tie downs for restraint in vehicles for transporting individuals with impaired mobility. If possible, use a permanently installed car seat and its belt system. Stow the wheelchair in the boot. Those individuals responsible for the transportation should be familiar with the current legislation for transport as well as with the operation of the vehicle and its restraint system. Four-point restraint systems must be used to secure the wheelchair to the vehicle.

### 2.3. Use on public roads

The wheelchair is designed for outdoor use. Please keep in mind that you are using public roads and therefore have to observe all road traffic regulations.

Do not endanger other road users through inconsiderate driving, especially on pavements.

### 2.4. Notes on electromagnetic interference

Despite compliance with all applicable EMC Directives and standards, it is possible that the power wheelchair may interfere with other electrical devices (e.g. mobile phones, alarm systems) or that its driving performance will be reduced by those devices.

If you notice such behaviour, switch off either the third-party device or your power wheelchair and move it beyond the range of interference.

 It is possible that your power wheelchair will interfere with other devices which emit electromagnetic fields and that they will interfere with your chair.

### 2.5. Brakes

 Make sure that you brake your power wheelchair in good time. Particularly when approaching people or when driving downhill.

Operating brake:

When in motion the motors act as an operating brake. To slow the wheelchair down slowly, move the steering lever and joystick slowly back to the neutral position (initial position).

Parking brake:

When it has come to a halt, switch the wheelchair off using the on/off switch (see Chapter 4.1).

If the drives are engaged, you cannot push or move the power wheelchair (see Chapter 4.4).

The function of the operating and parking brake fulfils the requirements of StVZO.

Pushing mode:

In order to be able to push the wheelchair, pull the release levers on both sides of the wheelchair upwards (see Chapter 4.4).

If the wheelchair is switched on, then the battery status display will flash.

For pushing mode you can switch the wheelchair off.

 Please note that push mode by disconnecting the electromagnetic brakes must only be used on flat ground. The electromechanical brake is then disabled. Non-observance means danger to life.

 The braking distance is significantly longer on slopes than on flat ground. Bear this in mind when going down ramps and brake in time.

Emergency brake:

Press the on/off switch to initiate an emergency stop when driving.

This will lead to intense deceleration. This can lead to injuries and is therefore only to be used in emergencies.

 An emergency stop must only be carried out in an emergency as it can lead to injuries.

## 2.6. Purpose

The Terra power wheelchair has been designed for outdoor use for adults who cannot walk. Unless otherwise described, the instructions in this instruction manual are directed at the occupant of the chair as the operator. The Terra power wheelchair is able to handle longer distances and go up or down obstacles outdoors. It is classified as Category B in accordance with DIN EN 12184.

The maximum user weight is 136 kg.

Please note: We can only provide a warranty if the product is used under the conditions specified and for the intended purposes.

## 2.7. Indications

Provision of this power wheelchair is suitable for adults with significant through to fully-developed impairment of mobility/walking with structural and/or functional damage of the lower extremities, e.g. through

- amputation
- paralysis
- loss of limb
- limb defects/deformity
- joint contractions
- joint damage
- musculoskeletal/neuromusculoskeletal movement disorders
- other illnesses

Provision of power wheelchairs is generally only advised where use of manual wheelchairs is no longer possible because of the disability and the power wheelchair can be used properly.

## 2.8. Contraindications

Provision of this power wheelchair is not suitable for children, and for adults with:

- severe equilibrium disorders
- impaired and insufficient vision
- strongly impaired cognitive abilities

## 2.9. Declaration of conformity

We Bischoff & Bischoff GmbH declare under our sole responsibility, that the Terra wheelchair meets all the requirements of MDR 2017/745 that are applicable.

## 2.10. Service life

The expected service life of the power wheelchair is up to five years, if it is used correctly, and the safety, care and maintenance instructions are followed. The power wheelchair can be used beyond this period if it is in a condition allowing its safe usage.

## 2.11. Notes on range

The range of the Terra is up to 35 km. The achievable range depends on different factors:

- ground properties
- way of driving
- battery condition
- load
- ambient temperature

Please note that the range stated by us was calculated under test conditions:

- batteries which are as new and fully charged
- ambient temperature of 21 °C
- max. user weight

- level, firm ground
- constant driving speed

The following factors affect the driving performance and will thereby possibly reduce the range by up to 50%:

- poor battery charge level
- low ambient temperature (below 10 °C)
- frequent starting and braking
- electromagnetic fields (see Chapter 2.4)

Never use your power wheelchair with empty or almost empty batteries. You may endanger yourself and others by stopping unexpectedly.

### 3. Description of Product and Contents

#### 3.1. Inspection on Delivery

All Bischoff & Bischoff GmbH products undergo a proper final inspection in our company and are labelled with the CE mark (see Chapter 3.3).

The wheelchair will be delivered in a special cardboard box at factory settings. Keep the box after unpacking, if possible. It is useful for later storage of the product or for sending it back.

Delivery and first instructions are taken care of by your qualified medical supply dealer.

If the product is shipped by rail or road, the goods should be checked immediately for transport damages in the presence of the bearer.

Check the contents for completeness (see Chapter 3.2) and make sure nothing is damaged. Should you find any irregularities or damage, please contact our customer service. You can find the contact details for Customer Service on the back cover of this instruction manual.

#### 3.2. Contents

After receipt of the goods, please check the contents for completeness right away. The contents consist of:

- outer packaging,
- wheelchair,
- legrests,
- charger,
- any accessories
- this Instruction manual

### 3.3. Type plate and serial number

The type plate (Fig. 01) and serial number are located under seat surface.

- A** Manufacturer logo
- B** Manufacturer information
- C** MD Medical device
- D** CE mark
- E** Warning! - read the User manual
- F** Important! - read the User manual
- G** Model designation
- H** Model number
- I** Max. user weight
- J** Date of manufacture
- K** Classification in accordance with DIN EN 12184
- L** maximum speed
- M** UDI number
- N** Serial number

The type plate contains the following information:

- A** Manufacturer logo: Bischoff & Bischoff logo.
- B** Manufacturer information: Bischoff & Bischoff GmbH, Becker-Göring-Straße 13, D - 76307 Karlsbad.
- C** MD Medical device symbol.
- D** CE mark.
- E** Warning symbol: exclamation mark in a triangle.
- F** Information symbol: lowercase 'i' in a circle.
- G** Model designation: Model: Terra.
- H** Model number: REF 8350.
- I** Max. user weight: 136 kg.
- J** Date of manufacture: 2020-06.
- K** Classification: Class: B (Outdoor | Indoor).
- L** Maximum speed: 6 km/h.

Below the type plate are two barcodes:

- M** UDI (Unique Device Identifier): (01)04250886504963
- N** Serial number (SN): 8350V0000000135

### 3.4. Overview

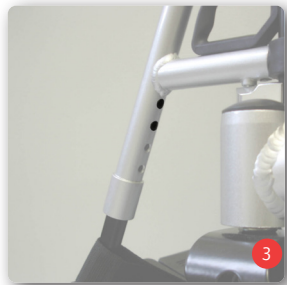
### 3.5. Basic setup

Before using it for the first time, adjust the wheelchair to suit your body size. Adjusting the wheelchair allows long periods of fatigueless driving in a physiologically correct position. The basic setup of the wheelchair is described in this section. You should carry these out in collaboration with the medical equipment retailer.

### 3.5.1. Mechanical legrests

#### Lower leg length

- The lower leg length can be adjusted.
- To do this undo the two Allen screws (Fig. 03).
- Adjust to the required height.
- Re-tighten the screws. Use medium-strength screw lock liquid to fix in place.

For secure fixing, make sure that two screws are always used!

#### Footplate angle

- The angle of the footplate can be adjusted.
- Undo the screw fixing on the side of the footplate (Fig. 04).
- Adjust the footplate to the required angle.
- Re-tighten the screw. Use medium-strength screw lock liquid to fix in place.

### 3.5.2. Elevating legrests (optional)

Assemble, disassemble and swing them away as for the standard legrests.

To adjust, undo the clamp lever, then adjust the legrest to the required length and tighten the clamp lever

The horizontal adjustment works as follows:

- Push the locking lever forwards.
- by lifting or lowering the footplate receiver tube you can move the legrest to the required position.
- Release the locking lever; legrest locks into place automatically.

Never stand on the footplate and when swinging it away, watch your fingers as there is a risk of pinching.

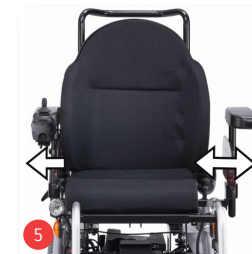
To change seat, the legrest must be swung away or removed. Never sit on the legrest. Risk of tipping over!

### 3.5.3. Seat

#### Seat width

The seat width can be adjusted using the sideguards (Fig. 5). Adjusting the sideguards is described in Chapter 3.5.5 Sideguards.


Make sure that both halves of the seat are adjusted evenly.

#### Seat depth

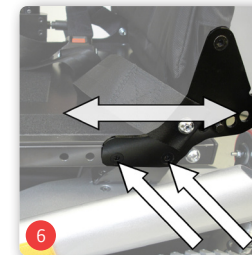The seat depth can be adjusted. You should then use a readjusted seat cushion.


To adjust the seat depth, undo the backrest fixings, push the backrest forwards or backwards and then re-tighten the screws (Fig. 06).

#### Seat angle - mechanical


This description only applies if you have a configuration with a mechanically-adjustable seat angle.

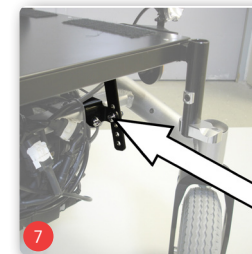- The default setting for the seat angle is 0° to the back.
- The seat angle can be adjusted backwards in steps.
- To do this undo the fixing screws on the telescopic profile under the front of the seat.
- Pull the screws out and adjust to the required seat angle (Fig. 07).
- Put the screws into the next possible position and re-tighten them. Please use medium-strength screw lock liquid to fix the screw nut in place.

#### Backrest angle - mechanical

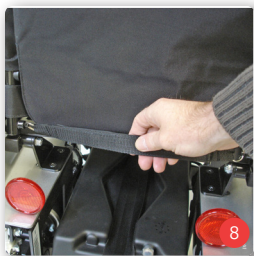The backrest angle of your power wheelchair can be adjusted easily. You also have the option of folding the backrest fully forwards.

- In order to fold the backrest, pull the locking pin using the strap on the backrest and then fold the backrest down (Fig. 08).
- Pull the wire behind the backrest. When doing this hold the backrest firmly with your other hand.
- When you release the wire, the bolts will click into the holes.


Make sure that both bolts, right and left are completely in place.

### 3.5.4. Slack in the backrest

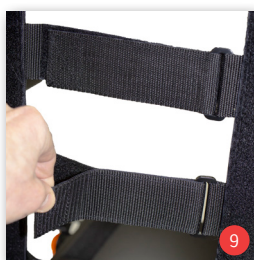A well-adjusted backrest will make it easier to sit for long periods of time more comfortably and effectively reduces the risk of pressure points. When doing this make sure there is sufficient slack in the pelvic area.

- Remove the cushion support of the backrest sling.
- Undo the hook and loop fasteners on the belt straps (Fig. 9).
- Adjust the straps in sequence, starting from the bottom.
- Put the cushion support of the backrest sling back in place.

### 3.5.5. Sideguards

Make sure that the wheelchair is switched off before removing the sideguards.

The sideguards can be completely removed for side transfers. Note that the sideguards can be removed but not completely separated without disconnecting the cable for the control units. In addition the height of the armrests can be adjusted. Adjust the sideguards to move to the side.

### Removing sideguards

- To remove the sideguards, undo the clamp lever screw (Fig. 10) and pull the armrest out upwards.
- When fitting the sideguards make sure that they are pushed in to the limit and fixed back in place.

### Adjusting the height

- First take the sideguards out of the bracket.
- Remove the check screw (Fig. 11) and screw it into the required position in the armrest.
- The lowest hole is the maximum height setting for the armrest. Use medium-strength screw lock liquid to fix in place.

### Moving sideways

- Undo the screws of the relevant sideguard (Fig. 12) under the seat and push the sideguard to the required position.
- Re-tighten the fixing and secure with medium-strength screw lock liquid.

### Moving lengthwise

- First take the sideguards out of the bracket.
- Undo the armrest screws.
- Move these to the required position
- Screw the armrest firmly back onto the sideguard.

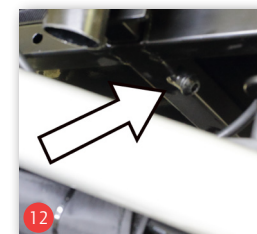
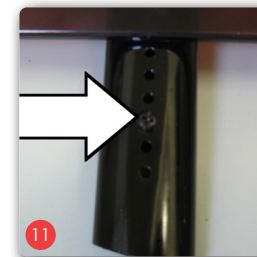
Make sure that the adjustment is only within the marked area!

### 3.5.6. Control unit

The control unit can be fitted below the armrest, either on the right or left. The adjustment must be carried out by your dealer, to guarantee that the wheelchair operates safely

Note that the cable to the control unit must be relocated. Consult a specialist to do this.

**Armrest gap**

- You can adjust the position of the control unit to your forearm length.
- Undo the hand screw on the outside of the sideguard.
- Fit the control unit in position and re-tighten the screw (Fig. 13).

**Swivel**

- The control unit can be swivelled to the side. It clicks into the armrest extension and can be swung fully outwards. (Fig. 14).

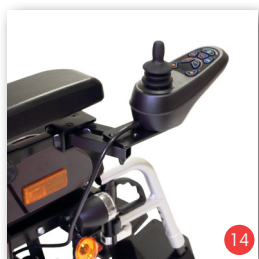
 Using the control unit is explained in Chapter 4. "Using the wheelchair"

**3.5.7. Lap belt**

The lap belt is fitted to the seat frame.

- Adjust the belt length to suit your requirements, by positioning the buckle accordingly.
- Fasten the belt with the buckle (Fig. 15).
- Open the belt by pressing the red button on the buckle.



**4. Using the wheelchair**

Once you have adjusted the wheelchair to your personal needs, familiarise yourself with the functions of the control unit.

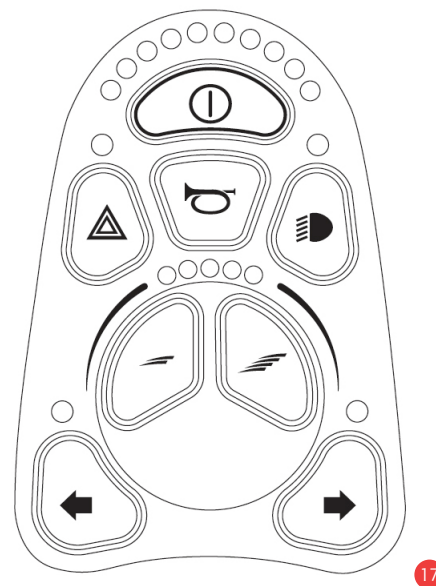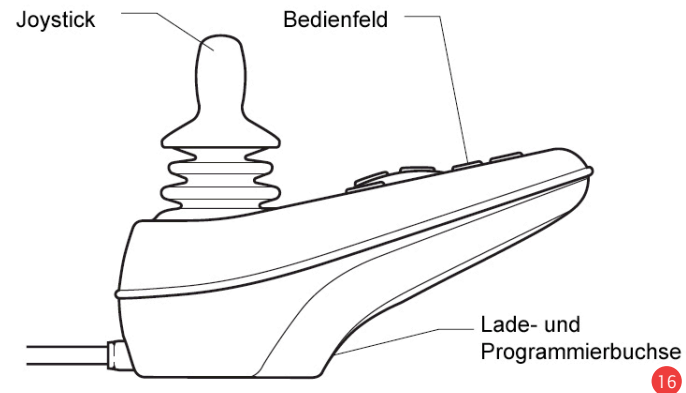**4.1. Functional description of the VR2 control unit**

**Headlight**

Use this button (Fig. 18) to switch on the active lighting of your wheelchair at dusk or when going through dim areas such as, for example, tunnels. You are then more visible to other road users.

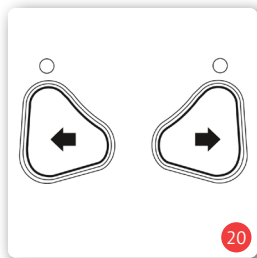
**Hazard lights**

Use this button (Fig. 19) to switch the hazard lights on in emergency situations when the wheelchair is not fit to drive. All four hazard lights will then flash and indicate to other road users that your wheelchair is not fit to drive. When switching the control unit off, the hazard lights will remain on and can only be switched off when the control unit is re-activated.

**Indicators (left/right)**


Use this button to switch on the left or right indicator of your wheelchair (Fig. 20), to indicate to other road users that you would like to change your direction to the left/right. By pressing this button again, you will switch the indicators back off.

**Battery status display**

You can check the battery's charge level on the battery status display (Fig. 21). If the display is in the green area, then sufficient charge is available. If the display is in the red area, then only short journeys are possible and the battery should be recharged (see Charging batteries Chapter 7.3.1).

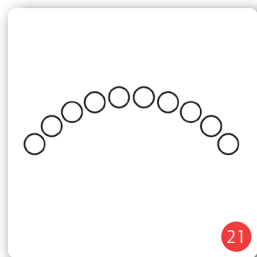If the battery display is constantly lit, then the power wheelchair is fully functional. If there is a fault, this is shown by rapid flashing. First check the battery charge level and whether the drives are engaged. If the battery charger continues to flash, contact your dealer.

**ON/OFF button**

Use this button (Fig. 22) to switch your wheelchair into drive mode.

**Speed level display**

The speed level display (Fig. 23) shows the pre-set speed level. The possible settings are from one to five LED lamps. The speed levels indicate indirectly the maximum achievable speed when deflecting the joystick, i.e. in speed level five, the highest maximum speed is achievable and in level one, the lowest.

**Adjusting the speed level**

Use these buttons (Fig. 24) to increase (left) or decrease (right) the speed level, i.e. the maximum achievable speed. Speed regulation is through operation of the joystick.

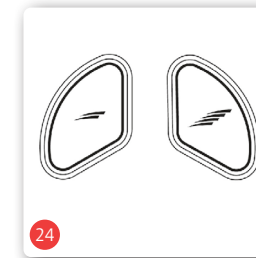
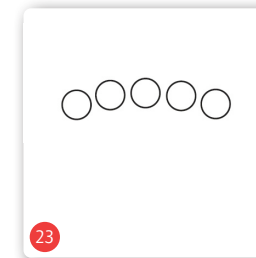
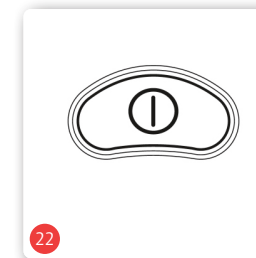
**Horn**

The horn (Fig. 25) enables you to draw attention to yourself in difficult situations.

**Immobiliser**

In order to protect your wheelchair against unauthorised use or theft, park the wheelchair in an enclosed space or activate the immobiliser.

- With the control unit switched off, press the ON/ON/OFF button until you hear a beep. Then release the button.
- Push the joystick in the direction of travel, against the direction of travel and then into the neutral position. A beep will sound each time. The control unit is now disabled.
- Switch the control unit on using the ON/OFF button.
- Push the joystick in the direction of travel, against the direction of travel and then into the neutral position. A beep will sound each time. The control is now enabled again.

**Using for the first time**

Sit on the seat of your Bischoff & Bischoff wheelchair and place your feet on the footplates.

Make sure that you are sitting as far back in the seat as possible. Make sure that loose clothing such as, for example, scarves, shirts, etc. do not hang out of the wheelchair to the side where they could get caught in the wheels!

**4.2. Getting in and out of the chair****Side transfer**

- Position the wheelchair and the current or future seat surface side by side, as close as possible to one another.
- If you are transferring from another wheelchair or indoor frame, secure these by applying the parking brakes.
- The Terra must be secured so that it cannot roll away unintentionally. To do this, switch the control unit off by pressing the ON/OFF button (when the control unit is switched off, all displays are unlit).
- Remove the armrest/sideguard (Chapter 3.5.5).
- Flip the footplates to the side or remove the legrests.
- Now slide sideways onto the other seat surface.
- Please make sure that you are sitting as far back in the seat as possible.
- Re-fit the armrests and legrests back in their initial positions.

**Frontal transfer**

- First swing the footplates upwards or remove the legrests.
- Position the Terra and the current or future seat as close as possible to one another, facing each other.
- If you are transferring from another wheelchair or indoor frame, secure these by applying the parking brakes.
- Now slide onto the seat surface by rotating your body.
- Then hang the legrests back in their initial position.

**4.3. Driving the wheelchair**

Before setting off, check that the electromagnetic brake is properly locked!

- Make sure that you are sitting comfortably and securely.
- Pressing the ON/OFF button to switch the Terra into drive mode.
- All electrical drive functions are now available to you.

**Driving functions**

Your dealer can adjust the values for speed, acceleration and deceleration individually to your requirements, taking the legal requirements into account.

At the start, choose a lower speed level and increase it during the journey as required.

You can start driving using the joystick. The joystick is used both for increasing the speed and changing the direction of travel. When the joystick is in the central position, the wheelchair is braked mechanically and secured against rolling away.

Driving in a straight line: To drive in a straight line, push the joystick forwards carefully. The further you push the joystick forwards, the faster your wheelchair will go. When you let go of the joystick, the drive will be braked automatically.

Driving backwards: To drive backwards, push the joystick backwards carefully. The further you push the joystick backwards, the faster your wheelchair will go. When you let go of the joystick, the drive will be

braked automatically.

**Cornering:** To change the direction of travel, tilt the joystick to the left or right. This is also possible with the joystick deflected forwards or backwards, i.e. when driving forwards and backwards.

 Warning! Use on slopes is only permitted when the seat and backrest are in an upright position.

 Warning! Always put the seat and backrest into an upright position when you are driving the wheelchair.

#### 4.4. Pushing mode

If you want to push the wheelchair in specific situations, you must release the brakes. In freewheel mode, the wheelchair is easy to push.

 Warning! The electromagnetic brake is then disabled.

- The drives can be disconnected and engaged using the red and black levers (Fig. 26).
- To push the power wheelchair, pull the red lever upwards. The switch will click into place automatically.
- To engage the drives pull the black knob upwards, it will then click into place automatically.

 Please note that push mode by disconnecting the electromagnetic brakes must only be used on flat ground. The electromechanical brake is then disabled. Non-observance means danger to life.

 Warning! Always use both drives at the same position (risk of accident).

 Attention: danger to life! The electromagnetic brakes must only be disconnected when on flat ground.

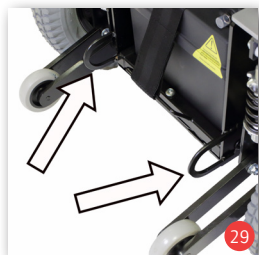
#### 4.5. Legrests

In order to make it as easy as possible to get on and off, you can flip the footplates upwards or swing the legrests out or remove them by lifting them upwards.

- To swing the legrest out, release the catch (Fig. 27).
- In the "out position" the legrest can be completely removed (Fig. 28).
- With electrically-adjustable legrests, you must first disconnect the connecting cable under the seat.
- To fit the legrests, repeat the procedure in reverse order.
- Swing the legrest back to the front until you hear it clearly click into place.


### 4.6. Lifting and holding points

To load the power wheelchair you should ideally use ramps due to the weight. If you still need to lift the wheelchair, grip it by firmly-fixed frame parts or the hold points (Fig. 28 and Fig. 29). Do not hold it by the casing, electronic parts and other attachments. For transportation you can secure the power wheelchair using the two eyelets, at the front and back (Fig. 29 and Fig. 29). These brackets are not for transporting individuals, to do this see Chapter 2.2.

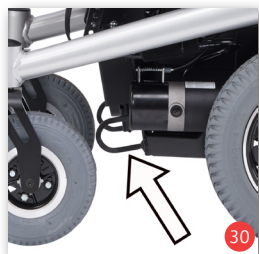Attention: danger to life! The lifting and holding points must not be used for transporting individuals.

## 5. Accessories

### 5.1. Comfort seat and backrest (optional)

For greater comfort, the wheelchair can be fitted with a comfort seat and comfort backrest.

### 5.2. Headrest (optional)

If needed the wheelchair can also be fitted with a headrest.

### 5.3. Elevating, width-adjustable legrest (optional)

For further adjustment options for the legrest, elevating, width-adjustable legrests are available

### 5.4. Width-adjustment guide (optional)

If required, the wheelchair can also be fitted with a width adjustment guide. This enables adjustment of the legrests to the seat width.

### 5.5. VR2 attendant control unit (optional)

With the VR2 attendant control unit, the wheelchair can be controlled by an attendant.

### 5.6. Pneumatic tyres (optional)

If required, the wheelchair can be fitted with pneumatic tyres.

### 5.7. Kerb climber (optional)

The kerb climber will make it easier for you to get up steps of up to 100 mm (Fig. 31).

Using the kerb climber requires skill and experience. Practise with the kerb climber on clear ground first, with an attendant.

When using the kerb climber, observe all the safety instructions for the wheelchair.

Attention: risk of falling! Always drive up to the kerb head on. Driving up at an angle can lead to accidents.

You should ideally use ramps to get over obstacles. Only use the kerb climber if no ramps are available.

- Drive carefully up to the kerb and make sure that you can get up the kerb height with the legrests.
- Make sure that you only drive the up to the kerb head on.
- You must only go up the kerb in the slowest drive mode.
- Place the kerb climber carefully onto the kerb and accelerate slowly until you have gone up the obstacle with the front wheels.
- Then continue driving slowly until you have also gone up the obstacle with the rear wheels.

### 5.8. Crutch holder (optional)

Put the crutch in the crutch holder and then fix it to the backrest. Never step on the crutch holder and never use it as a tipping aid (Fig. 32).

### 5.9. Rearview mirror (optional)

With the optional rearview mirror, which is fitted to the armpad, you have better all-round visibility. You can simply push or pull it to the required position (Fig. 33).

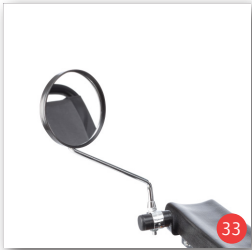
When driving, respect other road users.

### 5.10. Fixing a bag (optional)

If you want to fit a bag to your wheelchair, you can hang the straps of the bag available over the back tubes of your wheelchair. Bags are also available as accessories.

Please note that reflectors must not be covered.

## 6. Technical Data

### Dimensions and weights

Maximal payload (kg):	229
Max. user weight (kg):	136
Total length with legrests (mm):	1060
Total length without legrests (mm):	820
Width (mm):	590 - 680
Total height (mm):	Standard: 1045 Comfort seat: 1170
Folded height (mm):	800 - 850
Seat width (mm):	410
effective seat width:	500 (can be adjusted using sideguards)
Seat depth (mm):	450 to 550 (adjustable in 3 steps)
Front seat height (mm):	480
Seat angle (mechanical) (°):	-2° - 10°
Backrest height (standard seat) (mm):	550
Backrest height (Comfort seat)(mm):	630 - 670 (option)
Backrest angle (mechanical) (°):	90° - 120°
Kerb weight (kg):	93.8
Weight of heaviest part (kg):	28.2 (frame)
Armrest height to seat surface (without cushion) (mm):	280 - 320
Armrest length (mm):	400
Armrest width (mm):	85
Front armpad length to backrest tube (mm):	370 - 450
Lower leg length (seat surface to footplate) (mm):	380- 460
Length x width footplate(s) (mm):	170 x 150

Footplate(s) angle (°):	0 - 25°
<b>Drive characteristics</b>	
Turning radius (mm)	780
Turning circle (mm):	1560
Turning area (in 3 moves) (mm):	
Static stability (°):	10 uphill 10 downhill 10 sideways

Maximum speed forwards (kph)	
Germany:	6
International:	10
Min. braking distance from max. speed* (mm):	1100
Range (km)	up to 32 (with 55 Ah battery)
Climbing capacity (°)	8°
max. obstacle height (mm):	50

### Operating forces

Control lever (N):	< 13.5
Buttons (N):	< 5
Engaging/disconnecting the motor brake (N):	< 60

### Operating conditions

Temperature:	-25 - +50°C
Humidity:	20 - 80%

### Storage conditions

Temperature:	-40 - +65°C
Humidity:	45 - 60%

### Tyres

Drive wheels (inch):	14
Castors (inch):	8

Anti-tip wheels (inch):	3
Tyre pressure (bar/PSI):	(3.5/50)
<b>Drive</b>	
Type of drive:	Rear wheel drive
Nominal motor power (W)	
Germany and International:	2 x 300 W (6 kph)
International:	2 x 450 W (10 kph)
Rotational speed (RPM)	
Germany and International:	3000 RPM (6 kph)
International:	4500 RPM (10 kph)
Battery:	2 x12 V / 56 Ah
Control unit	VR2 R-NET (optional)
<b>Material</b>	
Lighting	Terra SX: none Terra: In accordance with StVZO
Frame:	Steel, painted
Seat and back cushion:	Nylon
Sideguard, legrest	Aluminium
*Please note that the braking distance may increase on uneven, steep ground.	
The dimensions stated are below the manufacturing tolerances of +/- 10 mm.	

## 7. Maintenance instructions for the user

### 7.1. Cleaning and disinfection

#### Cleaning

Make sure that the power wheelchair is switched off before you start cleaning.

- The frame parts and casings can be wiped with a damp cloth. If there is heavy contamination, you can use a mild household cleaner.
- The wheels can be cleaned with a damp brush with plastic bristles (do not use a wire brush!)
- You can clean dust and minor contamination from your seat, backrest and armrests with a damp cloth. Do not use any aggressive cleaners as these could damage the surface of the material. The upholstery can be washed with 40 °C warm water.
- Make sure that electrical or electronic components do not come into contact with water!

#### Disinfection

Before disinfection clean the foldable power wheelchair thoroughly in accordance with the instructions:

- Use a household disinfectant for wipe-down disinfection.
- Spray the upholstery/pads and handles as appropriate.
- Leave the cleaner to work in accordance with the manufacturer's specifications, before you wipe it off.

When doing this, make sure that the electrics and electronics do not come into contact with liquid.

Warning! Do not use high-pressure or steam cleaners.

Please observe the enclosed overview on care, hygiene and maintenance work

## 7.2. Check Before Use

### 7.2.1. Checking the brakes

Make sure the brake system is functioning properly before each use.

- Start up carefully and then release the joystick. The foldable power wheelchair must come to a halt as usual.
- When stationary, the electromechanical brake must be locked in place with an audible click. Notify your dealer of defects immediately and discontinue use of the foldable power wheelchair!
- The motor brake is working properly if you cannot push the power wheelchair when it is switched off. However if it is possible to do this, the electromagnetic brake is defective. In this case, you must stop using the power wheelchair. Contact your dealer immediately.

### 7.2.2. Checking the lighting, indicators and horn

Before setting off each time, check that the horn works properly.

- If, after pressing the appropriate button on the control panel, there is no reaction from the unit which is being tested, notify your dealer immediately and take the foldable power wheelchair out of use.

### 7.2.3. Checking the tyres

The tyres will wear over time.

- Therefore check they are intact before setting off each time.
- Check the tread depth of the tyres. The air pressure must match that specified in the technical data (see Chapter 5).

Contact your dealer if the tyres are defective or worn.

## 7.3. Batteries and fuses

Your wheelchair is fitted with maintenance-free AGM batteries, so you do not need to check the liquid level.

If your wheelchair is not used for a longer period of time, you must connect the charger every 4 weeks, otherwise the batteries could be damaged by deep discharge.

Before setting off each time, check the charge status of your battery by looking at the battery status display on your control unit. If the battery status display is only showing red LED's, you should not travel longer journeys and connect the battery charger to the wheelchair immediately. We recommend that you connect the charger to the wheelchair after every trip.

### 7.3.1. Charging batteries

Only use approved chargers to charge the batteries (type according to technical data or supplied by the Bischoff & Bischoff specialist dealer). Also observe the instructions for use of the charger!

Do not charge frozen batteries.

Batteries should be charged between 0° and 40° C.

For your very first use, we recommend that you drive the electric wheelchair until 3 bars are still visible in the display.

(see section 4.1) and then charge the batteries again.

If this state of charge is displayed, you should recharge the batteries as soon as possible to avoid a deep discharge.

A recharge takes approx. 12 - 16 hours.

During subsequent use, you can recharge the batteries at any battery level.

You can also interrupt the charging process if the batteries are not yet full, e.g. because you want to continue driving.

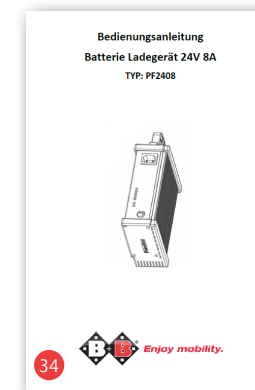
However, if possible, always charge to 100 % so that you have as much range as possible.

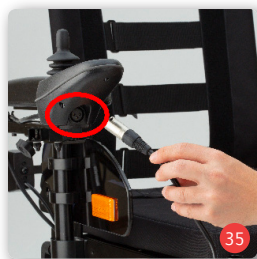
Charging procedure:

First switch off the electric wheelchair .

Before you start the charger, check that the power supply is 230 V.

Connect the charger to the mains socket (230V) and then to the charg-

ing socket of the electric wheelchair .  
The charging socket is located on the side below the ignition lock (Fig. 35 / 36).

When charging is complete, disconnect the mains plug and then disconnect the electric wheelchair from the charger.  
When not in use, the battery should be charged once a month to avoid deep discharge.

A deep discharge considerably shortens the life of the battery and can also damage it.


Batteries should be stored upright, dry and frost-free.

As soon as the range decreases noticeably with fully charged batteries, you should replace them.

**i** Only use an approved charger to charge the batteries (see technical data). Please also follow the operating instructions for the charger!

### 7.3.2. Replacing the batteries

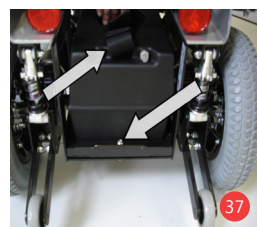
When working on the batteries be extremely careful, avoid using metal tools, touching the battery terminals and follow the manufacturer's instructions.

Only use batteries which are approved for this wheelchair (see technical data).

Make sure that the wheelchair is switched off, before you start to replace the batteries.

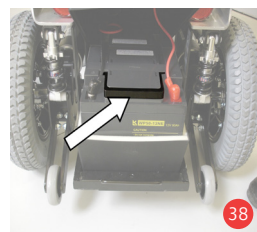- Undo the cover of the battery compartment (Fig. 46).
- First undo the anchor fixing and pull the battery slides out (Fig. 47).
- Undo the battery cable. First the red, then the black.
- Lift the batteries out carefully.
- Lift the new batteries into the designated space and connect the contacts. Note the correct colour coding (red and black).

- Secure the batteries with the belt strap and push the battery compartment back into its initial position, it will click back into place automatically. Then screw the battery cover back on.

**i** If the service life of the batteries has expired, i.e. they can no longer be charged sufficiently, you must not dispose of the batteries with household waste. When you buy new batteries, old batteries will be taken back by the dealer and disposed of properly!

### 8. Maintenance information for specialised retailers

A maintenance plan can be made available to you by our Customer Service.

**i** You must check your wheelchair regularly in accordance with the maintenance plan and make sure that it is in perfect working order.

### 9. Maintenance instructions

Typical wear parts are brakes and tyres. Contact your dealer for spare parts.

**i** Check your wheelchair regularly in accordance with the maintenance plan.

We recommend that you have your scooter inspected by your dealer at least once a year.

If there are faults or defects, take the wheelchair to your dealer immediately for repair.

The information and documentation required for repair and maintenance shall be supplied to the mobility dealer on request.

If you have questions, you can contact the Bischoff & Bischoff customer

service. You can find the contact details for Customer Service on the back cover of this instruction manual.

**Warning!** This work should only be carried out by an expert. There is danger of fatal injury!

## 10. Repairs

Repairs and servicing work must only be carried out by authorised partners (medical supply stores or dealers). To find a partner in your area, please contact our Customer Service on tel.: You can find the contact details for Customer Service on the back cover of this instruction manual.

## 11. Use by others

The wheelchair is designed for reusage by other persons. Before it is passed onto another user, it must be serviced by a dealer and hygienically cleaned. When passing on the wheelchair, don't forget to hand over this Instruction manual and the address of your dealer to the new user.

The wheelchair must be checked in accordance with the maintenance plan and be in proper technical condition.

## 12. Transport

To load the power wheelchair you should ideally use ramps due to the weight. If you still need to lift the wheelchair, grip it by firmly-fixed frame parts or the hold points (Fig. 29 and Fig. 30). Do not hold it by the casing, electronic parts and other attachments. For transportation you can secure the power wheelchair using the two eyelets (Fig. 29 and Fig. 30). For use as a vehicle seat, see Chapter 2.2.

## 13. Storage/Shipping

If the wheelchair is to be stored or shipped, all loose parts which are not fixed, except for the drive wheels, must be removed and packed

If you wish to dispose of the product yourself, contact local recycling companies to enquire about the regulations that apply to your place of residence.

individually in an appropriate cardboard box. The power circuit must be open when stored for long periods of time. The individually packed parts can then be packaged together in a larger cardboard box. We recommend keeping the original packaging and storing it with these purposes in mind so that it is available if required. Your wheelchair then has optimum protection against environmental influences during storage or transport.

## 14. Disposal

If your wheelchair is no longer being used and is to be disposed of, contact your mobility dealer.

## 15. Warranty

1. For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from

production and material defects. In case of evidence of material or manufacturing defect, defective parts will be replaced free of charge.

2. This warranty excludes claims for the following reasons:

- a) normal wear and tear, which include but is not limited to the following parts where fitted; Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.
- b) overloading the product such as, for example, by exceeding the maximum user weight or load.
- c) the product or part has not been maintained or serviced in accordance with the Instruction manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.
- d) Accessories have been used which are not original accessories.
- e) The product or part has been damaged through negligence, accident or incorrect use.
- f) changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
- g) Repairs have been carried out, before our customer service has been informed of the circumstances.

3. To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.

4. For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1).

5. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.

6. If the repair fails after a reasonable time, the customer can, at his discretion, request a reduction, compensation or the termination of the purchase contract.

7. The guarantee is subject to the law of the country in which the product was purchased from Bischoff & Bischoff.

Observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your dealer can provide you with this.

## Warranty certificate

**Product: Terra / Terra SX**

Serial number:\*

\* (To be filled in by the dealer)

**Dealer:**

Date and seal

(Pay attention to our general terms and conditions.)

\*Signs are located on the seat bracket

Non-observance of the operating instructions as well as improper use, maintenance jobs and technical changes or adaptations without the permission of the Bischoff & Bischoff company, lead to guarantee loss and the loss of product liability in general.

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