Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi
Manuale di istruzioni

DH-44 L XL
DH-44 XL

DE


EN

ES

FR

IT

Enjoy mobility.

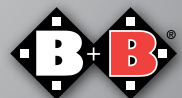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www.bischoff-bischoff.com

Index

1.	Preliminary remark	34	9.	Use in the shower	52
1.1.	Signs and symbols	34	10.	Care instructions for the user	53
2.	Important safety instructions	35	10.1.	Cleaning	53
2.1.	Indication	36	10.2.	Disinfection	53
2.2.	Contraindication	36	11.	Maintenance	54
2.3.	Declaration of Conformity	36	11.1.	Maintenance instructions	54
2.4.	Responsibility	37	11.2.	Maintenance instructions for the specialised trade.	55
3.	Intended use	37	12.	Repair	55
4.	Product and delivery overview	37	13.	Storage and dispatch	55
4.1.	Inspection of the delivery	37	14.	Waste disposal	56
4.2.	Scope of delivery	38	15.	Passing on Reuse	56
5.	Overview	38	16.	Useful life	56
5.1.	Shower chair with backrest	39	17.	Guarantee	57
5.2.	Shower stool	40			
5.3.	Type plate and serial number	41			
6.	Overview and assembly	42			
6.1.	Shower chair with backrest Individual parts	42			
6.1.1.	Overview of individual parts	43			
6.2.	Assembling the shower chair	44			
6.3.	Shower stool without backrest	46			
6.3.1.	Overview of the individual parts	47			
7.	Assembling the shower stool	48			
7.1.	Seat height adjustment	49			
7.2.	Check before use	49			
8.	Technical data	50			

1. Preliminary remark

Dear user,
You have chosen a high-quality shower chair/stool from Bischoff & Bischoff. Thank you for your confidence in us. These operating instructions contain all the necessary information for assembly, operation and care. Please read and follow these operating instructions, including all safety instructions, carefully before using the product for the first time. These instructions are an integral and necessary part of the product. Keep the operating instructions to hand and pass them on if you pass on the product. For people with visual impairments, this document is available as a PDF file on our website [www. bischoff-bischoff.com](http://www.bischoff-bischoff.com).

Note

Despite careful research and editing, our operating instructions may contain incorrect information. We reserve the right to make technical changes. Illustrations may include accessories. The latest version of the operating instructions can be found in the download area on our website: www.bischoff-bischoff.com

1.1. Signs and symbols

Important! Indicates particularly useful information in the respective context.

Attention! Indicates particularly safety-relevant information. Observe the operating instructions!

2. Important safety instructions

- Max. user weight of the DH-44 XL and DH-44 L XL is 170 kg.
- Use only at home, if necessary with the support of an assistant.
- Be aware of the risk of pinching when setting up.
- The seat is only suitable for sitting on.
- Avoid leaning out far to the side. Danger of tipping!
- The shower chair/stool must be secured against unauthorised use!
- No additions or modifications may be made to the product.
- The shower chair/stool may only be used with accessories approved by Bischoff & Bischoff.
- Exposure to sunlight can cause the surfaces of the shower chair/stool to become very hot and lead to burns. Therefore, avoid direct sunlight.
- At low ambient temperatures, metal parts can cool down considerably, so avoid skin contact.
- The seat and upholstery materials meet the flammability resistance requirements of EN 1021-2 and ISO 8191-2, but do not expose upholstery or other parts to sources of fire, such as cigarettes.
- Reporting of incidents.

If you as the operator, user or their relatives discover possible defects or functional limitations, please contact an authorised specialist dealer immediately. Operators, users or their relatives should inform their authorised dealer who provided them with the product of any suspected serious incidents that directly or indirectly had or could have one of the following consequences and which affect them. You can also report this to the competent higher federal authority.

- the death of a patient, user or other person,

- the temporary or permanent serious deterioration in the state of health of a patient, user or other person,
- a serious threat to public health.

Caution! Parts heated by sunlight can cause skin injuries.

2.1. Indication

The use of the shower chair/stool for adults is indicated in the event of unsteadiness or inability to stand due to functional impairment:

- of the lower extremities with restricted or cancelled mobility
- muscle strength (e.g. loss of strength due to malignant diseases, chronic, advanced cardiovascular and lung diseases) and/or
- Damage to the vestibular functions or tendency to fall due to other causes with a sufficiently stable trunk posture for sitting and sufficient residual functions of the upper extremities.

2.2. Contraindication

This shower chair/stool for adults is unsuitable for

- Inability to sit
- severe limitations of cognitive ability
- Perceptual disorders

2.3. Declaration of Conformity

We Bischoff & Bischoff GmbH declare under our sole responsibility that the shower stool DH-44 XL and the shower chair DH-44 L XL comply with all requirements of MDR 2017/745 that are applicable.

2.4. Responsibility

We can only provide a warranty if the product is used under the specified conditions and for the intended purposes, if modifications, extensions, repairs and maintenance work are only carried out by persons authorised by us to do so and if the shower chair/stool is used in accordance with all instructions for use. A warranty is also excluded in the event of damage caused by improper handling of fire sources, moisture and liquids such as hairdryers, cigarettes, cleaning agents or chemicals.

3. Intended use

The everyday aid supports adults with restricted mobility who have the specified indications in using bathing or showering facilities at home.

The maximum user weight of the DH-44 XL / L XL of 170 kg must not be exceeded. The recommended body height is 1.6 m to 2.0 m. The chair must not be used as a climbing aid or for similar purposes.

4. Product and delivery overview

4.1. Inspection of the delivery

All Bischoff & Bischoff products undergo a proper final inspection and are labelled with the CE mark.

The Bischoff & Bischoff shower chair/stool is supplied in a special cardboard box. Delivery and instruction of the product is carried out by a qualified medical supply retailer.

After unpacking, keep the cardboard box if possible. It can be used to store or return the product at a later date.

In the case of dispatch by rail or forwarding agent, the goods must be checked immediately for transport damage in the presence of the bearer. If there is any damage, please contact our customer service immediately.

Check that the contents are complete (see section 4.2) and undamaged. In the event of irregularities or damage, please contact our customer service.

4.2. Scope of delivery

Your new Bischoff & Bischoff shower chair/stool is delivered in a Bischoff & Bischoff original box. To prevent damage during transport, attachments are supplied separately. The scope of delivery in the special box includes


- Shower chair or shower stool
- Operating instructions

5. Overview

To make handling your Bischoff & Bischoff shower chair/stool as easy as possible, we have also documented the written instructions with detailed illustrations.

The following illustrations are intended to help you with the designation and interaction of the individual assemblies (see Fig. 1 and 2).

5.1. Shower chair with backrest

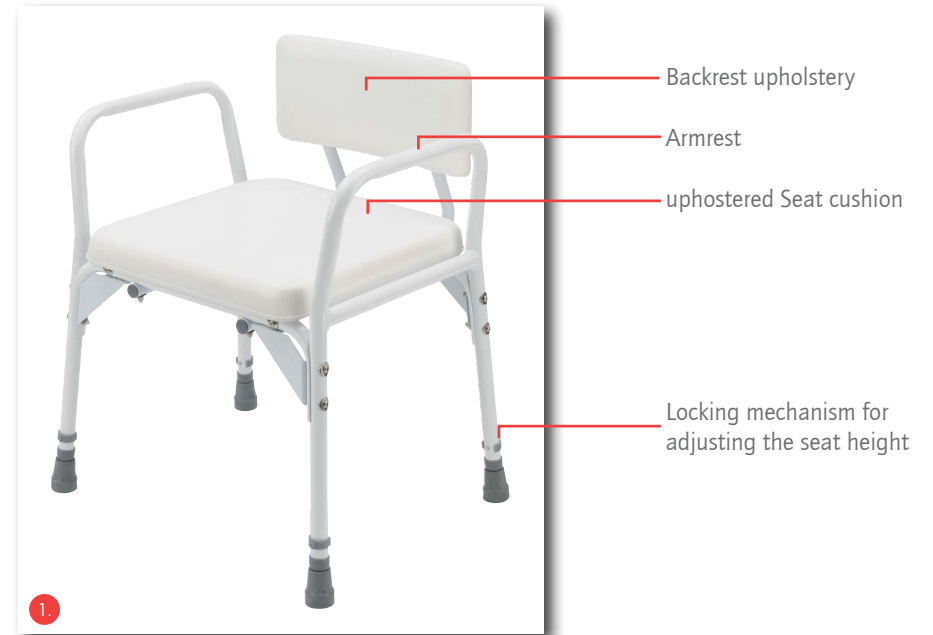
5.2. Shower stool

5.3. Type plate and serial number

The type plate and serial number are located on the frame tube below the seat.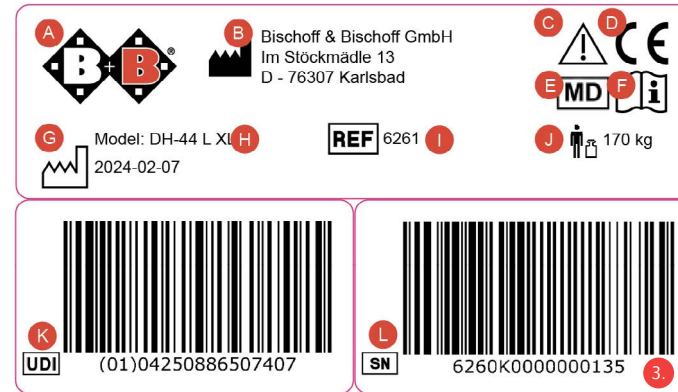


- A** Manufacturer logo
- B** Manufacturer information
- C** Attention! - Observe instructions for use
- D** CE mark
- E** Medical device
- F** Important! - Observe instructions for use
- G** Production date
- H** Model designation
- I** Model number
- J** Max. user weight
- K** UDI Part number
- L** Serial number

6. Overview and assembly

You can adjust your shower chair/stool yourself with a little manual dexterity. If you have any questions, please contact your specialist dealer.

6.1. Shower chair with backrest Individual parts

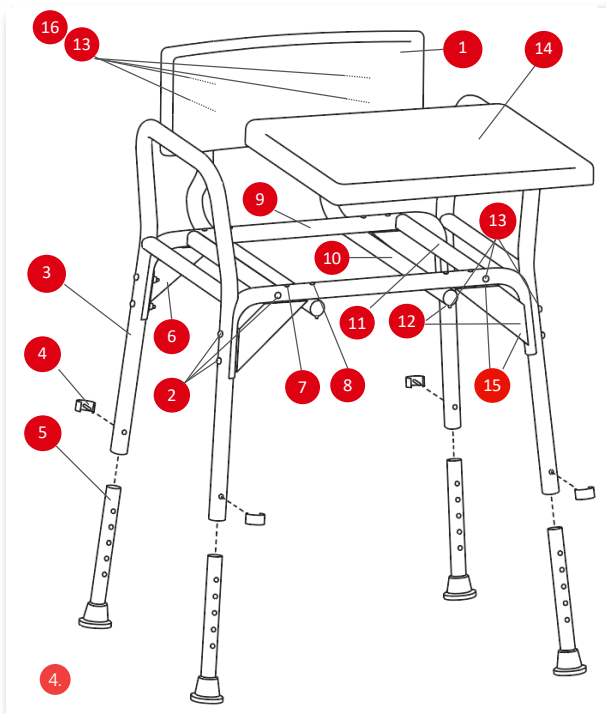6.1.1. Overview of individual parts

- 1 Back cushion
- 2 Hexagon socket screw M6 x 40 (12x)
- 3 Side panel with armrest (2x)
- 4 Locking pin (4x)
- 5 Chair leg with rubber foot (4x)
- 6 Support plate (4x)
- 7 Hexagon socket screw M6 x 45mm (4x)
- 8 Countersunk hexagon socket screw M6 x 50mm (4x)
- 9 Seat tube (2x)
- 10 Backrest tube (2x)
- 11 Cross tube (2x)
- 12 M6 nut (12x)
- 13 M6 wavy washer (20x)
- 14 Seat cover (1x)
- 15 M6 straight washer (12x)
- 16 M6 x 35 Allen screw (4x)

6.2. Assembling the shower chair

1. You will need the locking pins (Fig. 5) to attach the chair legs to the side frame.

2. Insert the chair base into the side section and then lock it at any height using the locking pin.

Caution! Ensure that the locking pins are correctly seated, otherwise there is a risk of accident.

3. Then screw the two cross tubes to the seat tubes. Do not tighten the screws (M6 x 40 mm) firmly at first so that the other parts can be fitted more easily (Fig. 7).

The side panel with armrest must be fitted together with the support plates (M6 x 40 mm screws). Use the corrugated washers on the outside and the flat washers on the inside (Fig. 8).

5. Then screw the two backrest tubes to the frame of the shower chair/stool from the underside (Fig. 9). To do this, insert the countersunk hexagon socket screw M6 x 50 mm through the hole from above. Screw the nut to the underside with a corrugated washer.

Please ensure that you fit the backrest tubes from the underside, otherwise the seat cushion can no longer be attached (Fig. 9).


6. Then attach the backrest upholstery using the 4 enclosed screws (M6 x 35 hexagon socket screws and corrugated washers). Ensure that the screws are the correct length, otherwise the backrest upholstery will be damaged. Now tighten all screws firmly.

7. Finally, the seat cushion is mounted with 4 screws (hexagon socket screws M6 x 45mm and flat washer). The seat cushion is horizontally symmetrical so that the front side shown here can also be turned to the rear.

6.3. Shower stool without backrest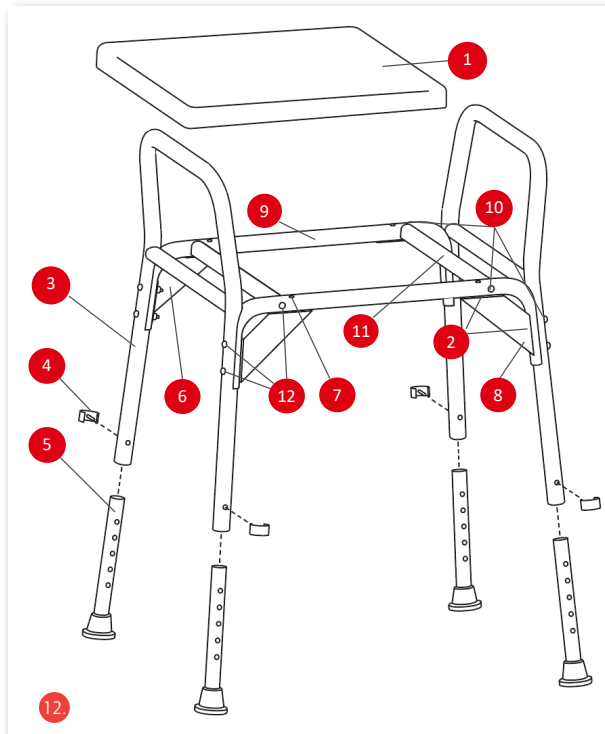

6.3.1. Overview of the individual parts

- 1 Seat cushion (1x)
- 2 Straight washer M6 (12x)
- 3 Side panel with armrest (2x)
- 4 Locking pin (4x)
- 5 Chair leg with rubber foot (4x)
- 6 Support plate (4x)
- 7 Hexagon socket screw M6 x 45mm (4x)
- 8 Nut M6 (8x)
- 9 Seat tube (2x)
- 10 Corrugated washer M6 (12x)
- 11 Cross tube (2x)
- 12 Hexagon socket screw M6 x 40mm (12x)

Caution: Risk of trapping! There is a risk of entrapment during assembly.

7. Assembling the shower stool


1. You will need the locking pins (no. 4) to attach the chair legs to the side frame.

2. Insert the chair base into the side frame and then lock it at any height using the locking pin.

Caution! Ensure that the locking pins are correctly seated, otherwise there is a risk of accident.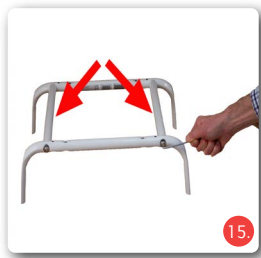

3. Then screw the two cross tubes to the seat tubes (Fig 15). Do not tighten the screws (M6 x 40 mm) firmly at first, as this will make it easier to fit the other parts.

4. Mount the side panel with armrest together with the support plates (M6 x 40 screws). Now tighten all screws firmly.

5. Finally, the seat cushion is mounted with 4 screws (M6 x 45mm). The seat cushion is horizontally symmetrical, so that the front side can also be turned backwards.

7.1. Seat height adjustment

Seat height

The DH-44 L XL and DH-44 XL models have four non-slip feet and are height-adjustable in five stages. Carry out this work in pairs and lay the chair down.

- Pull the locking pin out of the chair leg and adjust the shower chair/stool to the desired height (Fig. 18/19).
- After adjustment, make sure that the locking pins are correctly engaged again (Fig. 20).

7.2. Check before use

Before use, ensure that all screws are tight and that the chair is standing on a level surface.

After adjusting the height, it is particularly important to ensure that the locking pins are correctly engaged.

The rubber feet are a wearing part and must not be damaged. If they are damaged, they can be replaced by your specialist dealer if necessary.

8. Technical data

	DH-44 XL	DH-44 L XL
Dimensions		
Total width:*	595 mm	595 mm
Depth*:	510 mm	510 mm
Height:	625 - 725 mm	690 - 790 mm
Seat height:	455 - 555 mm	455 - 555 mm
Seat width:	470 mm	470 mm
Width between armrest:	540 mm	540 mm
Seat depth:	430 mm	430 mm
Seat cushion L x B x H	425 x 430 x 40 mm	425 x 430 x 40 mm
Armrest height	175 mm	175 mm
Backrest height	n.a.	260 mm
Armrest L x B	260 x 25	260 x 25
Weights		
User weight:	170 kg	170 kg
Weight with seat cushion:	4,4 kg	5,6
Materials		
Rahmen:	Aluminium powder-coated white	
Seat cushion and backrest:	PVC, water resistant	
Armrest:	PE (Polyethylen)	
Flammability:	flame retardant	

	DH-44 XL	DH-44 L XL
Equipment		
	Upholstered seat	
Colour:	white	
Delivery condition:	In individual parts with assembly instructions	
Operating conditions		
Temperature:	-20 to +50 °C	
Humidity:	20 to 80 %	
Heat resistant	to 125°C	
Storage conditions		
Temperature:	-20 to + 65 °C	
Humidity:	45 to 60 %	
HMV-No.	04.40.03.1033	04.40.03.2050
*Incl. rubber disc on the rubber foot. The specified dimensions are subject to manufacturing tolerances of +/- 10 mm		

9. Use in the shower

Caution! Avoid leaning out far to the side. Danger of tipping!

Use of the shower chair/stool

Use the shower chair/stool like a normal stool. Do not lean backwards or out to the side, as this may cause it to tip over. The upholstery material is made of water-repellent plastic.

Caution! Avoid leaning far out to the side as there is a risk of tipping. Observe the operating instructions!

Lifting points

You can lift the shower chair/stool by the metal frame to carry it over obstacles or steps. Avoid lifting the chair by the seat.

Beware of the risk of slipping! There is a risk of slipping when sitting down or standing up in wet rooms, especially if there is soap on the floor.

10. Care instructions for the user

10.1. Cleaning

The seat and backrest materials can be cleaned with warm water using a sponge and/or a cloth with a soft brush. For stubborn dirt, a commercially available cleaning agent can be added to the water.

Note: Never use aggressive cleaning agents, such as solvents / abrasive cleaners or hard brushes.

Plastic parts are best cleaned with a commercially available plastic cleaner. Please observe the product information for the cleaning agent.

- Plastic panelling is attacked by non-ionic surfactants and solvents, especially alcohols!
- Wipe the frame parts of the shower chair/stool with a damp cloth.
- For heavier soiling, also use a mild cleaning agent.
- Wash the armrests, seat and backrest upholstery with a mild detergent.
- Upholstery parts can be washed at 40°C. Dry at room temperature.

Caution! Do not use a high-pressure or steam cleaner.

10.2. Disinfection

The shower chair/stool must be disinfected after use. The shower chair/stool can be disinfected with a standard household disinfectant. Use approved spray disinfectants or alcohol-based wipe disinfectants.

- In the case of multiple product users, a commercially available disinfectant is prescribed.
- Only use wipe disinfectants approved by the Robert Koch Institute (RKI) and the Verbund für Angewandte Hygiene e.V. (e.g. Bacillol)

AF or Kohrsolin FF); for contact time and concentration, see manufacturer's recommendations.

- Caution: Damaged upholstery parts cannot be adequately disinfected. These should be replaced.
- Unsuitable disinfectants may cause damage to the surfaces. This can result in longer-term functional restrictions. Please refer to our enclosed overview of the most important care, hygiene and maintenance work.

Caution: Risk of infection! The shower stool must be cleaned after each use.

Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your specialist dealer will be happy to provide you with these.

11. Maintenance

11.1. Maintenance instructions

Your shower chair/stool is made of high-quality materials for long and comfortable use. Every product is thoroughly tested before it leaves our premises. Nevertheless, it is possible that a defect may occur. If you notice a fault with your shower chair/stool, it must no longer be used.

Please contact the medical supply retailer who supplied you with the product for repair. They will be happy to help you. Please check regularly before using your shower chair/stool that the rubber feet, armrests and backrest, the upholstery and the height adjustment are not damaged and are fully functional.

You must check your shower stool regularly according to the maintenance schedule and ensure that it is in perfect condition.

11.2. Maintenance instructions for the specialised trade.

Typical wearing parts are the rubber feet. Contact your specialist dealer for spare parts.

We recommend that you bring your shower chair/stool to your specialist dealer for inspection at least once a year.

In the event of faults or defects, return the shower chair/stool to your specialist dealer immediately for repair.

The specialist dealer will be provided with the necessary information and documents for repair and maintenance on request.

Please contact Bischoff & Bischoff customer service if you have any questions. The customer service contact details can be found on the back of these operating instructions or on our website www.bischoff-bischoff.com

Check your wheelchair regularly according to the maintenance schedule.

12. Repair

Repairs and maintenance work may only be carried out by authorised partners (medical supply stores or specialist dealers). can be carried out. Please call our customer service team to find a partner near you.

13. Storage and dispatch

If the shower chair/stool is to be stored or dispatched, all insertable and unattached parts must be removed and packed individually. The individually packed parts can then be packed together in a larger box. It is advisable to keep the original packaging and store it for

these purposes so that it is available when needed. Your shower chair/stool will then be optimally protected against environmental influences during storage or transport. When storing your shower chair/stool, please observe the information in the technical data. To prevent mould growth and damage to the upholstered parts, the storage location should be as dry as possible and not exposed to direct sunlight.

14. Waste disposal

If your shower chair/stool is no longer in use and is to be disposed of, please contact your specialist dealer.

If you would like to dispose of the waste yourself, enquire with local recycling companies about the regulations in your area.

15. Passing on Reuse

The shower chair/stool is suitable for reuse. Before it is passed on to another user, it must be serviced and sanitised by a specialist dealer. When passing the shower chair/stool on to another user, please remember to pass on these operating instructions and the address of your specialist dealer to the new user.

The shower chair must be checked according to the maintenance schedule and be in perfect working order.

16. Useful life

The expected service life of the DH-44 XL and DH-44 L XL is up to 5 years if used as intended and the safety, care and maintenance instructions are followed. The shower chair/stool can continue to be used beyond this period if it is in a safe condition.

17. Guarantee

1. The warranty period for products supplied by Bischoff & Bischoff is 24 months from the date of purchase. The products are delivered free of manufacturing and material defects. If there is evidence of a material or manufacturing defect, defective parts will be replaced free of charge.
2. Warranty claims are void if a repair or replacement of the product or a part is necessary for the following reasons:
 - a) Normal wear and tear, including in particular the following parts, if installed: Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.
 - b) Overloading the product, e.g. exceeding the maximum user weight or payload.
 - c) The product or part has not been maintained or serviced in accordance with the operating instructions, the care and hygiene instructions or the manufacturer's recommendations listed in the maintenance instructions.
 - d) Accessories were used that were not original accessories.
 - e) The product or a part has been damaged due to negligence, accident or improper use.
 - f) Changes/modifications have been made to the product or parts that deviate from the manufacturer's specifications.
 - g) Repairs were carried out before our customer service department was informed about the respective issue.
3. To assert a warranty claim, please notify your specialist dealer immediately, giving a precise description of the problem. The product must be repaired by a specialist dealer authorised by Bischoff & Bischoff.
4. For parts that are repaired or replaced under warranty, the warranty shall be extended to the remaining warranty period for the product in accordance with Section 1.
5. Original spare parts installed at the customer's expense are guaranteed for 12 months after installation.

6. If the rectification of defects fails after a reasonable period of time, the buyer may, at his discretion, demand a reduction of the remuneration or withdraw from the contract.

7. The warranty is subject to the law of the country in which the Bischoff & Bischoff product was purchased.

Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your specialist dealer will be happy to provide you with these.

A warranty period of one year applies to replacement devices and repairs.

Failure to follow the operating instructions, improperly performed maintenance work, and in particular technical modifications and additions (attachments) without the consent of Bischoff & Bischoff GmbH will void both the warranty and general product liability.

Note

Despite careful research and editing, our operating instructions may contain incorrect information. We reserve the right to make technical changes. Illustrations may include accessories. The latest version of the operating instructions can be found in the download area on our website: www.bischoff-bischoff.com

Guarantee certificate

Product: DH-44 XL / DH-44 L XL

Model number:*

Serial number:*

* (to be entered by the dealer)

Specialist dealer:

Date and stamp

Please note our general terms and conditions)

*Information signs are located on the side frame

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