Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi
Manuale di istruzioni

Capero

DE


EN

ES

FR

IT

Enjoy mobility.

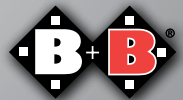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Notizen

Index

1. Introduction	28	6. Use	39
1.1. Signs and symbols	28	7. Technical data	40
2. Important safety instructions	29	8. Maintenance instructions for the user	42
2.1. General safety instructions	29	8.1. Cleaning	42
2.2. Brakes	31	8.2. Disinfection	42
2.3. Indications	31	8.3. Rollator check by the user	43
2.4. Contraindications	32	8.3.1. Brake Inspection	43
2.5. Declaration of conformity	32	8.3.2. Checking the tyres	43
2.6. Responsibility	32	8.3.3. Maintenance advice	44
2.7. Service life	32	8.3.4. Maintenance instructions for the retailers	44
3. Purpose	33	9. Storage/shipping	44
4. Description of product and contents	33	10. Re-use	45
4.1. Inspection on delivery	33	11. Repairs	45
4.2. Type plate and serial number	34	12. Transport	45
4.3. Contents	35	13. Disposal	45
4.4. Summary	35	14. Warranty certificate	46
4.5. Reflectors	36		
4.5.1. Front / side reflectors	36		
4.5.2. Rear reflectors	36		
5. Assembly/adjustments	37		
5.1. Unfolding	37		
5.2. Folding	37		
5.3. Attaching the bag	37		
5.4. Adjusting the push handles	37		
5.5. Adjusting / activating the brakes	38		
5.6. Wheels and tyres	38		
5.7. Tipping Aid	39		

1. Introduction

Dear user,

By purchasing a Bischoff & Bischoff Capero rollator, you are now the owner of a top quality product that is setting new standards in terms of its variability. This instruction manual is intended to make you familiar with the operation and use of the rollator.

Please read and follow the instruction manual carefully before using your new rollator for the first time. The instruction manual is an integral and necessary component of the product. Keep the instruction manual where you can refer to it, and if you transfer the rollator to another user, please provide the manual to the new owner.

For visually impaired users, this document is available as a PDF file at our website www.bischoff-bischoff.com.

Any repairs and adjustments require special technical training and may therefore only be carried out by authorized Bischoff & Bischoff dealers.

1.1. Signs and symbols

Important! This is especially useful information on a given subject.

Warning! These are especially safety-relevant instructions. Follow the directions in the manual!

Note!

Despite careful research and editing, incorrect information may slip into our operating manuals. Subject to technical changes. Figures may contain accessories. The most up-to-date version of the operating manual is available for download from our website:

www.bischoff-bischoff.com

2. Important safety instructions

2.1. General safety instructions

To avoid falls and dangerous situations, you should first practise using your new rollator on level, manageable terrain. An accompanying person is recommended in this case.

- Only drive on level, solid ground.
- Be aware of the risk of entrapment between moving parts when adjusting, using and maintaining the wheelchair.
- The driver must not be under the influence of alcohol or drugs.
- Before using the rollator, please check that all attached parts are properly fastened.
- Keep clothing and body parts away from gaps between moving parts!
- Avoid collisions of any kind! Avoid driving into an obstacle (step, kerb) without braking or jumping down from a step.
- Replace the tyres as soon as the tread is worn down.
- When parking the rollator, apply both parking brakes.
- Use the parking brake only for parking, it must not be used to slow down the drive.
- Avoid driving on uneven, loose surfaces.
- The seat and upholstery materials meet the requirements of resistance to flammability according to EN 1021- 2/ ISO 8191-2, nevertheless do not expose upholstery or other parts to fire sources - such as cigarettes.
- The wheelchair may only be used to transport one person. The wheelchair must not be used to transport loads or more than one person, note the maximum load.

Improper use can lead to dangerous situations!

- Note that upholstered parts that are exposed to direct sunlight can heat up and cause skin injuries when touched. Therefore, cover these parts or protect the wheelchair from sunlight.

Protect the wheelchair from sunlight to avoid burns from heated components.

- Also note that the frame and upholstered parts can cool down considerably in winter. If possible, do not park the wheelchair outside in cold weather.
- Observe the storage and operating conditions.
- Max. user weight for Rollator Capero is 136 kg
- Please note there is a danger of trapping fingers and/or clothing when folding and unfolding. Do not catch any clothing or body parts between the moving parts.
- Please note the minimum insertion depth of the handles bars!
- Each brake only affects one running wheel!
- The bag can be loaded with max. 5 kg.
- When cleaning, do not use any aggressive or corrosive cleaning agents!
- Check the rollator for correct assembly before every use. The brake operation should be checked.
- The brakes must be inspected by a specialist dealer every six months.
- Make sure your brakes are operating properly before each use.

The rollator is equipped with reflectors on the left and right as well as on the front and rear of the frame. There are also additional reflectors on the left and right of the front wheel fork.

For your own safety, make sure that the reflectors are not covered so that you can be seen better in traffic.

Reporting incidents:

If you, as the operator, user or their relatives, detect possible defects or functional limitations, please contact an authorised dealer immediately. Operators, users or their relatives should inform the dealer who supplied the dealer who provided you with the product of any suspected serious incidents which directly or indirectly have had or could have one of the following and by which they are affected. You can also report this to the competent higher federal authority report

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of the health of the state of health of a patient, user or other persons,
- a serious risk to public health.

2.2. Brakes

Service brake:

The service brake of the rollator is operated with the two hand brake levers on the push handles (see chapter 5.5). The service and parking brakes each act on a wheel.

Parking brake:

Push the brake levers down to activate the parking brake; pull them up to release the parking brake (see chapter 5.5).

2.3. Indications

- The rollator is designed to assist adults with limited mobility when walking indoors and outdoors, e.g. in the home, when shopping or when going for walks outside.
- The seat allows the user to rest during longer periods of walking.
- Please use the rollator on a level and firm surface.
- The rollator may not be used as an aid for climbing or for transportation of heavy loads.
- The recommended height is min. / max.: 1,65 m / 1,90 m.

2.4. Contraindications

The provision of a walking aid is unsuitable for adult people with:

- Joint damage / joint contractures
- Limb loss on the arms
- Perceptual disorders
- severe balance disorders
- reduced and insufficient vision
- severe limitations of cognitive abilities
- Inability to sit

2.5. Declaration of conformity

We are solely responsible for issuing the EU declaration of conformity. We declare that the listed product complies with Regulation (EU) 2017/745 and, where applicable, other relevant Union legislation.

2.6. Responsibility

We are only able to provide a warranty if the product is used in accordance with the specified conditions and for the intended purposes. If modifications, repairs or maintenance works are made, please make sure only authorised persons perform these works in accordance to the instructions of Bischoff & Bischoff.

2.7. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life is up to 5 years. Beyond this time the rollator can be used continuously, provided it is in a safe condition.

3. Purpose

The rollator is used by adults with limited mobility for the indicated indications to maintain, promote and ensure walking and also to provide support during everyday activities (transport function, sitting down when exhausted). The maximum user weight of 136 kg must not be exceeded for the Capero.

4. Description of product and contents

4.1. Inspection on delivery

All Bischoff & Bischoff rollators are subject to a proper final inspection in our house and receive the CE mark.

The rollator is supplied in a special box. After unpacking save the box if possible. You may need it later for storage or to return the product.


Assembly, adjustment to your body size and first instructions are taken care of by your qualified medical supply dealer.

If delivered by rail or a forwarding agent, the product is to be checked with regard to transportation damage at once in the presence of the shipper.

Check the contents for completeness and make sure nothing is damaged. Should you find any irregularities or damage, please contact our customer service.

4.2. Type plate and serial number

The type plate (Fig. 1) and serial number are located on the bottom of the frame.

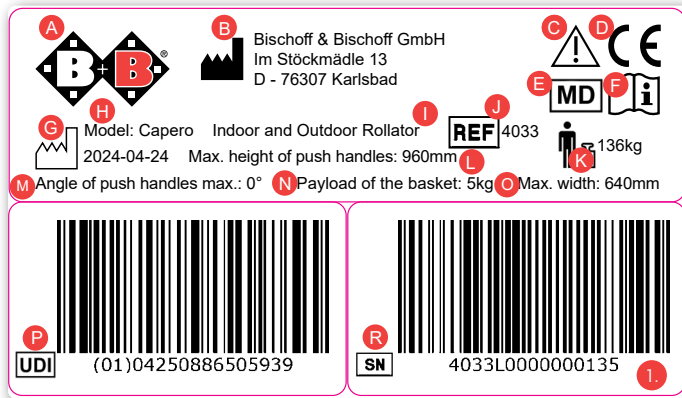- A Manufacturer's logo
- B Manufacturer's information
- C Caution! - Follow instructions for use
- D CE mark
- E Medical Device
- F Important! - Follow instructions for use
- G Date of manufacture
- H Model designation
- I Application class
- J Model number
- K Maximum load capacity
- L Max height of the push handles
- M Angle of the push handles
- N Carrying capacity of bag
- O Required dimensions
- P UDI number
- R Serial number

4.3. Contents

After receiving the goods, please check the contents for completeness right away. The content consists of:

- Outer package
- Rollator pre-assembled
- Instruction manual
- Bag

4.4. Summary

Product overview Capero (Fig. 2)

4.5. Reflectors

4.5.1. Front / side reflectors

4.5.2. Rear reflectors

For your own safety, make sure that the reflectors are not covered, as this will make it easier to see you in traffic.

5. Assembly/adjustments


5.1. Unfolding

- Position the rollator in the travel direction.
- Release the locking brakes (see sec. 5.5.).
- Press the frame tubes of the seat downward to unfold the rollator (Fig. 5). Make sure the tubes are aligned properly and snap into place.

5.2. Folding

- Activate the locking brakes.
- Hold the rollator firmly with one hand on the handle. With your other hand pull the grip on the seat upward (Fig. 6).
- When doing so you must overcome the resistance of the locking device.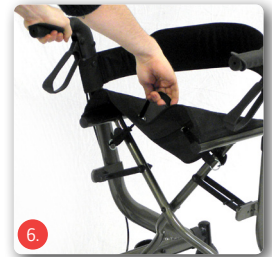

5.3. Attaching the bag

- The rollator is unfolded. Activate the locking brakes.
- Position yourself against the travel direction in front of the rollator.
- The bag can be inserted from above in the guide rails on the left and right on the rollator (Fig. 7).

5.4. Adjusting the push handles

- Now adjust the push handles so that they are located at the height of your wrists. Select the nearest hole in the push tube.
- The rollator is unfolded. Activate the locking brakes.
- Hold the push handle firmly with one hand. With the other hand, activate the locking device (Fig. 9).
- Move the push handle to the desired height until it latches audibly.
- Repeat this procedure on the other side.
- Check to make sure that both the push handles are at the same height.

The push handles can also be removed with this procedure. To do so, press the pushbutton and pull the push handle up and out.

5.5. Adjusting / activating the brakes

Service brake

To activate the service brake, pull both handbrake levers upward on the push handles upward (Fig. 9). Service and locking brake act on the fixed wheels.

Locking brake

- Press down on both brake levers to activate the locking device (Fig. 10).
- To release the locking brake pull both brake levers back into middle position.

Adjusting the brakes

We recommend adjusting the brakes together with your specialist dealer. If you have the necessary technical knowledge, you can do this yourself as well.

The brake is set properly, if the brake pins do not touch the wheel and the locking brake can be activated easily.

- Loosen the locknut (Fig. 11, arrow) so the adjustment nut can turn.
- Now turn the wide adjustment nut further inwards or outwards to increase or decrease the braking effect.
- Retighten the locknut.
- Repeat the procedure on the other brake if necessary.

5.6. Wheels and tyres

Your rollator is equipped with flat-proof PU wheels 190 x 43 mm. in front and 175 x 43 mm in back.

5.7. Tipping Aid

The tipping aid is intended to make it easier for users to overcome curbs and small obstacles.

Step on the tipping aid with your foot (Fig. 12) and at the same time pull on the push handles. The front wheels lift slightly off the ground and with a little force you can push the rollator slightly past the obstacle.

6. Use

Walking

Make your first attempts to walk with the rollator on a level, firm surface. Familiarise yourself with the function of the brakes. Let someone accompany you during this initial period.

For an anatomically correct gait, the push handles must be set at proper height (see sec. 5.4.).

- Place yourself in the rollator so that your ankles are at the height of the rear wheels.
- Stand with good posture.
- Grasp the push handles with both hands and your arms bent at a slight angle.

When walking with the rollator try to stay as close as possible with the body to the push handles. This increases the stability and safety against tipping over; in addition, avoid a crooked back. Try to generally walk in the rollator and not behind it. Take small steps and do not push the rollator too far ahead.

You can determine the change of direction and speed by pushing the rollator. Always use the brakes to slow down your movement. The rollator could otherwise turn with the unbraked wheel and the braked wheel, and could fall over.

Go around obstacles whenever possible. Try to use ramps or take lifts. If this is not possible, the rollator must be carried over the obstacle. Find a second person to help you do this, if possible.

The seat

The rollator is also designed to be used as a seat by the user for short breaks. It is therefore equipped with a seating area and a back support. You must absolutely activate the locking brakes before sitting down. The rollator could roll away and you could fall.

Avoid leaning out to the side, as the walker could tip over – risk of injury!

Lifting points

You can lift the rollator on the grips or frame sections, in order to carry it over obstacles or steps.

7. Technical data

Dimensions and weights

Type:	Lightweight rollator
Model:	4033 - platin 4034 - night blue
Height (height of push handles):	860 mm - 960 mm
Length:	765 mm
Width:	636 mm
Height folded:	860 mm - 960 mm
Length folded:	765 mm
Width folded:	236 mm
Width between handles:	470 mm
Width of handles:	30 mm
Angle of push handles max.:	0°
Seat height:	549 mm
Seat width:	450 mm
Effective seat width:	465 mm
Seat depth:	250 mm
Turning Radius: Radius/Diameter:	686 / 1372 mm
Weight:	5.8 kg (without accessories)

Weight with bag, back-strap and cane holder:	6.2 kg
Max. load:	136 kg
Max. load of bag:	5.0 kg

Tyres

Rear wheels:	175 x 43 mm
Steered wheels:	190 x 43 mm

Material

Frame and all attachments corrosion-resistant	
Frame:	Aluminium, with corrosion-resistant coating
Screws:	galvanised/anodised
Handles:	PP
Attachments:	Nylon
Seat:	Vinyl coated PE
Flammability:	flame-retardant

Operating conditions

Temperature:	0 to 50 °C
Air humidity:	20 to 80 %

Storage conditions

Temperature:	-20 to + 60 °C
Air humidity:	45 to 60 %
HMV-No.:	10.50.04.1275

Equipment

- Including bag and cane holder
- Anatomically shaped handles
- Combined operating and locking brake
- Stable aluminium design, corrosion-resistant coating
- Non-rotating handholds

The specified dimensions are subject to manufacturing tolerances of +/- 10 mm.

8. Maintenance instructions for the user

8.1. Cleaning

The seat and back materials can be cleaned with warm water using a sponge and/or a soft brush. In case of stubborn dirt, a commercial cleaning agent can be added to the water.

Note: Never use aggressive cleaning agents such as solvents / scouring agents or hard brushes.

- Plastic parts are best cleaned with a commercially available plastic cleaner. Please observe the product information of the cleaning agent.
- Plastic claddings are attacked by non-ionic surfactants and solvents, especially alcohols!
- Wipe the frame parts of the wheelchair with a damp cloth. For heavier soiling, also use a mild cleaning agent.
- Clean the wheels with a damp brush with plastic bristles (do not use a wire brush!).
- Do not use a high-pressure cleaner or automatic carwash.
- Wash the push handles, arm pads, seat and back pads with a mild detergent.
- Upholstery parts can be machine washed at 40°C. Do not machine dry, only dry at room temperature.

8.2. Disinfection

- Before disinfection, the wheelchair must be thoroughly cleaned. The wheelchair can be disinfected with a household disinfectant. Use approved spray disinfectants or alcohol-based wipe-on disinfectants.
- In the case of multiple product users, a commercially available disinfectant is prescribed.
- Use only wipe-on disinfectants approved by the Robert Koch Institute (RKI) and the Association for Applied Hygiene (Verbund

für Angewandte Hygiene e.V.) (e.g. Bacillol AF or Kohrsolin FF); see manufacturer's recommendations for exposure time and concentration.

- Caution: Damaged upholstery parts cannot be sufficiently disinfected. These should be replaced.
- Under certain circumstances, the surfaces can be affected by unsuitable disinfectants. This may result in long-term functional limitations.

You can find the latest version of the care and hygiene instructions in the download area on our website: www.bischoff-bischoff.com.

Attention: Danger of infection! If there are several users, the wheelchair must be cleaned after each use.

Attention! Do not use a high-pressure or steam steam cleaner.

8.3. Rollator check by the user

8.3.1. Brake Inspection

Make sure your brakes are operating properly before each use. The rollator should not roll away when the brakes are applied. The rollator should be easy to drive without grinding noises and with good direction stability when the brakes are released. If you notice unusual behaviour when braking, inform your qualified mobility dealer immediately and do not use the rollator.

8.3.2. Checking the tyres

Check whether the tyres are intact every time before driving. If the tyre is damaged, it must be repaired by an authorised dealer.

8.3.3. Maintenance advice

We recommend that the rollator is inspected by a specialist dealer at least once a year.

In case of malfunctions or defects on the rollator, it must be immediately handed over to the specialist dealer for repair.

The repair and replacement of parts is carried out by the specialist dealer.

On request, the specialist dealer will be provided with the necessary information and documents for repair and maintenance.

If you have any questions, please contact the Bischoff & Bischoff customer service.

You must check your rollator regularly according to the maintenance schedule and find it to be in good working order.

8.3.4. Maintenance instructions for the retailers

Maintenance must be carried out in accordance with the Bischoff & Bischoff maintenance schedule. A maintenance schedule is available on request. A training programme for the specialised trade is also available.

The rollator must be inspected in accordance to the service plan and must be in good condition.

9. Storage/shipping

If the rollator is to be stored or shipped, it is advisable to retain the original packaging.

Your rollator will have an optimum protection against environmental

influences during storage and transport.

To avoid the formation of mould and damage to the upholstered parts, the storage location should be as dry as possible and not exposed to direct sunlight.

10. Re-use

The rollator is suitable for re-use. For re-use, the rollator should firstly be thoroughly cleaned and disinfected in accordance with the enclosed care and cleaning instructions. Finally, the product should be professionally checked for general condition, wear and tear and damage and if necessary, repaired by an authorised specialist.

11. Repairs

Repairs and maintenance measures are only permitted to be carried out by authorised partners (medical houses or specialist suppliers). For a partner near you, please call our customer service team.

12. Transport

To make the rollator as easy to handle as possible for transport, remove all detachable parts.


Fold the rollator for transport in a car (Fig. 13).

13. Disposal

A worn out rollator has a valuable source of material, which can be recycled into the industrial cycle. Therefore, do not throw your rollator away with household waste, but allow it to be professionally disposed of by a state approved disposal department. Your local rubbish collection will have information regarding this. The environment will thank you.

If you wish to dispose the rollator yourself, please contact a local recycling company for the disposal regulations of your local jurisdiction.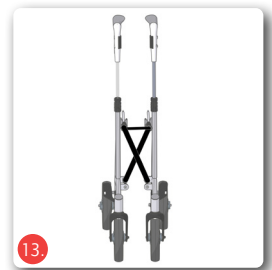

14. Warranty certificate

1. For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from production and material defects. Where it can be demonstrated that there is a material or manufacturing defect, defective parts will be replaced free of charge.

2. Claims from this warranty shall not arise, if a repair or replacement of a product or a part is required for the following reasons:

- a) normal wear and tear, which include but is not limited to the following parts where fitted: Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.
- b) overloading the product such as, for example, by exceeding the maximum user weight or load.
- c) the product or part has not been maintained or serviced in accordance with the operating manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.
- d) Accessories have been used which are not original accessories.
- e) The product or part has been damaged through negligence, accident or incorrect use.
- f) changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
- g) Repairs have been carried out, before our customer service has been informed of the circumstances.

3. To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.

4. For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1).

5. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.

6. If the rectification of defects fails after a reasonable period of time, the purchaser may, at its discretion, demand a reduction of the remuneration or withdraw from the contract.

7. The warranty is subject to the law of the country in which the Bischoff & Bischoff product was purchased.

In addition, please observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your specialist dealer will be happy to provide you with these.

Warning! Non-observance of the instruction manual, improper maintenance and in particular any technical changes and supplements (added fittings) without the approval of Bischoff & Bischoff will generally lead to forfeiture of warranty and product liability claims.

Warranty-Certificate

Product: Capero

Model number:*

Serial number:*

* (to be filled in by authorized dealer)

Retailer:

Date and stamp

(Please note our general business terms and conditions)

*The serial number is located on the side frame.

Note!

Despite careful research and editing, incorrect information may slip into our operating manuals. Subject to technical changes. Figures may contain accessories. The most up-to-date version of the operating manual is available for download from our website: www.bischoff-bischoff.com

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Bischoff & Bischoff GmbH