


# ALEVO - 4040/4041/4042

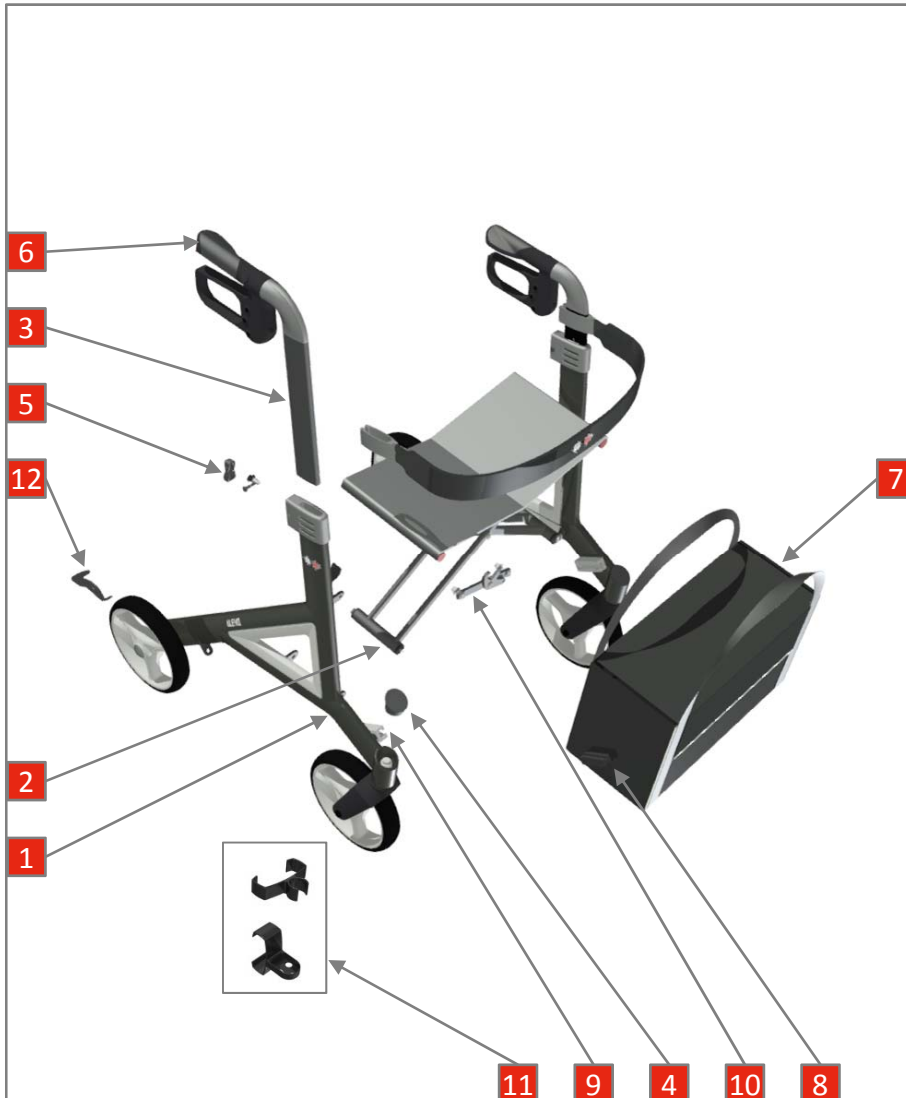


# BG1 – Frame




object list - BG1 - Frame		
object	AN	description
1	n/a	carbon frame right, mokka
1	n/a	carbon frame left, mokka
1	n/a	carbon frame right, aubergine
1	n/a	carbon frame left, aubergine
1	n/a	carbon frame right, silver
1	n/a	carbon frame left, silver
2	4040108000	crossbar complete
3	4040112000	handle tube right
3	4040113000	handle tube left
4	4040114000	plastic cap for castor
5	4040115000	locking bolt with spring for handle tube
6	4040130000	handle grip right with reflector foil
6	4040131000	handle grip left with reflector foil
7	4045146000	shopping bag
8	4040147000	guiding profile shopping bag
9	4040148000	shopping bag holder
10	4040151000	seat lock with spring + nut left
10	4040152000	seat lock with spring + nut right clamp
11	4040155000	for cane holder
n/a	4040805000	cane holder complete
12	4040180000	tip-assist right with anti-slip pad
12	4040181000	tip-assist left with anti-slip pad

# BG1 – Frame

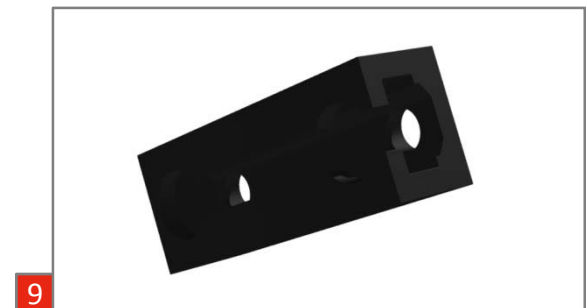

# BG1 – Frame

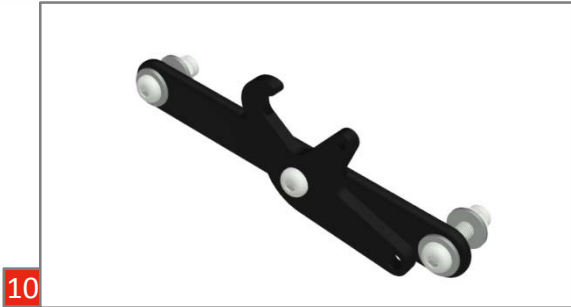

# BG4 - front wheel



object list - BG4 - front wheel		
object	AN	description
1	4040450000	8" PU front wheel with bearings
2	4040401000	Spacer 6mm for profile tire
3	4040402000	axle bolt for front wheel M8x75 + nut + spacer
4	4040410000	front fork with bearings and steering bolt

# BG4 - front wheel




# BG5 - rear wheel



object list - BG5 - rear wheel		
object	AN	description
1	404050000	8" PU rear wheel with bearings
2	404050200	axle bolt for rear wheel M8x80
3	404050500	plastic cover rear wheel
4	404058000	Profile tire
5	404058100	refitting set for profile tire


# BG5 - rear wheel

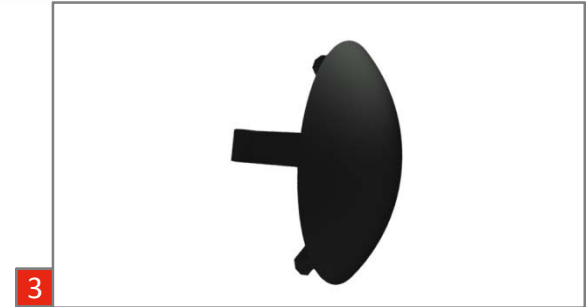





# BG6 - brake

Enjoy mobility.

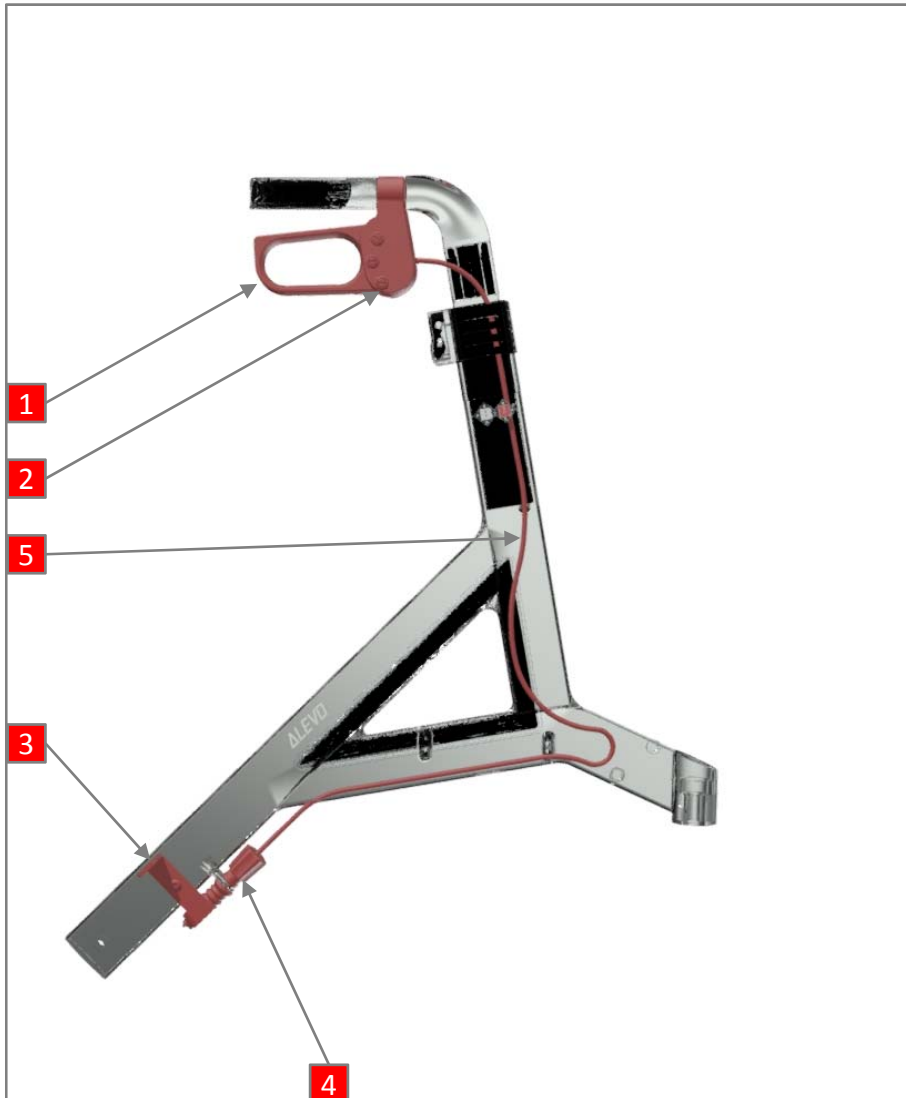

object list - BG6 - brake		
object	AN	description
1	4040622000	brake housing and lever right
1	4040623000	brake housing and lever left
2	4040624000	screw for brake lever
3	4040625000	brake sheet left complete
3	4040628000	brake sheet right complete
4	4040626000	brake parts set
5	4040627000	brake cable complete

# BG6 - brake

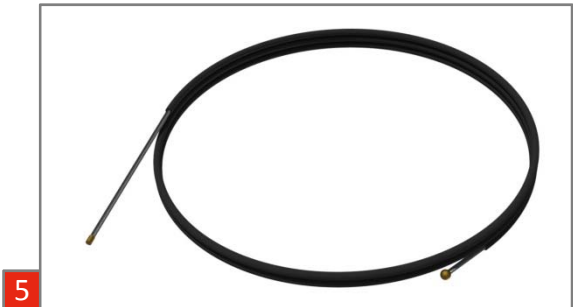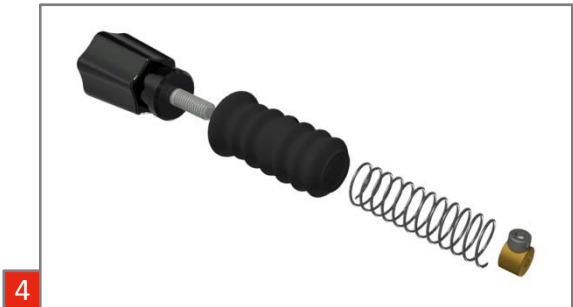


# BG9 - Seat and Backrest



object list - BG9 - Seat and Backrest		
object	AN	description
1	4040902000	backrest with star-grip screws
2	4040903000	star-grip screw for backrest
3	4040905000	seat with strap and seat tube protector

# BG9 - Seat and Backrest

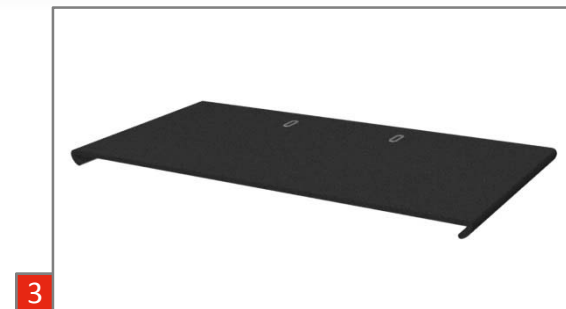



### Stückliste - BG8 - Zubehör

object	AN	description
1	4040801000	Bottle holder
2	4040803000	Umbrella
3	4040805000	Cane holder

# accessories

