Bedienungsanleitung
User manual
Manual de instrucciones
Mode d'emploi

TS-1


DE

EN

ES

FR

Enjoy mobility.

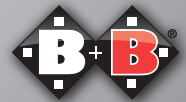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Index

1.	Preface	23
1.1.	Signs and symbols	23
2.	Important safety instructions	24
2.1.	Indications	24
2.2.	Contraindications	25
2.3.	Service life	25
2.4.	Purpose	25
2.5.	Declaration of Conformity	25
2.6.	Responsibility	25
3.	Description of product and contents	26
3.1.	Inspection on delivery	26
3.2.	Contents	27
3.3.	Type plate and serial number	28
3.4.	Summary	29
4.	Assembly and adjustments	30
4.1.	Attaching the wheels	30
4.2.	Fitting the backrest	30
4.3.	Arm rests	30
4.4.	Foot rests	31
4.5.	Seat	32
4.6.	Brakes	32
5.	Operation	33
5.1.	Getting in and out	33
5.2.	Moving around	35
5.3.	Using the commode	35
6.	Transport	35
7.	Storage and shipping	36
8.	Repairs	36
9.	Technical data	36
10.	Care instructions for the user	37
11.	Maintenance	38
12.	Disposal	39
13.	Transferral of the commode chair to another user	39
14.	Warranty certificate	39

1. Preface

Dear user,

Having purchased this Bischoff & Bischoff commode chair, you now have a top German product that will simplify personal hygiene at home.

This instruction manual will familiarise you with the operation and use of your Bischoff & Bischoff commode chair. Please read the instruction manual carefully before using the product for the first time.

We hope you enjoy your new product!

1.1. Signs and symbols

Important! Indicates particularly useful information in the specific context.

Caution! Indicates information that is specifically relevant to safety. Follow the operating manual!

2. Important safety instructions

- Max. user weight TS-1: 120 kg.
- No overcoming of obstacles! No tilting by pressing the backrest.
- Only ever push it on a flat and horizontal surface.
- When stationary, always apply both wheel brakes.
- Be aware of the risk of becoming trapped when setting up and folding the seat.
- The seat is only for sitting.
- Avoid leaning out too far to the side. Risk of tipping!
- Make sure that the backrest screws are secure.
- The commode chair must not be used by unauthorised persons (particularly children)

Reporting incidents:

If you, as the operator, user or their relatives, detect possible defects or functional limitations, please contact an authorised dealer immediately. Operators, users or their relatives should inform the dealer who supplied the dealer who provided you with the product of any suspected serious incidents which directly or indirectly have had or could have one of the following and by which they are affected. You can also report this to the competent higher federal authority report

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of the health of the state of health of a patient, user or other persons,
- a serious risk to public health.

2.1. Indications

Walking disability or serious difficulty in walking of adult persons as a result of:

- paralysis
- loss of limb
- defect or deformity in limbs
- joint contractures or joint injury
- other conditions

•

2.2. Contraindications

The commode chair is inappropriate for adult persons:

- with reduced or restricted sitting ability
- with serious cognitive restrictions

2.3. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life of the commode chair is up to 5 years. The commode chair can continue to be used beyond this period, provided the product is in a safe condition.

2.4. Purpose

The use of a commode chair is necessary if the user is unable to use or reach a toilet because they are handicapped.

The maximum user weight of 120 kg for the TS-1 must not be exceeded.

2.5. Declaration of Conformity

As the manufacturer of the product Bischoff & Bischoff declares TS-1 meet the requirements of Directive MDR 2017/745 in full.

2.6. Responsibility

We are able to provide warranty cover only if: the product is used in accordance with the specified conditions and for the intended purposes, if modifications, extensions, repairs, and maintenance work are carried out only by persons authorised to do so by us and if the commode chair is used in accordance with all instructions for use.

3. Description of product and contents

3.1. Inspection on delivery

All Bischoff & Bischoff products are inspected before they leave our factory and the CE mark is applied.

The Bischoff & Bischoff commode chairs are delivered in a special box. The product is delivered and instruction in its use is given by a qualified specialized dealer.

If the product is delivered by rail or road transport it must be inspected for shipping damage in the presence of the delivery person.

If any damage is found, please note the following:

- record the facts
- prepare a declaration of assignment; all claims resulting from the damage are assigned to the delivery company
- return the record of facts, consignment note and declaration of assignment to Bischoff & Bischoff GmbH.

In the event of shipping damage...

- Leave the goods and packaging undisturbed and do not use the goods.
- Report the damage to the transport company in writing as instructed below.
- Contact our customer service.
- A representative of the transport company must be informed of any damage without delay in order to make a claim for compensation.

Externally obvious damage or losses must be confirmed in writing on consignment note by the delivery person before accepting the package.

Concealed damage that is not obvious until the package is opened must be reported immediately in writing to the transport company.

All deadlines must be met:

- courier service and post within 24 hours
- rail delivery within 7 days
- road transport within 4 days

after receipt of the package.

Bischoff & Bischoff GmbH will require the original consignment note with confirmation of damage.

Compensation from the insurer is not possible without these documents.


3.2. Contents

The scope of supply in the special box includes:

- Assembly components commode chair
- Commode container
- Seat cushion
- Instruction manual

3.3.Type plate and serial number

The type plate (see fig. 01) and serial number are located on the frame.

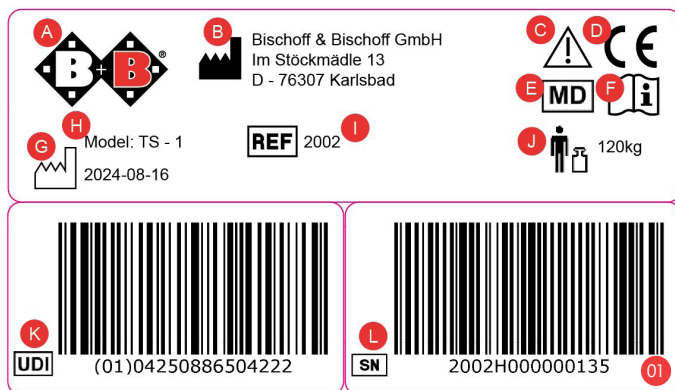- (A) Manufacturer's logo
- (B) Manufacturer's information
- (C) Caution! - Follow instructions for use
- (D) CE mark
- (E) Important! - Follow instructions for use
- (F) Medical device
- (G) Model designation
- (H) Model number
- (I) Date of manufacture
- (J) Maximum load capacity
- (K) UDI item number
- (L) Serial number

3.4. Summary

To make it as easy as possible to handle your Bischoff & Bischoff commode chair, we have documented the written information with detailed illustrations in addition. The following illustrations should help clarify the names and interactions of the individual components (see figs. 1).

4. Assembly and adjustments

Your new Bischoff & Bischoff commode chair is delivered in a Bischoff & Bischoff original box. Attached parts are supplied separately to prevent damage during transport.

You can carry out the adjustments of your commode chair yourself with a little manual dexterity. If you have any queries, please contact your specialized dealer.

4.1. Attaching the wheels


Insert the wheel into the guide tube as far as it will go. Make sure that the hole in the guide tube and the hole in the wheel's quick-release axle are aligned (Fig. 02).

Then insert the locking pin into the guide tube from the outside inwards to secure the wheel's quick-release axle (Fig. 03).

4.2. Fitting the backrest

- Push the backrest into the frame tubes on the back, and fix it in place using the hand screws (see Fig. 04).
- To remove the backrest, undo the hand screws, unlock the armrests (see Fig. 03) and pull the backrest up and out of the fixtures.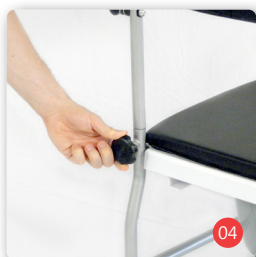

Make sure that the tubes are completely plugged into each other, otherwise the locking can loosen when leaning against the backrest!

4.3. Arm rests

The armrests of TS-1 can be swivelled downwards in order to allow sideways transfer and make it easier to use the chair.

- Unlock the armrests by pulling out the pin (see Fig. 05). The armrests can then be swivelled downwards.
- When swivelling the armrests upwards, make sure that you hear them audibly lock into place.

The armrests of TS-1 SL can be removed by pulling out the pin and pulling the armrest up and out of the frame tubes.

4.4. Foot rests

The footrests can be fitted and removed in just a few steps.

Attaching the leg rests

Hang the leg rests on the guide pins and then fold them inwards (Fig. 06).

Folding up the foot plates

The foot plates (Fig. 07) can be folded upwards by simply lifting them. This can be useful when storing the TS-1 in a restricted space.

Foot rest height adjustment

The foot rests can be adjusted in height according to the individual length of the lower leg of the user.

- Detach the nut cover
- Undo the hexagon nut M8 (Fig. 08).
- Slide the foot rests to the desired position.
- A suitable height of the foot rests is achieved when the user's thighs rest flush with the seat with the feet on the foot rests.
- Re-tighten the hexagon nut and check that the foot rests are securely in position. Re-attach the nut cover.
- Repeat the procedure on the opposite side.

TS-1 | Commode chair

Folding away and removing the foot rests

To facilitate getting in and out the commode chair, the foot rests can be folded away and removed.

- Undo the locking mechanism.
- Swing the foot rests outwards.
- To remove them, first swing the foot rests outwards (Fig. 09); they can then be pulled up out of their mountings.
- Follow these instructions in reverse order to fit the foot rests.

4.5. Seat

Seat

The seat has a non-slip bottom. It can be easily removed before using the toilet.


Commode seat

Removing commode seat (only for TS-1):

- Pull front edge of seat up from the base (Fig 10).
- Lift commode seat upwards.

Positioning commode seat:

- Slide back edge of seat with the open corners to the back.
- Press front rounded edge of seat down to the stop.
- Seat must lock into position.
- Make sure that it is locked.

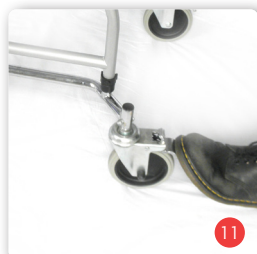4.6. Brakes

The wheels of TS-1 are fitted with locking brakes. To lock the brakes, push down on the overhanging part of the pedal with your foot (Fig. 11). To release the brake, step on the innermost part of the pedal to push it down (Fig. 12).

5. Operation

5.1. Getting in and out

It is easy to get in and out of the TS-1. The foot plate and the two arm rests of TS-1 can be folded out of the way. The armrests of TS-1 SL can be removed.

Getting in and out from the side:

- Move the commode chair as close as possible to the seat on which you are sitting.
- Make sure that the seat is in a secure and stable position.
- Lock the commode chair by setting the parking brakes.
- Fold the arm rest out of the way.
- Then fold the foot plate up.
- Now slide sideways onto the seat of the TS-1.
- Make sure that you are sitting as far back in the seat as possible.
- Now the arm rest and the foot plate can be returned to their normal position.

Getting in and out from the front:

- Move the commode chair as close as possible to the front of the seat.
- Lock the commode chair by setting the parking brake.
- Fold the foot plate up.
- Make sure that the seat is in a secure and stable position.
- Now slide onto the seat of the TS-1 by turning your body around. Make sure that you are sitting as far back in the seat as possible. Now fold the foot plate down and place your feet on it.

5.2. Moving around

The commode chair can then be moved by an assistant. The assistant can move, steer and brake the commode chair using the push handle.

Commode chairs have only limited anti-tip and anti-slip security, so do not use the commode chair to overcome obstacles, only use the commode chair on even ground. Particular care is necessary when braking. Leaning out of the commode chair to an extreme degree increases the risk of tipping over.

5.3. Using the commode

Using the commode

Remove the commode container from the commode chair by pulling it out of the guides from the back.

Usage on the toilet

The commode chair can be used on the toilet without the commode container. First remove the commode container as described above (Fig. 13). Now the TS-1 can be moved over any standard toilet.

Lifting points

You can lift the commode chair by the metal frame in order to carry it over obstacles or stairs. Avoid lifting the commode chair by the seat and backrest.

6. Transport

To make the commode chair as easily transportable as possible, the backrest with armrests can be removed, as can the foot rests.

7. Storage and shipping

If the wheelchair is to be stored or shipped, all removable and unsecured parts must be removed and packed separately. The separately packed parts can then be packed into one larger box. We recommend keeping the original packaging so it is available for this purpose if required. It provides optimum protection for your wheelchair from environmental influences during storage or transport.

8. Repairs

Repairs and maintenance must be done by authorised specialized dealers only. To find a specialized dealer in your area please contact our customer service department at +49 (0)700/6000 7070.

9. Technical data

Dimensions

Width (mm):	550
Length (mm):	880
Height (mm):	990 *
	970 - 1097 **
Seat width (mm):	440
Seat depth (mm):	450
Seat height (mm):	485* (without seat cushion)
	495 to 546** (without seat cushion)
Seat up to armrests (mm):	255 (without seat cushion)
Seat up to backrest (mm):	255 (without seat cushion)
Backrest height (mm):	480

Weights

Max. weight of the user (kg):	120* 160**
Empty weight (kg):	12.5* 16**

Materials

Frame:	Steel powder-coated
Seat and back:	PVC
Flammability:	flame-retardant

Ambient conditions

Temperature:	0 to 50 °C
Air humidity:	20 to 100 %
Heat-Resistant up to	125 °C

Storage conditions

Temperature:	-20 to + 60 °C
Air humidity:	45 to 60 %

Standard equipment for: *TS-1 / **TS-1 SL

The specified dimensions are subject to manufacturing tolerances of +/- 10 mm

10. Care instructions for the user

10.1. Cleaning

Clean the material of the seats and seat backs with warm water and a sponge and/or a soft brush. A standard cleaning agent can also be used for stubborn soiling.

Note: never use aggressive cleaning agents such as solvents, scouring agents or hard brushes.

Plastic parts are best cleaned with a standard plastic cleaner. Please pay particular attention to the product information of the cleaning agent.

10.2. Disinfection

The commode chair can be disinfected using a general-purpose household disinfectant. Use approved spray-disinfectants or alcohol-based wipe-down disinfectants.

A list of the disinfectants approved by the Robert Koch Institute can be found at www.rki.de

Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your specialized dealer will be happy to provide you with these.

Your commode chair is made of high-quality materials for a long life of comfortable use. Every product is thoroughly checked before it leaves our premises. Nevertheless it is possible a defect may arise. If a defect arises with your commode chair, do not continue to use it. With regard to repairs, contact the specialized dealer who supplied the product to you. He will be pleased to assist you. We further recommend that the commode chair is serviced at least once a year. Your specialized dealer receives a servicing schedule from us, together with the documentation required for inspections. Our Customer Service department will be pleased to answer any questions you may have with regard to process, and to offer advice and information on the free-phone number.

You must have your commode chair regularly checked in accordance with the service schedule, and verified as being in good condition.

12. Disposal

If your commode chair is no longer being used and is to be disposed of, please contact your specialized dealer.

13. Transferral of the commode chair to another user

The commode chair is suitable for re-use. Before it is transferred to another user it should be serviced by the mobility dealer and

If you wish to dispose of the product yourself, contact local recycling companies to enquire about the regulations that apply to your place of residence.

1. For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from production and material defects. Where it can be demonstrated that there is a material or manufacturing defect, defective parts will be replaced free of charge.

2. Claims from this warranty shall not arise, if a repair or replacement of a product or a part is required for the following reasons:

- a) normal wear and tear, which include but is not limited to the following parts where fitted: Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.
- b) overloading the product such as, for example, by exceeding the maximum user weight or load.
- c) the product or part has not been maintained or serviced in accordance with the operating manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.
- d) Accessories have been used which are not original accessories.
- e) The product or part has been damaged through negligence, accident or incorrect use.
- f) changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
- g) Repairs have been carried out, before our customer service has been informed of the circumstances.

3. To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.

4. For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1).
5. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.
6. If the rectification of defects fails after a reasonable period of time, the purchaser may, at its discretion, demand a reduction of the remuneration or withdraw from the contract.
7. The warranty is subject to the law of the country in which the Bischoff & Bischoff product was purchased.

In addition, please observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your specialist dealer will be happy to provide you with these.

Warning! Non-observance of the instruction manual, improper maintenance and in particular any technical changes and supplements (added fittings) without the approval of Bischoff & Bischoff will generally lead to forfeiture of warranty and product liability claims.

Warranty certificate

Product: TS-1

Serial number:*

* (to be entered by dealer)

Dealer:

Date and stamp

(Please note our general business terms and conditions)

*See the side frame for information stickers

Note!

Despite careful research and editing, incorrect information may slip into our operating manuals. Subject to technical changes. Figures may contain accessories. The most up-to-date version of the operating manual is available for download from our website:

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